

Product Catalogue 2024 Orthoses



**ORTHOSERVICE
RO+TEN**

Take care feel better

“

Although we consider our formation a happy and lucky event, we founded and manage our group with a sense of purpose. We pursue our objectives everyday with trust, perseverance and competence.”

*Pamela Pirovano and Paolo Rossi,
Founders, Swiss Headquarters*



Scan the qr code
and discover more about
who we are and
our history



Content

ORTHOSES

Figurative index..... 6

NECK 11

Soft collars 12
Semirigid collars 13
Rigid collars 13

BACK 15

Shoulder posture correctors..... 16
Support bands..... 16
Elastic orthopaedic orthoses 17
Semirigid orthopaedic orthoses 22
Rigid dorsolumbar braces..... 29
Hyperextension braces..... 30

SHOULDER 35

Functional shoulder braces 36
Clavicle splints 37
Acromioclavicular immobilisers 37
Internal rotation and 0°
abduction immobilisers..... 38
10°->20° abduction immobilisers..... 39
≥ 20° abduction immobilisers 41
External rotation immobilisers 42

ARM 43

Elasticated elbow braces 44
Rigid elbow braces 45
Humeral braces..... 46

FOREARM / WRIST 47

Arm slings 48
Epicondylitis straps 48
Elasticated fabric wrist supports..... 50
Wrist-hand immobilisers 51
Wrist immobilisers 51

HAND 53

HIP / THIGH 57

Thigh supports..... 58
Iliac supports..... 58
Hip orthoses 59

KNEE 61

Patella straps..... 62
Patella-stabilising knee supports 62
Knee orthoses 63
Knee supports with spiral stays 67
Knee orthoses
with articulated hinges 69
Knee immobilisers..... 75
Postoperative braces 76
Functional knee braces 78
Orthoses for gonarthrosis 80

LOWER LEG / ANKLE 83

Calf supports..... 84
Ankle supports 84
Ankle supports with figure-8 straps... 86
Ankle braces
with two support shells..... 88
Night-time positioning splints 88
Rigid walker boots 89
Walker boots with range of motion.... 93
Ankle foot orthoses..... 93

FOOT 95

Suspension boots..... 96
Heel cups and inserts 99
Hallux valgus retractors 100
Finished insoles 101
Post-op shoes 103

TRACTIONS AND OTHER 105

ORTHO 14 - PAEDIATRIC LINE 109

ORTHOSELF LINE..... 115

General terms and condition of sales... 118

A single company system with many production and management sites



RO+TEN S.r.l.
1978 - Verano Brianza, Italy



ORTHOERVICE AG
1996 - Chiasso, Switzerland



ORTOVENETA
2007 - Maser, Italy



ICS ORTHOSERVICE
2010 - Balti, Moldova



ORTHOERVICE DEUTSCHLAND GmbH
2011 - Baden-Baden, Germany



ORTHOERVICE POLSKA Sp. z o.o.
2021 - Częstochowa, Poland



ORTHOERVICE RO+TEN GROUP

- Servicing the orthoses sector since 1978
- Over 130 employees
- Accredited patents
- Export to 30 countries



OSUK Ltd
2024 - Sunderland, UK





Orthoses

ORTHOSERVICE
RD+TEN
Take care! Keep calm!

The presence of threaded holes and inserts on the ortho-
plex
associated with the
screws that are used
for the orthopaedic
tools to apply the inserts.
Other locking elements of the inserts disadvantageously
increase the size of the articulated rod which should
be small and as compact as possible.
The orthopaedic device
for adjusting an extension-flexion rotation
of the articulated rod is compact, stable and small.

NECK

page 11

SOFT COLLARS	SEMIRIGID COLLARS	RIGID COLLARS

cervilight **cervilight soft** **cervilight soft** **cervistable** **malibù** **philastab** **minerva**

BACK

page 15

SHOULDER POSTURE CORRECTORS	SUPPORT BANDS	ELASTIC ORTHOPAEDIC ORTHOSES

dorsolite **elastop** **elastop** **easyco basic** **easyco plus** **litecross93** **litecross90** **litecross91**

ELASTIC ORTHOPAEDIC ORTHOSES					SEMIRIGID ORTHOPAEDIC ORTHOSES			
lumboSKILL70	lumboSKILL72	lumboFIT70	lumboFIT72	crisscross78	flex-B	sat 1325	sat 1323	sat 1322 / 1322SH / 1354

SEMIRIGID ORTHOPAEDIC ORTHOSES						RIGID DORSOLUMBAR BRACES		
sat 1321	sat 1336	sat 1335	sat70	lumbofast	lumbopull 70	lumbopull 72	dorsopull 74	spinalplus 2.0

HYPEREXTENSION BRACES							
e-STAB	stabil-T	modular plus35k	modular plus35 acqua	modular plus36	modular plus45	connect	nhyco

SHOULDER

page 35

FUNCTIONAL SHOULDER BRACES	CLAVICLE SPLINTS	ACROMIOCLAVICULAR IMMOBILISERS	INTERNAL ROTATION AND 0° ABDUCTION IMMOBILISERS				
shouldercross	octofix (cvb)	acromion 2.0	shouldfix	shouldfix II	shouldfix open (stb)	shouldfix light	shouldfix 3

10°->20° ABDUCTION IMMOBILISERS					≥ 20° ABDUCTION IMMOBILISERS		EXTERNAL ROTATION IMMOBILISERS
omoplus 2.0	top-S 10°/15°	top II-S 10°/15°	top III-S 10°/15°	top 4-S	softab 20°	softab 45°/70°	extra-S 15°/30°

ARM

ELASTICATED ELBOW BRACES RIGID ELBOW BRACES HUMERAL BRACES









epilastik 32 **epilastik PLUS** **epiFIT 32** **elbo PRO** **elbo** **castop** **castop**

page 43

FOREARM WRIST

ARM SLINGS EPICONDYLITIS STRAPS ELASTICATED FABRIC WRIST SUPPORT










sling-up 2.0 **epifeel** **epi020** **epi320** **object 300** **object 600** **manulastik 19** **manulastik PLUS**

page 47

WRIST-HAND IMMOBILISERS

WRIST IMMOBILISERS








carpo 41 **manulite** **polFIT 19** **polFIT 17** **polFIT 21** **polFIT 22**

HAND

HAND





daumfix **rizostab** **stax**

page 53

HIP / THIGH

THIGH SUPPORTS ILIAC SUPPORTS HIP ORTHOSES










mioFIT 33 **object 33** **symplus** **hipoCROSS 2.0** **Rod for hipoCROSS 2.0** **hipo** **kafo** **kinesio**

page 57

KNEE

PATELLA STRAPS PATELLA-STABILISING KNEE SUPPORTS KNEE ORTHOSES










ice **object 10** **genuFIT 16** **genu416** **genuSKILL 04** **genuFIT 60** **genuFIT 63** **genuFIT 04**

page 61

KNEE ORTHOSES KNEE SUPPORTS WITH SPIRAL STAYS










object 60 **object 63** **object 04** **genuLASTIK 04** **genuLASTIK PLUS** **genuSKILL 08/08A** **genuFIT 08/08A** **object 08**

KNEE ORTHOSES WITH ARTICULATED HINGES



genuSKILL
15/15A



genuSKILL
26/26A



genuSKILL 28A



genuFIT
15/15A



genuFIT
29/29A



genuFIT
25/25A



genuFIT
30/30A



genuFIT
27/27A



object 15

KNEE IMMOBILISERS



object 20



immok
0°/20°



go up basic



go up PRO



go up open



go up +
go up REHAB



go up T



go up pcl

POSTOPERATIVE BRACES

FUNCTIONAL KNEE BRACES



pluspoint
FAST



pluspoint 4



pluspoint 3



pluspoint
3 short



genuFEEL OA



KOA



ORTHO-A

ORTHOSES FOR GONARTHROSIS

LOWER LEG / ANKLE
page 83

CALF SUPPORTS



mioFIT 34



malleoLASTIK 37



malleoLASTIK
PLUS

ANKLE SUPPORTS



malleoFIT 37



object 37



object 80

ANKLE SUPPORTS WITH FIGURE-8 STRAPS



malleoLITE



malleoFIT 81

ANKLE SUPPORTS WITH FIGURE-8 STRAPS



object 81



OSC active



malleorapid



malleostrong
PRO

ANKLE BRACES WITH TWO SUPPORT SHELLS



airstrong

NIGHT-TIME POSITIONING SPLINTS



malleonite

RIGID WALKER BOOTS



equal400



equal450



airstep walker

RIGID WALKER BOOTS



airstep walker
short



airstep tight
walker diabetic



airstep walker
diabetic



airstep tight
walker



airstep tight
walker short



airstep
diabetic set

WALKER BOOTS WITH RANGE OF MOTION



r.o.m walker

ANKLE FOOT ORTHOSES



carbon afo



afo light

ANKLE FOOT ORTHOSES



afo
extrastrong



affix



affix
OPTION

FOOT

page 95

SUSPENSION BOOTS



heelifit classic



heelifit glide



heelifit afo



heelifit traction

HEEL CUPS AND INSERTS



talufit 50



talufit 52



talusoft 04

HALLUX VALGUS RETRACTORS



hallus valgus



havar



podifer 20



podifer 30



podifer 40



podifer 50



podifer 60



stepper
SURG



stepper
DIGIT



stepper
CAL

FINISHED INSOLES

POST-OP SHOES

TRACTIONS AND OTHER

page 105


TRACTIONS AND OTHERS



cervitrac



trac option

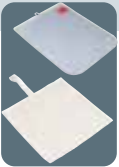

sacca
sacchetto



duraband



bi-bike

ORTHO14

page 109

ORTHO 14



ORTHO 14
100



ORTHO 14
105



ORTHO 14
305



ORTHO 14
310



ORTHO 14
315



ORTHO 14
500



ORTHO 14
502



ORTHO 14
702/3

ORTHO 14



ORTHO 14
800



ORTHO 14
805



ORTHO 14
810

ORTHOSELF

page 115

ORTHOSELF



EPI-S32



MANU-S19



MIO-S33



GENU-S60



MIO-S34



MALLEO-S37





NECK



cervilight

Foam cervical collar

cervilight M8300 | M8301



REINFORCED

INDICATIONS

- Slight distortions of cervical spine (whiplash)
- Torticollis
- Cervical vertigo
- Arthritis of cervical spine



FEATURES

- Semi-rigid foam structure
- Covered with tubular jersey
- External band reinforcement made of non-toxic leather-like material
- Back Velcro® fastening system
- 100% washable (warm water (30°C) and neutral soap)
- Radiolucent

Item	M8300 • M8301	
Size	universal	
Height cm	7,5	10
Colour	white	

cervilight soft M8302



INDICATIONS

- Slight distortions of cervical spine (whiplash)
- Torticollis
- Cervical vertigo
- Arthritis of cervical spine



FEATURES

- Semi-rigid foam structure
- Covered with tubular jersey
- Back Velcro® fastening system
- 100% washable (warm water (30°C) and neutral soap)
- Radiolucent

Item	M8302			
Size	S	M	L	XL
Neck circ. cm	30/35	36/41	42/47	48/53
Height cm	6-8-10	6-8-10	6-8-10	6-8-10
Colour	white			

cervilight soft M8304



SHAPED

INDICATIONS

- Slight distortions of cervical spine (whiplash)
- Torticollis
- Cervical vertigo
- Arthritis of cervical spine



FEATURES

- Semi-rigid foam structure with rounded edges
- Covered with tubular jersey
- Back Velcro® fastening system
- 100% washable (warm water (30°C) and neutral soap)
- Radiolucent
- Perfect anatomical shape

Item	M8304											
Size	S	S	S	S	M	M	M	M	L	L	L	L
Neck circ. cm	28/34	29/35	30/36	31/37	30/36	31/37	32/38	33/39	36/44	37/46	38/48	39/49
Length cm	47	49	50	52	50	52	54	56	58	60	62	64
Height cm	6	8	10	12	6	8	10	12	6	8	10	12
Colour	blue											

cervistrong


Reinforced neck brace

M20304



INDICATIONS

- Mild or moderate distortions of the cervical spine (whiplash).
- Neck-induced vertigo
- Cervical osteoarthritis
- Moderate discopathies with associated cervicobrachial pain syndrome

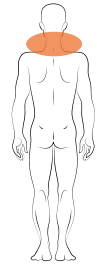

FEATURES

- Core made of semi-rigid foam ((with rounded edges) reinforced with polyethylene inside the brace
- Tubular jersey cover
- Velcro® fastening at back of neck
- Fully washable
- X-ray transparent
- Anatomically shaped design

Item	M20304			
Size	S	M	L	XL
Neck circ. cm	30/35	36/41	42/47	48/53
Height cm	6-8-10-12	6-8-10-12	6-8-10-12	6-8-10-12
Colour	grey			

cervilight schanz

Cervical collar, schanz type with adjustable Velcro® straps


M1505



ADJUSTABLE HEIGHT

INDICATIONS

- Moderate to severe distortions of cervical spine (whiplash)
- Cervical vertigo
- Arthritis of cervical spine
- Serious disc disease or slipped discs associated with cervicobrachial symptoms
- Use after cervical spine surgery

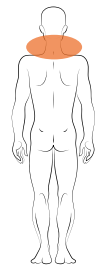

FEATURES

- Rigid plastic structure perforated for ventilation
- Foam edges covered with non-toxic leather-like
- Back Velcro® fastening system
- Height can be adjusted with Velcro® system
- Radiolucent

Item	M1505			
Size	S	SL	M	L
Neck circ. cm	30/35	35/40	35/40	40/45
Front height cm	9/12	9/12	10/14	10/14
Colour	white			

Neck > Rigid collars

cervistable

Bivalve cervical collar with open trachea

M1530/T



INDICATIONS

- Severe to serious distortions of cervical spine (whiplash)
- Multiple disc diseases of cervical spine
- Cervical trauma
- Arthritis of cervical spine
- Pathological failure due to cervical metastasis
- Use after cervical spine surgery



FEATURES

- Structure made of wrapping bivalve with Back Velcro® fastening system
- Structure made of non-cytotoxic polymeric foam in compliance with UNI EN ISO 10993-5
- Rigid plates for back, front and central support
- Radiolucent and waterproof
- Shaped for a perfect adhesion to chin, neck and shoulders
- Open trachea and holes for ventilation
- Prepared to be connected with chest stabiliser or hyperextension

Item	PRO-1530/T			
Size	S	M	L	
Neck circ. cm	30/35	35/40	40/45	
Height* cm	2 1/4" = 6 cm	3 1/4" = 8 cm	4 1/4" = 11 cm	5 1/4" = 13 cm
Colour	beige			

* Vertical distance between upper shoulder and chin

OPTION: MIP35/PC Connect, connecting plate to ModularPlus or e-STAB

OPTION: M1522H head band

malibù

Bivalve cervical brace

M3005



INDICATIONS

- Severe to serious distortions of cervical spine (whiplash)
- Multiple disc diseases of cervical spine
- Cervical trauma
- Pathological failure due to cervical metastasis
- Use after cervical spine surgery



FEATURES

- KYDEX® plastic brace
- Covered with PLASTAZOTE® pad
- Velcro® fastening system
- Possible manual adjustment through screws and small knob
- Open trachea
- Radiolucent
- Anatomic design

Item	M3005
Size	universal
Altezza cm	adjustable
Colour	grey

philastab

Dorsal-sternum stabiliser for Cervistable collar

M1521



INDICATIONS

- Cervical and dorsal fractures
- Cervical spondylolisthesis
- Multiple discopathy
- Cervical and breastbone fractures due to osteoporosis or metastasis
- Use after spine surgery and as an immobiliser in the event of first aid



FEATURES

- PVC struts
- PLASTAZOTE® cover
- Connections with fixing buttons to Cervistable collar
- Velcro® fastening system - lower part
- Radiolucent

Item	M1521
Size	universal
Colour	beige

minerva


Bivalve cervico-dorsal brace

M3002



INDICATIONS


- Cervical and dorsal fractures
- Cervical spondylolisthesis
- Multiple discopathy
- Cervical and breastbone fractures due to osteoporosis or metastasis
- Use after spine surgery and as an immobiliser in the event of first aid

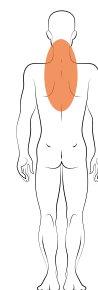

FEATURES

- KYDEX® brace with PLASTAZOTE® pad
- Open trachea
- Velcro® fastening system
- Light and easy to wear
- Radiolucent
- Heat mouldable

Item	M3002		
Size	S	M	L
Neck circ. cm	25/30	31/38	39/45
Height cm	33	38	45
Colour	beige		

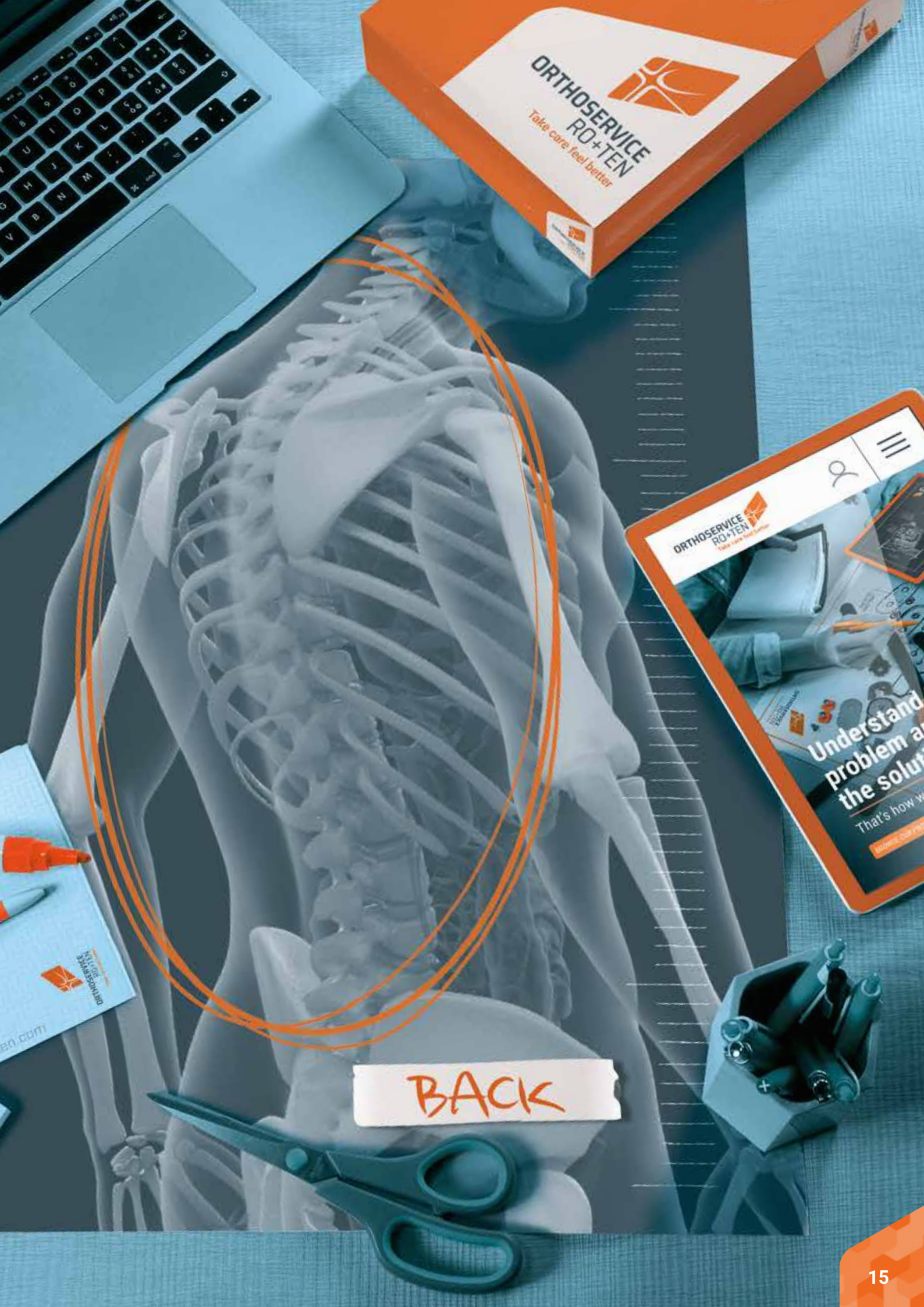

ORTHOSERVICE
RO+TEN
Take care feel better

ORTHOSERVICE
RO+TEN
Take care feel better

Understand
problem and
the solution

That's how we...

BACK

Back > Shoulder posture correctors

dorsolite

Back orthopaedic brace

M91111



INDICATIONS

- Useful for the correction of a curved back, kyphosis and scapular winging
- A useful aid for back pain caused by poor posture



FEATURES

- The support and straps are made of PE foam in a bi-layer of velveteen and technical material a back pulley system provides better traction when fastening
- The straps can be adjusted and shortened using the Y-Velcro® straps to adjust overall length and shoulder straps position
- Fastens with a snap-fit buckle
- Easy to put on, just like a backpack

Item	M91111
Size	universal
Colour	cream

Back > Support bands

elastop

Post-operative thoracic band support

elastop

M55113



INDICATIONS

- Thoracic contusions
- Post-operative hospital care



elastop

Post-operative lumbar band

elastop

M55114



INDICATIONS

- Post-operative and post-natal containment support
- Umbilical hernias
- Eventration
- Abdominal hernias



FEATURES

- **Only M55113:** The two elasticated bands allow the belt to adapt and fit well onto the body
- **Only M55114:** The three elastic bands facilitate a good fit which easily adapts to the body
- Breathable
- Front Velcro® fastening system

Item	M55113					
Size	XS	S	M	L	XL	XXL
Chest circumf cm	<100	100/110	110/130	130/140	140/150	150/160
Colour	white					

easyco

Ribbed elastic orthosis

easyco basic

M23770B
M23770S



INDICATIONS

- Paravertebral muscle contractures
- Mild traumas
- Mild lumbago



easyco plus

M23870B
M23870S





INDICATIONS

- Paravertebral muscle contractures
- Mild traumas
- Mild lumbago
- Mild sciatic lumbalgia and cruralgia
- Post-operative recovery



FEATURES

- Structure made of ribbed elastomer fabric
- 4 lightweight and flexible rods on the back (width 20mm, height 20/22cm)
- Front stays fixed in an oblique position
- Front Velcro® closure
- **Only Easyco basic:** Reinforced elastic back
- **Only Easyco plus:** Criss-crossed external elastic tie-rods as a reinforcement, with a Velcro® fastening system

Item	Easyco basic M23770B • M23770S Easyco plus M23870B • M23870S					
Size	XS	S	M	L	XL	XXL
Waist circumf. cm	50/60	60/75	75/90	90/105	105/120	120/135
Colour	beige / black					
	 M23770B M23870B					 M23770S M23870S

litexcross

Elastic orthosis with rigid and thermoformable pads to stabilise the lumbar rachis and relieve pressure

litexcross93

M1093C
M1093S



LOWERED

INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery



litexcross90

M1090C
M1090S



LOW

INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery
- Results of vertebral compressions of the lumbar spine
- Treatment of transverse process fractures



litexcross91

M1091C
M1091S



HIGH

INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery
- Results of vertebral compressions of the lumbar spine
- Treatment of transverse process fractures



FEATURES

- Structure in elastic Sensitive® fabric, breathable, laminated, rip proof, and antistatic
- Adjustable pulley system with elastic straps for custom fit support
- Velcro® front fastening system
- Spiral stays fixed in an oblique position reinforce the abdominal section
- Adapts to the pathology of the patient
- 2 front and back thermoformable pads, on medical prescription
- Optional rigid stays, mouldable on medical prescription; to insert in the 4 linear back pocket

Item	Litexcross93 M1093C · M1093S Litexcross90 M1090C · M1090S Litexcross91 M1091C · M1091S						
Size	XS	S	M	L	XL	XXL	XXXL*
Pelvic circ. cm	70/80	80/90	90/100	100/110	110/120	120/130	130/140
Colour	M1093C cream · M1093S black / *XXXL only black lowered - M1090C cream · M1090S black - M1091C cream · M1091S black						



OPTION:

- Litexcross93:** 9ST012/L optional rigid stays cm 16
- Litexcross90:** 9ST012/L optional rigid stays cm 22
- Litexcross91:** 9ST012/L optional rigid stays cm 26



Do you want to be always updated on the latest news from the Orthoservice Ro+Ten world?

Scan the QR code and visit the news section



lumboskill

Elastic orthosis with rigid and thermoformable pads to stabilise the lumbar rachis and relieve pressure

lumboskill70 M33170



INDICATIONS

- Lumbago
- Sciatic lumbalgia and cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery



lumboskill72 M33172



INDICATIONS

- Lumbago
- Lumbar sciatica and cruralgia
- Spondyloarthrosis and lumbar discopathy
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Postoperative recovery
- Results of vertebral compressions of the lumbar spine
- Treatment of transverse process fractures



FEATURES

- The graduated design of the elastic compression fabric on the front and on the sides provides more compression in the middle and a looser fit around the edges; made of polyester and elastomer
- The back is made of BreathSkill fabric:
 - soft and breathable
 - 3D weaving
 - special laminated covering for extra resistance
- crossed strengthening straps provide variable and customisable compression
- The abdominal section is reinforced with oblique spiral stays
- Front Velcro® fastening system adapts to the pathology of the patient
- 2 front and back thermoformable pads, on medical prescription
- Optional rigid stays, mouldable on medical prescription; to insert in the 4 linear back pocket

Item	Lumboskill70 M33170 Lumboskill72 M33172					
Size	XS	S	M	L	XL	XXL
Waist circ. cm	50/60	60/75	75/90	90/105	105/120	120/135
Colour	grey					

OPTION:

- Lumboskill70:** 9ST012/L optional rigid stays cm 20/22
- Lumboskill72:** 9ST012/L optional rigid stays cm 24/26



How to don
and wear our products
in a few easy steps

Scan the
QR code and view
our video gallery



lumbofit

Elastic orthosis with rigid and thermoformable pads to stabilise the lumbar rachis and relieve pressure

lumbofit70 MT1170



INDICATIONS

- Lumbago
- Lumbar sciatica and cruralgia
- Spondyloarthritis and lumbar discopathy
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Postoperative recovery



lumbofit72 MT1172



INDICATIONS

- Lumbago
- Lumbar sciatica and cruralgia
- Spondyloarthritis and lumbar discopathy
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Postoperative recovery
- Results of vertebral compressions of the lumbar spine
- Treatment of transverse process fractures



FEATURES

- Separate, large-scale tension straps provide support and can be individually adjusted
- Front and sides made of viscose staple fibre and elastomer material
- Back part made of breathable Aer-OS fabric: skin-friendly, non-cytotoxic, latex free
- Adapts to the pathology of the patient
- The abdominal section is reinforced by spiral stays fixed in an oblique position
- Velcro® front fastening system
- 2 front and back thermoformable pads, on medical prescription
- Optional rigid stays, mouldable on medical prescription; to insert in the 4 linear back pocket

Item	Lumbofit70 MT1170 • Lumbofit72 MT1172					
Size	XS	S	M	L	XL	XXL
Waist circ. cm	50/60	60/75	75/90	90/105	105/120	120/135
Colour	black					

OPTION:

LumboFIT70: 9ST012/L optional rigid stays cm 22

LumboFIT72: 9ST012/L optional rigid stays cm 26

crisscross78

Elastic orthosis with rigid and thermoformable pads to stabilise the lumbar rachis and relieve pressure

M1078 | M1078N



INDICATIONS

- Backache
- Minor injuries to the lumbosacral spine and paravertebral lumbar contractures
- Spondyloarthrosis and lumbar discopathy
- Postoperative recovery



FEATURES

- Cotton and elastane fabric
- Front Velcro® fastening system back reinforced by crossed straps
- Rear crossover support
- Adapts to the pathology of the patient
- 2 front and back thermoformable pads, on medical prescription
- Optional rigid stays, mouldable on medical prescription; to insert in the 4 linear back pocket

Item	M1078 • M1078N					
Size	XS	S	M	L	XL	XXL
Waist circ. cm	50/60	60/75	75/90	90/105	105/120	120/135
Colour	M1078 beige · M1078N black					



OPTION:

9ST012/L optional rigid stays 22 cm

flex-B

Padded dorso-lumbar orthopaedic orthosis with a reinforced structure

M33009



INDICATIONS

- Back pain caused by arthrosis or kyphosis
- Recovery from fractures of the thoracic and/or upper lumbar vertebrae
- Support for thoracic vertebral compression fracture caused by osteoporosis or metastasis
- Other conditions as prescribed by the doctor



FEATURES

- Made of reinforced CarbonTrace, a stretchy, breathable, ladder proof, antistatic and bacteriostatic fabric reinforced with carbon fibre
- Adjustable padded shoulder straps with non-release Velcro® fastening system
- The elasticated rear panel of the shoulder straps can be positioned at any height
- Abdominal elastic straps provide variable support for a custom fit
- Front Velcro® fastening system
- The abdominal section is reinforced with oblique spiral stays
- Mouldable inner frame in radiolucent aluminium alloy

Item	M33009					
Size	XS	S	M	L	XL	XXL
Pelvic circumf. cm	70/80	80/90	90/100	100/110	110/120	120/130
Colour	grey					

OPTION:

frame M33009/T Size 1-L, this special frame fits sizes XS to M - with XXL back height



sat

Orthosis with rigid and thermoformable pads to stabilise the lumbar rachis and relieve pressure

sat 1325
M1325



ULTRA LOW • TAILORED

sat 1356
M1356



ULTRA LOW • REGULAR

sat 1323
M1323



LOW • TAILORED

sat 1355
M1355



LOW • REGULAR

sat 1322
M1322



HIGH • TAILORED

INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery
- After-effects of stable compression fractures of lumbar vertebrae
- Treatment of fracture of transverse processes

INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operation period
- Results of vertebral lumbar relaxations
- Treatment of fracture of transverse processes
- Degenerative lumbar scoliosis
- Spondylosis with or without vertebral listhesis



sat 1322SH
M1322SH




sat 1354
M1354



sat 1321
M1321



sat 1353
M1353

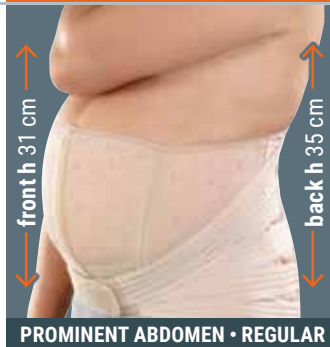

FEATURES

- 98% reinforced anelastic cotton for optimal containment; breathable and natural for best comfort
- Anatomical design shaped for the legs
- Traction system:
 - Reinforced elastic straps for targeted pressure and perfect stabilisation of the lumbar rachis, latex free
 - Pulley system for easy fastening
- Front Velcro® fastening system, with comfortable grip for easy closing. Shaped for maximum comfort
- Optional Rigid stays, moldable on medical prescription, to be inserted in the 4 linear back pockets.
- 2 rigid front and back thermoformable pads, on medical prescription
- Adapts to the pathology of the patient
- **Only Sat 1321 and 1353:** Lateral Velcro® fastening system with pocket.

INDICATIONS

- Lumbalgia
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operation period
- Results of vertebral lumbar relaxations
- Treatment of fracture of transverse processes
- Degenerative lumbar scoliosis
- Spondylosis with or without vertebral listhesis



TAILORED FIT

Item	Sat 1325 M1325 • Sat 1323 M1323 • Sat 1322 M1322 Sat 1322SH M1322SH • Sat 1321 M1321					
Size	XS	S	M	L	XL	XXL
Pelvic circumf cm	75/85	85/95	95/105	105/115	115/125	120/135
Colour	beige					

REGULAR FIT

Item	Sat 1356 M1356 • Sat 1355 M1355 Sat 1354 M1354 • Sat 1353 M1353					
Size	XS	S	M	L	XL	XXL
Pelvic circumf cm	70/80	80/90	90/100	100/110	110/120	120/130
Colour	beige					

OPTION:

- Sat 1325, Sat 1356, Sat 1323 e Sat 1355:**
9ST012/L optional rigid stays cm 26
- Sat 1322, Sat 1322SH, Sat 1354, Sat 1321 e Sat 1353:**
9ST012/L optional rigid stays cm 30

sat

Padded orthosis for stabilising and supporting the dorso-lumbar rachis, with rigid and thermoformable pads

sat 1336
M1336



sat 1335
M1335



INDICATIONS

- Back pain due to arthrosis or kyphosis
- After-effects of fractures of dorsal vertebrae and/or upper lumbar vertebrae
- After-Effects of stable compression fractures of the dorsal spine due to osteoporosis or metastasis
- Other indications on medical prescription



FEATURES

- 98% reinforced anelastic cotton for optimal containment; breathable and natural for best comfort
- Anatomical design shaped for the legs
- Traction system:
 - Reinforced elastic straps for targeted pressure and perfect stabilisation of the lumbar rachis, latex free
 - Pulley system for easy fastening
- Front Velcro® fastening system, with comfortable grip for easy closing. Shaped for maximum comfort
- Optional rigid stays, moldable on medical prescription, to be inserted in the 4 linear back pockets
- 2 rigid front and back thermoformable pads, on medical prescription
- Adapts to the pathology of the patient
- Padded underarm straps

TAILORED FIT

Item	Sat 1336 M1336					
Size	XS	S	M	L	XL	XXL
Pelvic circ. cm	75/85	85/95	95/105	105/115	115/125	125/135
Colour	beige					

REGULAR FIT

Item	Sat 1335 M1335					
Size	XS	S	M	L	XL	XXL
Pelvic circ. cm	70/80	80/90	90/100	100/110	110/120	120/130
Colour	beige					

OPTION

Sat 1336, Sat 1335: 9ST012/L optional rigid stays cm 38



sat70

Orthosis with anterior-posterior traction for stabilising and supporting the dorso-lumbar rachis with a rigid frame

M1370



SPALLACCIO

INDICATIONS

- Back pain caused by arthrosis or kyphosis
- Recovery from fractures of the thoracic and/or upper lumbar vertebrae
- Management of thoracic vertebral compression fractures caused by osteoporosis or metastasis
- Other indications on medical prescription



FEATURES

- Dorsal traction: an innovative reverse system (anterior-posterior) for optimal support
- Lumbar traction: stretch straps provide targeted pressure and perfect stabilisation
- Front Velcro® fastening
- Shoulder straps with underarm cell
- Adapts to the pathology of the patient
- Cotton fabric with minimal elastane content
- Bonded two-layer MTP shoulder straps
- Rigid reinforced stays provided, mouldable on medical prescription; to insert in the 4 linear back pockets

Item	Sat 70 M1370					
Size	XS	S	M	L	XL	XXL
Pelvic circumf. cm	70/80	80/90	90/100	100/110	110/120	120/130
Colour	beige					

lumboFAST

Orthosis with a facilitated traction system to stabilise the lumbar rachis and relieve pressure with rigid thermoformable pads

M23070



PATENT:
registered
community design;
registered design
in CH, UK



INDICATIONS

- Lumbago
- Sciatic lumbalgia and cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Postoperative recovery
- After-effects of stable compression fractures of the lumbar vertebrae
- Treatment of fracture of transverse process



Involved spine section L2-L5

M23070P

PATENT:
registered
community design;
registered design
in CH, UK



ADDITIONAL PADS KIT

INDICATIONS

- Lumbago
- Sciatic lumbalgia and cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas of the lumbosacral column and paravertebral lumbar contractures
- Postoperative recovery
- After-effects of stable compression fractures of the lumbar vertebrae
- Treatment of fracture of transverse processes
- Degenerative lumbar scoliosis
- Spondylosis with or without vertebral listhesis



Involved spine section L1-L5

FEATURES

- Bi-coupled base fabric: a three-dimensional mesh on the inside with an exclusive printed technical fabric on the outside. Rigid for maximum hold and breathable for maximum comfort
- Lanyard and pulley traction system for optimum stabilisation and maximum traction, with minimum effort
- Lanyard terminals have an anatomical and easy grip
- Two sizes only: the rear adjustment system allows you to cut your orthosis to a perfect size
- A rigid thermoformable pad applied with snap buttons provides stability to the column and protects against contact with lanyards
- Polypropylene reinforcement to protect the lanyard
- Front handles for easy Velcro® closure
- Thermoformable front pads to be inserted in the pockets provided
- Anatomical and modern design
- Lightweight
- Option: thermoformable pads kit (large back and sides)

Item	PR1-23070	
Size	1	2
Pelvic circumf. cm	70-105	105-145
Colour	grey	

OPTION:

M23070P Additional pads kit for Lumbofast



To discover
the hidden details
of Lumbofast

Watch the
3D video

Discover
the new 360°
feature



lumbopull

Orthosis with easy-pull tensioning straps for stabilising and supporting the lumbar spine

lumbopull70 M90070



INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery
- After-effects of stable compression fractures of lumbar vertebrae
- Treatment of fractures to transverse processes



lumbopull72 M90072



INDICATIONS

- Lumbago
- Sciatic lumbalgia and lumbar cruralgia
- Spondylarthrosis and lumbar discopathies
- Slight traumas to the lumbosacral column and paravertebral lumbar contractures
- Post-operative recovery
- After-effects of stable compression fractures of lumbar vertebrae
- Treatment of fractures to transverse processes
- Degenerative lumbar scoliosis
- Spondylosis with or without vertebral listhesis



FEATURES

- Compound fabric (3D fabric on the inside, special fabric on the outside): high rigidity to ensure the greatest possible stabilisation; breathable for the greatest possible comfort when wearing
- System of serially arranged pulleys and tensioning straps (exclusive) in order to achieve optimum stabilisation and tensioning with minimum application of strength. The closure multiplies the force exerted by the patient and ensures the even distribution of force along with optimum adaptability to the individual anatomy
- Elastic rear insert (not structural) in order to prevent the back coming into contact with the straps and to apply various optional items using Velcro® fastener
- Anatomically shaped and easy to put on
- Slim strap on the rear (h = 25 mm)
- Large adhesive surface for maximum setting options
- Paravertebral brace reinforcements
- Light
- Front pellote with reinforcement welded inside. With insert for the hand to make putting on easier

Item	M90070 • M90072				
Size	XS	S	L	XL	XXL
Pelvic circumf. cm	70/85	85/100	100/115	115/130	130/145
Colour	chalk white				

OPTION:

M70010 small lumbar pad, size 24x23

M70020 large lumbar pad, size 24x31

M90074 back splint



dorsopull74

Padded dorso-lumbar back splint for the orthopaedic Lumbopull orthosis

M90074



INDICATIONS

- Back pain due to arthrosis or kyphosis
- After-effects of fractures of dorsal vertebrae and/or upper lumbar vertebrae
- After-effects of stable compression fractures of the dorsal spine due to osteoporosis or metastasis



FEATURES

- When combined to Lumbopull 70 or 72, it becomes a padded shoulder orthopaedic orthosis corset with shoulder straps
- Self-shaping dorsal stays included
- There is also the option of inserting a formable, stable internal frame which is made of aluminium alloy and radiolucent
- New length adjusting shoulder straps with an elastic panel at the back enable a secure fit and the brace to be worn like a backpack
- Two-fold compound cell made of PE foam

Item	M90074			
Size	XS	S	M	L
Distance T1-L5	35/41	41/47	47/54	54/61
Colour	chalk white			

spinalplus

Spinal brace for osteoporosis

spinalplus 2.0
M33049



PATENT: EP2502608



INDICATIONS

- Dorsolombalgia after healing of stable compression fractures of dorsolumbar vertebrae
- Stable compression fractures of dorsolumbar vertebrae caused by osteoporosis or metastatic tumors
- Hyperkyphosis of the thoracic spine, post-traumatic or caused by arthrosis
- Juvenile osteochondrosis



FEATURES

- Malleable radiotranslucent aluminium alloy inner frame
- Patented strap system
 - Fastens around the abdomen with a corset
 - A unique tensioning system – all the rear straps and the shoulder straps can be tightened by pulling the pads
 - A special system magnifies the force exerted to double the tension of the shoulder straps – maximum effectiveness with minimum effort
- Dorsal-abdominal straps with a low-friction strap
- Dorsal-abdominal straps attached directly to the metal frame with self aligning buckles and press studs
- The abdominal corset ensures greater comfort and stability:
 - Velcro® fastening
 - Adjustable length
 - Bonded stretchy MTP section and technical stretch fabric
- Biodynamic, stretchy dorsal insert
- Padded shoulder straps with the option of pre-setting their positions
- PE cell bonded with velour and technical fabric with bonded edges

Item	M33049			
Size	XS	S	M	L
Distance T1-L5 cm	35/41	41/47	47/54	54/61
Pelvic circumf. cm	70/125	70/125	70/125	70/125
Colour	blue			

OPTION:

M33049/P Inflatable pad

spinalplus
MT1049



PATENT: EP2502608



INDICATIONS

- Dorsolombalgia after healing of stable compression fractures of dorsolumbar vertebrae
- Stable compression fractures of dorsolumbar vertebrae caused by osteoporosis or metastatic tumours
- Hyperkyphosis of the thoracic spine, post-traumatic or caused by arthrosis tumours
- Juvenile osteochondrosis



FEATURES

- Padded shoulder straps with position settings which can be set prior to use
- Velcro® closure
- Elastic insert with biodynamic effect
- Mouldable inner frame in aluminium alloy, X-ray permeable (except for the snap fasteners)
- Abdominal and dorsal tensioning straps, made from comfortable belt material
- Buckles with self-aligning mounting points
- State-of-the-art system of tensioning straps
- Two-layer PE cell with suede and mesh fabric cover

Item	PR1-T1049			
Size	XS	S	M	L
Distance T1-L5 cm	35/41	41/47	47/54	54/61
Colour	blue			

e-STAB

3-point hyperextension brace

M22035



PATENT: registered community design



TELESCOPIC

INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset



FEATURES

- Structure:
 - A lightweight aluminium alloy frame without any sharp metal edges for maximum comfort
 - Telescopic mechanism: the brace can be adjusted at the sides with the screws, without having to remove it, both vertically and horizontally
- Pelvic band: tilting; telescopic; the end parts are made of a resistant and flexible polymer for optimal adaptability to the anatomy of the patient; grub screws allow the tilting to be secured in position at 0° or 18°
- The angle of the underarm joints can be adjusted using the notched system that corresponds with the angle of the underarm pads (every 7,5°)
- Moveable sternal plate
- Fastening system:
 - Lightweight and resistant lever, in acetal resin, with safety lock
 - Exclusive belt length adjustment system with double-pass buckle and self-locking pre-fastening system
- Anatomical dorsolumbar plate
- Removable padded parts, fixed with buttons/snaps. Made of PE foam, bi-coupled with elastane blend fabric, with water repellent treatment

Size	1	1S	2	2S	3	3S
Pelvic circ. (cm)	50-75		75-105		105-135	
Height* (cm)	41-54	30-41	41-54	30-41	41-54	30-41

* measured from the proximal side of the corpus sterni to the upper edge of the pubic bone (tuberculum pubicum)

OPTION:

MIP35/PC connection plate between the Cervistable collar and the hyperextensor
MIP35/P sternal hemi-plate for PaceMaker wearers
MP1000/IA underarm protection sleeve in a non-toxic hypoallergenic material

stabil-T

3-point hyperextension brace

MP1000



INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset



FEATURES

- A lightweight aluminium alloy frame without any sharp metal edges for maximum comfort
- The brace can be adjusted at the sides with screws and at the top with a telescopic mechanism (the screws do not need to be removed)
- The angle of the underarm joints can be adjusted using the notched system that corresponds with the angle of the underarm pads (7.5°)
- An innovative polymer spring enables the pelvic band to be tilted (it can also be secured in position at 0° and at 18° using grub screws)
- Moveable sternal plate
- A new strap with three-step adjustable length and a clip to hold any excess strap
- A plastic lever fastening with a safety lock
- The padded parts are water-repellent and can be immersed in water: PE foam covered with non-toxic synthetic leather
- The back plate can be worn horizontally and vertically

Item	MP1000									
Size	XS	S	MS	M	ML	LS	L	XLS	XL	
Pelvic circumf. cm	50/60	60/75	75/90	75/90	75/90	90/105	90/105	105/115	105/115	
Height cm*	30/37	34/42	34/41	36/46	40/52	36/46	40/52	39/49	42/54	
Colour	grey									

* measured from the proximal side of the corpus sterni to the upper edge of the pubic bone (tuberculum pubicum)

OPTION:

MIP35/PC connection plate to cervical collar CerviStable
MIP35/P sternal half-sized pad for pace-maker wearers
MP1000/IA underarm protective lining in non-toxic, non-allergic material
MP10-S05 optional pad, can be shaped to fit

modular plus35K

3-point hyperextension brace

MP35K PLUS



INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset



FEATURES

- Frame made from light aluminium alloy
- Frame construction
- Movable sternal plate
- Back pad can be positioned either horizontally or vertically
- Adjusted via screws
- Spring oscillation of the pelvic strap (with the option of securing at 0° and 18°)
- Lever closure made of plastic with safety lock
- Newly designed strap with special hook closure system for adjusting the length
- Padded parts made of soft foam with non-toxic, synthetic leather cover
- Underarm protective lining in non-toxic, non-allergic material

Item	MP35K PLUS								
Size	XS	S	MS	M	ML	LS	L	XLS	XL
Pelvic circumf. cm	50/60	60/75	75/90	75/90	75/90	90/105	90/105	105/115	105/115
Hyperextensor height cm*	30/37	34/42	34/41	36/46	40/52	36/46	40/52	39/49	42/54
Colour	white								

* measured from the proximal side of the corpus sterni to the upper edge of the pubic bone (tuberculum pubicum)

OPTION:

- M35/4M** reinforced back pad
- MIP35/P** sternal half-sized pad for pace-maker wearers
- MIP35/PC** Connection plate to cervical collar CerviStable

modular plus35 acqua

3-point hyperextension brace

MP35A PLUS



INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset



FEATURES

- Frame made from light aluminium alloy
- Frame construction
- Movable sternal plate
- Back pad can be positioned either horizontally or vertically
- Adjusted via screws
- Spring oscillation of the pelvic strap (with the option of securing at 0° and 18°)
- Lever closure made of plastic with safety lock
- Newly designed strap with special hook closure system for adjusting the length
- Padded parts and belt in water-repellent material, suitable for use in water
- Underarm protective lining in non-toxic, non-allergic material
- Adjustable posterior Velcro® straps

Item	MP35A PLUS							
Size	S	M	M	M	L	L	XL	XL
		short		long	short		short	
Pelvic circumf. cm	60/75	75/90	75/90	75/90	90/105	90/105	105/115	105/115
Hyperextensor height cm*	30/37	34/42	36/46	40/52	36/46	40/52	39/49	42/54
Colour	white							

* measured from the proximal side of the corpus sterni to the upper edge of the pubic bone (tuberculum pubicum)


OPTION:

- M35/4M** reinforced back pad, can be shaped to fit

modular plus36

3-point hyperextension brace with pubic pad

MP36 PLUS

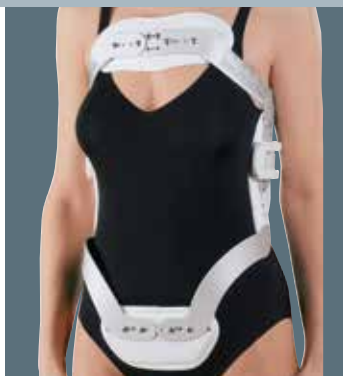

INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset



FEATURES

- Frame made from light aluminium alloy
- Frame construction
- Movable sternal plate
- Back pad can be positioned either horizontally or vertically
- Fixed pubic pad
- Adjusted via screws
- Lever closure made of plastic with safety lock
- Newly designed strap with special hook closure system for adjusting the length
- Padded parts made of soft foam with non-toxic, synthetic leather cover
- Underarm protective lining in non-toxic, non-allergic material

Item	MP36 PLUS			
Size	S	M	L	XL
Pelvic circumf. cm	60/75	75/90	90/105	105/115
Hyperextensor height cm*	30/37	36/46	40/52	42/54
Colour	white			

* measured from the proximal side of the corpus sterni to the upper edge of the pubic bone (tuberculum pubicum)

OPTION:

- M35/4M** reinforced back pad, can be shaped to fit
- M35/23-30** deltapectoralis-Pelotte

modular plus45

3-point hyperextension brace with pressure setting in the sternum region

MP45 PLUS



INDICATIONS

- Traumatic fractures (T10-L1/L2)
- Osteoporotic or metastatic-related compression fractures of the thoracic spine and the dorsolumbar transition (T10-L1/L2)
- Osteomalacia with fractures
- Dorsolumbar osteoarthritis
- Substitute for plaster corset



FEATURES

- Frame made from light aluminium alloy
- Frame construction
- Movable sternal plate
- Back pad can be positioned either horizontally or vertically
- Adjusted via screws
- Spring oscillation of the pelvic strap (with the option of securing at 0° and 18°)
- Lever closure made of plastic with safety lock
- Newly designed strap with special hook closure system for adjusting the length
- Padded parts made of soft foam with non-toxic, synthetic leather cover
- Toothed wheel under the arm to adjust the pressure against the sternum: each tooth equals 5° setting variation
- Underarm protective lining in non-toxic, non-allergic material

Item	MP45 PLUS							
Size	S	M	M	M	L	L	XL	XL
		short		long	short		short	
Pelvic circumf. cm	60/75	75/90	75/90	75/90	90/105	90/105	105/115	105/115
Hyperextensor height cm*	30/37	34/42	36/46	40/52	36/46	40/52	39/49	42/54
Colour	white							

* measured from the proximal side of the corpus sterni to the upper edge of the pubic bone (tuberculum pubicum)

OPTION:

- M35/4M** reinforced back pad, can be shaped to fit

connect

Connection plate between cervical collar Cervistable and the hyperextension braces Modular plus or e-STAB

MIP35/PC



USABLE WITH
HYPEREXTENSION BRACES

INDICATIONS

- Simultaneous illness within the range of the neck spinal column and the first thoracic vertebra

FEATURES

- Aluminum plate designed for simultaneous illness within the range of the neck spinal column and the first vertebra of the chest
- Connection plate for hyperextension brace and Cervistable Collar with or without Trachea opening. Will fit any 3-point hyperextension brace

Item	MP35KPL
Size	universal

nhyco

Cruciform spinal hyperextension orthosis

MD100

RDM: 188197



INDICATIONS

- Extension and immobilisation of the dorso-lumbar spine
- Post-operative immobilisation
- Hyperkyphosis and roundback
- Scheuermann disease
- Collapsed vertebra caused by osteoporosis
- Collapsed vertebra caused by metastasis




FEATURES

- Lightweight T6 aluminum alloy frame (allows a weight of less than 700 g)
- Adjustable and movable sternal and pubic pads
- Lumbar pad in 3 sizes: small (horizontal), regular, long
- Replaceable EVA cell-waterproof, non toxic and latex-free
- New PLUS fastening made from light, durable compound material. Simple to open and close, guarantees constant lumbar pressure. With safety fastening
- Simple telescopic adjustment of frame - no need to remove screws

Item	MD100		
Size	S	R	L
Distance sterno pube cm	27/34	34/41	40/53
Waist max. circ. cm	90	120	120
Colour	white		





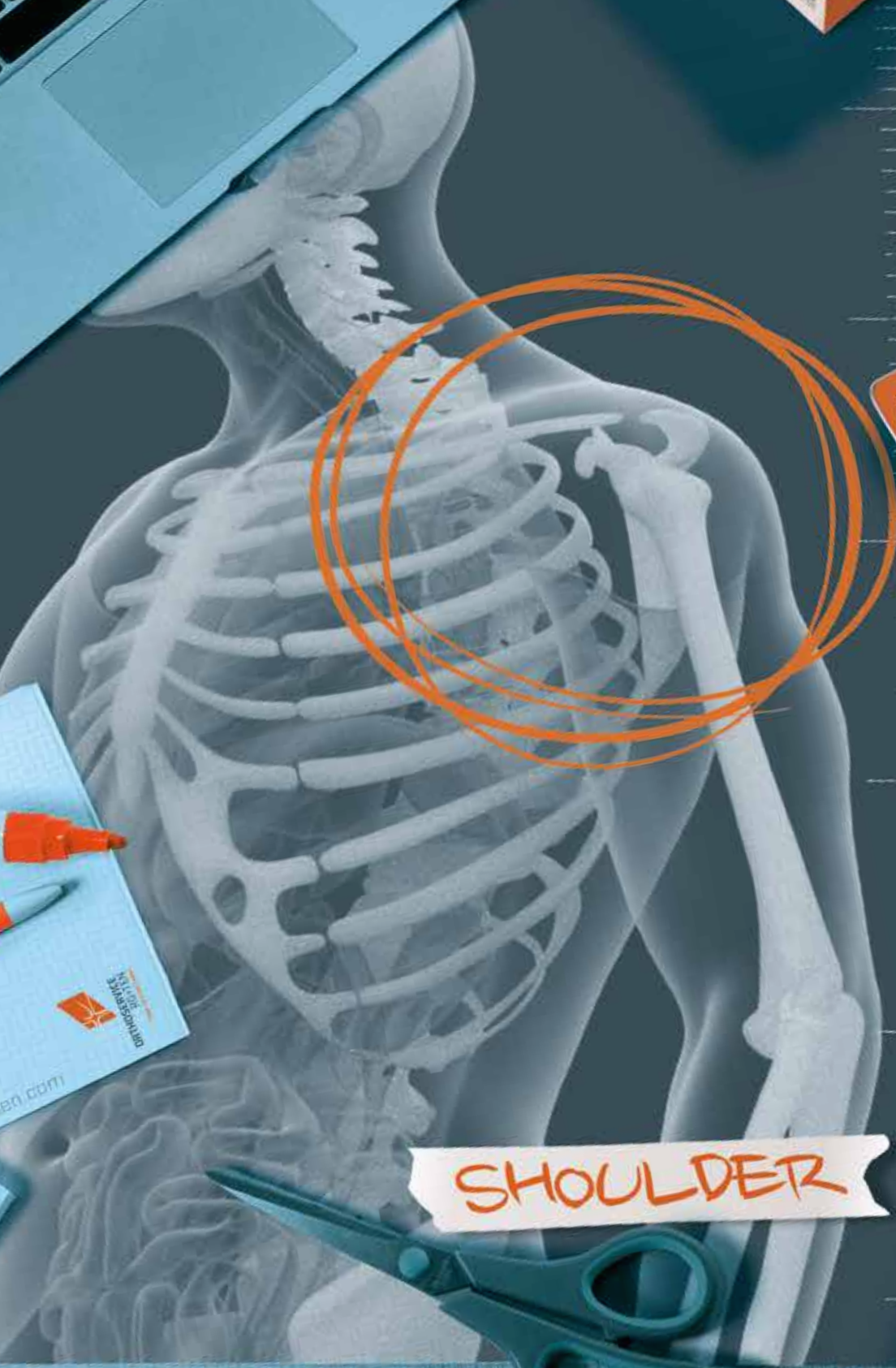

SHOULDER



shouldercross

Functional shoulder brace

M20726 R
M20726 L



INDICATIONS

- Conservative treatment of instability after both front and back glenohumeral dislocations
- Protection during sports or working activities at risk when waiting for surgery or after stabilisation surgery
- Use for shoulder rehabilitation treatment of painful hyperlaxity or functional instability
- Use three months after surgery to repair rotator cuff when starting work or sport, including when managing massive ruptures of rotator cuff



FEATURES

- BreathSkill fabric structure:
 - Soft and breathable
 - 3D fabric
 - External coverage made of a strong mesh fabric
- Crossover elasticated support straps for optimum joint stabilisation
- Lateral adjustable belt; partially elastic and underarm cell
- Adjustable arm with elastics and Velcro® fastening system

Item	M20726 R • M20726 L			
Size	S	M	L	XL
Circumf. max. biceps	22/26	26/30	30/35	35/40
Colour	grey			

specify right or left



**CONTAINMENT
ALSO IN THE REAR**



octofix (cvb)


Clavicle immobiliser

M8503



INDICATIONS

- Clavicle fractures
- Subluxation of the acromioclavicular joint

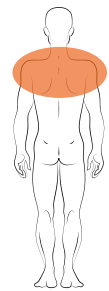

FEATURES

- Gauzed fabric with cell
- 8 type bandage with Back Velcro® fastening system

Item	M8503					
Size	XS	S	M	L	XL	XXL
Chest circumf. cm	55/70	70/85	85/100	100/110	110/120	120/135
Colour	white					

Shoulder > Acromioclavicular immobiliser

acromion 2.0

Brace for acromioclavicular joint subluxation

MS1063 R
MS1063 L




PATENT: EP2591757



INDICATIONS

- Conservative treatment for stabilisation of the acromioclavicular joint after traumatic dislocations of first-second-third degrees (at the discretion of the physician)
- Conservative treatment after fracture of far distal clavicle
- Read the dedicated article of Archives of Orthopaedic and Trauma Surgery

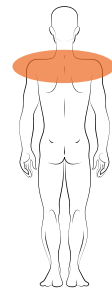

FEATURES

- System designed to push clavicle downward and the humerus upward to align acromion and clavicle (exclusive)
- Anatomical, flexible clavicle brace with polyethylene foam cell and rotating joints
- Lateral strap with buckle (orange) for a rapid removal
- Adjustable, comfortable elbow and forearm support, polyethylene foam semi-rigid cell for an easy application
- Adjustable fixing belt snap (white) with Velcro® fastening system
- Rear straps with both constant and pre-set tensioning
- Velour straps, adaptable size

Item	MS1063 R • MS1063 L
Size	universal
Colour	blue

specify right or left



Do you want
to learn more about
Acromion 2.0?

Watch
the video



Read
the scientific
publication



shouldfix

Arm and shoulder immobiliser


shouldfix M8409



WITH ELBOW POCKET SUPPORT

INDICATIONS

- Stable humeral fractures
- Gleno-humeral luxation
- Distorsions of the shoulder
- Post surgery humerus fractures and gleno-humeral luxation treatment
- Post surgery after tenorrhaphy of long head of humeral biceps

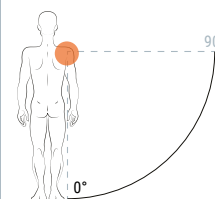

FEATURES

- Mixed cotton sling
- Immobiliser band made of soft padded fabric
- Shoulder straps with Velcro® fastening system
- Elbow pocket support

Item	M8409
Size	universal
Colour	black

fits right and left


shouldfix II M8412



WITH REMOVABLE ELBOW POCKET

INDICATIONS

- Stable humeral fractures
- Gleno-humeral luxation
- Distorsions of shoulder
- Post surgery humerus fractures and gleno-humeral luxation treatment
- Post surgery after tenorrhaphy of long head of humeral biceps

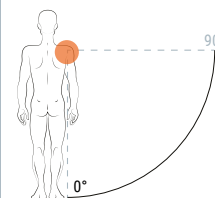

FEATURES

- Pocket sling made of Velcro padded fabric, perfectly adjustable
- Soft Velcro braces
- Exclusive mesh elbow-block band
- Fixation band made of padded fabric, perfectly adjustable

Item	M8412
Size	universal
Colour	black


fits right and left

shouldfix open M8410



INDICATIONS

- Stable humeral fractures
- Gleno-humeral luxation
- Distorsions of shoulder
- Post surgery humerus fractures and gleno-humeral luxation treatment
- Post surgery after tenorrhaphy of long head of humeral biceps

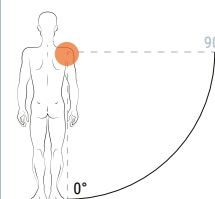

FEATURES

- Soft structure with external gauze nylon to match Velcro® fastening system
- Shoulder strap with snap closure

Item	M8410	
Size	Regular	Large
Waist circumference in cm	< 120	> 120
Colour	black	

fits right and left

shouldfix

Brace for shoulder immobilisation


shouldfix light M20413



WITH CHEST STRAP

INDICATIONS

- Gleno-humeral luxation
- Distorsions of shoulder
- Post surgery

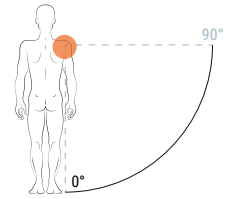

FEATURES

- Chest band made of velcro padded fabric, perfectly adjustable
- One arm band and two forearm bands made of Velcro® padded fabric to be freely positioned on chest band

Item	M-20413
Size	universal
Colour	black

fits right and left


shouldfix 3 M20414



WITH ELBOW SUPPORT

INDICATIONS

- Stable humeral fractures
- Gleno-humeral luxation
- Distorsions of shoulder
- Post surgery humerus fractures and gleno-humeral luxation treatment
- Post surgery after tenorrhaphy of long head of humeral biceps

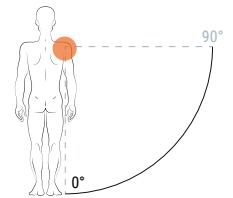

FEATURES

- Made of cotton and polyester with elbow pocket support
- Chest immobiliser strap with central padded fabric and Velcro® fastening system
- Shoulder straps with Velcro® fastening system with central padded fabric

Item	M20414			
Size	S	M	L	XL
Forearm length* cm	28/34	34/42	42/48	48/58
Colour	black			

fits right and left

*measurement taken from the elbow to the end of the closed fist

Shoulder > 10°->20° abduction immobilisers

omoplus 2.0


Shoulder brace with immobiliser in fixed abduction of 10°

M90080 R
M90080 L



INDICATIONS

- Post surgery (in abduction) to repair rotator cuff and soft tissue, glenoid labrum suture, capsular stabilisation

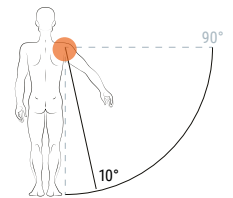

FEATURES

- Shoulder support (rotating buckles) on homolateral/injured shoulder to ensure its physiological posture, to prevent neckache
- Controlateral fixing strap with orange buckle for a rapid opening
- Back triangle straps for scapular stability
- Pad: high density foam rubber with gauze fabric cover and fabric with silver fibre in contact with the body
- Length adjustable immobiliser
- Arm pocket support made of breathable material: easy elbow mobility
- Adjustable abdominal strap with and snap closing
- Tilted toward arm support for a better comfort
- Supporting strap for thumb and ball for rehabilitative purposes

Item	M90080 R - M90080 L	
Size	S	L
Forearm length* cm	<38	>39
Circumf. max forearm cm	<32	>29
Colour	nero	

fits right and left

*measurement taken from the elbow to the end of the closed fist

top-S


Shoulder brace with immobilisers in fixed abduction of 10° or 15°

top-S 10°/15°


ME110 10°
ME210 15°



FIXED ABDUCTION OF 10° OR 15°

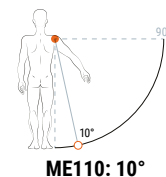

ME110: 10°

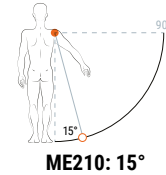


ME210: 15°

top II-S 10°/15°


ME111 10°
ME211 15°



FIXED ABDUCTION OF 10° OR 15°

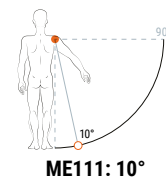

ME111: 10°

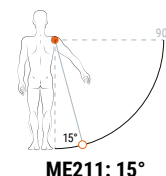

ME211: 15°

INDICATIONS


- Post surgery (in abduction) to repair rotator cuff and soft tissue, glenoid labrum suture, capsular stabilisation

top 3-S

ME112



FIXED ABDUCTION OF 10°

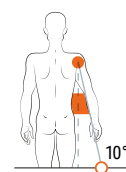

10°

FEATURES

- High density foam rubber with gauze fabric cover (to perfectly manage Velcro®) and fabric with silver fibre in contact with the body
- Protection neck sleeve
- External gauze nylon cover and sleeves with Velcro® fastening system
- Breathable arm pocket material
- Fixing straps with adjustable buckles with Velcro® fastening system
- Supporting strap for thumb
- **Only Top II-S:**
 - Pad with anteroposterior edge to improve stability
 - Ball for rehabilitative purposes
- **Only Top III-S:**
 - Pad with front edge for an excellent stability
 - Ball for rehabilitative purposes
 - Underarm strap to avoid neck pressure and increase rotatory stability

Item	Top-S ME110 10° • ME210 15°		
Size	S	M	L
Forearm length cm*	<34	35/39	>40
Colour	black		

*Measurement taken from elbow to end of closed fist fits right and left

Item	Top II-S • ME111 10° • ME211 15°	
Size	S	L
Forearm length cm*	<38	>39
Colour	black	

*Measurement taken from elbow to end of closed fist fits right and left

Item	Top III-S • ME112	
Size	S	L
Forearm length cm*	<38	>39
Colour	black	

*Measurement taken from elbow to end of closed fist fits right and left

Shoulder > 10°->20° abduction immobilisers


top 4-S ME410 R ME410 L



EXTERNAL ROTATION OF 10° OR 15°

INDICATIONS

- Post surgery (in abduction) to repair rotator cuff and soft tissue, glenoid labrum suture, capsular stabilisation

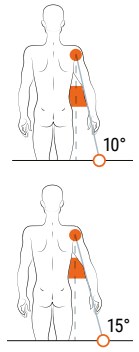

FEATURES

- High density foam pad with gauze fabric cover made of silver fibre in contact with the body
- Pad with front edge to improve stability. Tilted toward arm support for a better comfort
- Protection neck sleeve
- Cover and sleeves made of external gauze nylon to be fastened with Velcro®
- Arm pocket support made of breathable material
- Adjustable straps with buckles and Velcro® fastening system
- Supporting strap for thumb
- Abduction of 10° or 15°, driven application and positioning of arm pocket on pad

Item	Top 4-S • ME410 R • ME410 L
Size	universal
Colour	black

specify right or left

Shoulder > ≥ 20° abduction immobilisers

Softab

Shoulder brace with immobilisers in abduction


Softab 20° M84502



ABDUCTION 15°-20°

INDICATIONS

- Post-surgery of rotator cuff injury
- Nerve injury

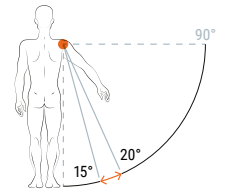

M84502: 15°-20°


Softab 45°/70° M84500



ABDUCTION ADJUSTABLE 45° OR 70°

INDICATIONS

- Tuberosity fractures with slight dislocation
- Adhesive capsulitis after mobilization
- Post-surgery of endo and arthroplasty of shoulder
- Post-surgery of rotator cuff injury
- Nerve injury

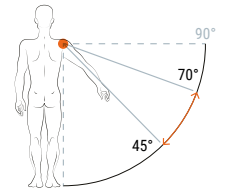

M84500: 45° or 70°

FEATURES

- High density foam pad
- Cover, chest straps and sleeves made of paired fabric with external gauze nylon to be fastened with Velcro®
- Shoulder strap with Velcro® fastening system
- **Only Softab 45°-70°:** Protection neck sleeve

Item	Softab 20° M84502 • Softab 45°/70° M84500
Size	universal
Colour	black

fits right and left

OPTION:

M84500/A - Hand and wrist support

extra-S

Shoulder brace with immobilisers in external rotation

extra-S 15°

ME113 R
ME114 L



EXTERNAL ROTATION OF 15°

extra-S 30°



ME115 R
ME116 L



EXTERNAL ROTATION OF 30°

INDICATIONS

- Post surgery (external rotation) to repair rotator cuff and soft tissue, glenoid labrum suture, capsular stabilisation
- Conservative treatment of first glenohumeral dislocations

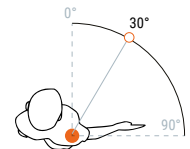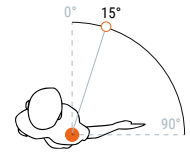


FEATURES

- High density foam pad with gauze fabric cover made of silver fibre in contact with the body
- Cover and sleeves made of external gauze nylon to be fastened with Velcro®
- Adjustable straps with buckles and Velcro® fastening system
- Protection neck sleeve
- Possible external rotation of 15° or 30°
- Supporting strap for thumb
- Ball for rehabilitative purposes
- Arm pocket support made of breathable material

Item	Extra-S 15° ME113 R • ME114 L Extra-S 30° ME115 R • ME116 L		
Size	S	M	L
Forearm length cm*	< 34	35/39	> 40
Colour	black		

*Measurement taken from elbow to end of closed fist
specify right or left

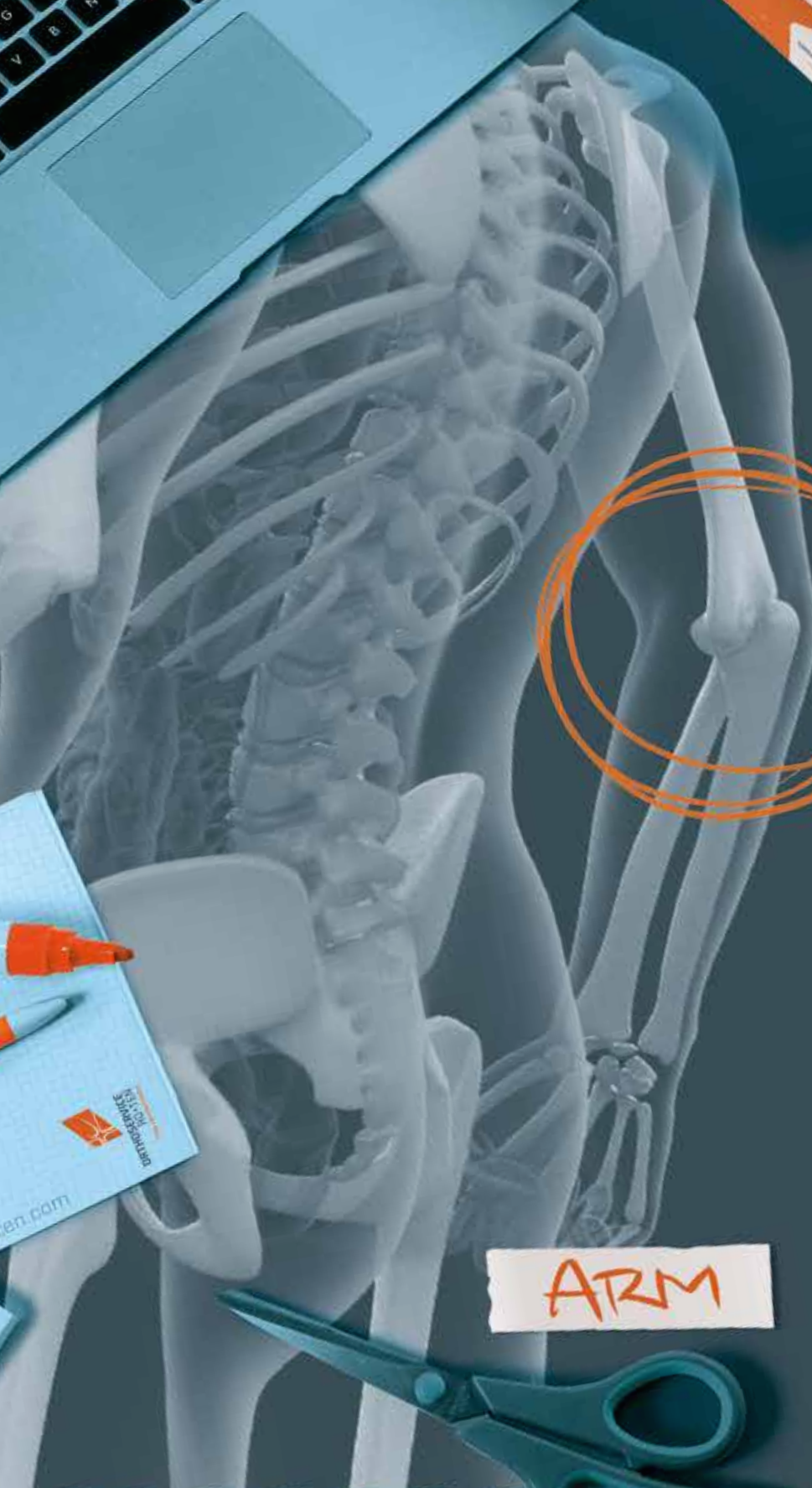

ARM



epilastik

Elasticated elbow sleeve

epilastik 32 M90132



epilastik PLUS M90132F



INDICATIONS

- Lateral epicondylitis (tennis elbow) and medial epicondylitis (golfer's elbow)
- Chronic pain from bursitis or tendinitis

FEATURES

- Stretch knit fabric featuring areas of different targeted compression
- Anatomically designed
- Silicone inserts shaped to redistribute compression on the epicondyle and epitrochlea and pressure pads on the forearm
- The edging is strengthened with silicone trim so the support feels softer and keeps its shape when being pulled on

Item	M90132 - M90132F						
Size	0	1	2	3	4	5	6
Forearm circ. 13 cm from the epicondyle cm	17/19	19/21	21/23	23/25	25/27	27/29	29/31
Colour	M90132 blue - M90132F yellow fluo						

fits right and left

epiFIT 32

Aer-OS elbow sleeve

MB1132



INDICATIONS

- Epicondylitis and medial epicondylitis
- Bursitis or tendinitis' chronic pain

FEATURES

- Breathable Aer-OS material
- Soft-X insert for maximum comfort
- Velcro® strap with nylon

Item	MB1132				
Size	XS	S	M	L	XL
Forearm circ. 13 cm from the epicondyle cm	20/23	23/25	25/30	30/35	35/40
Colour	black				

fits right and left



Try out the new 360° feature to explore all the details of Elbo Pro

Scan the QR code



elbo PRO

Telescopic elbow brace with graduated movement

M24017 R
M24017 L


New



TELESCOPIC

INDICATIONS

- Compound fractures of elbow, distal humerus, proximal radius and ulna
- Post-operative immobilisation following tendon and ligament repairs
- Useful in the rehabilitation phase following surgical and conservative treatment of elbow fractures
- Elbow subluxations and dislocations
- Chronic elbow injuries
- Elbow hyperextension
- Post-traumatic treatment

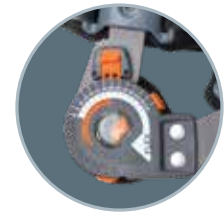

New "Pull&Set" flexion-extension adjustment joint, made of a polymer lightweight material allowing options for more precise control:

- Flexion: from -10° to +120° with intervals of 10°
- Extension: -10° to +90° with intervals of 10°
- Fast lock joint with a specific button, from - 10° to +90° with intervals of 10°

FEATURES

- Lateral articulated hinge made of a light aluminium alloy
- Telescopic length adjustment via easy-to-use buttons on the valves
- Polyamide valve with mouldable aluminium arm grip for perfect individual fit.
- Length-adjustable straps
- Strap clips supplied
- Heat-sealed cell made of bi-coupled MTP, with soft lycra in contact with the skin
- Length-adjustable shoulder strap, with padded shoulder protector
- Neutral hand support included

Item	M24017 R • M24017 L
Size	universal
Length of the hinges cm	36-46
Colour	gray

specify right or left



NEUTRAL
HAND
SUPPORT
INCLUDED



TELESCOPIC
REGULATION

elbo

Elbow Brace with graduated movement

elbo
MB1014



INDICATIONS

- Stable fractures of elbow, distal humerus, radius and proximal ulna
- Post-surgery immobilization due to tendons and ligaments repair
- Use for rehabilitation after conservative and surgical treatments of elbow fractures
- Elbow luxation and subluxation
- Elbow chronic injuries
- Elbow hyperextension
- Post-traumatic treatment



Articulated element made of light aluminum alloy and steel junction

- Adjustable extension from 0° to 90° (10° step)
- Adjustable flexion from 0° to 90° (10° step)
- Possible joint block from flexion from 0° to 90° every 10°

FEATURES

- Adjustable position of arm and forearm
- PE plastic to fit with the patient. Pad made of double layer MTP
- Fabric made of silver fibre in contact with the body (anti-bacterial and odor-free)
- Strap
- Articulated element made of light aluminum alloy and steel junction

Item	MB1014 R • MB1014 L	
Size	S	L
Arm circ. max. cm	20/28	28/40
Forearm circ. max cm	20/27	25/35
Colour	black-white	

specify right or left

OPTION:
MB1015 Neutral hand support system for Elbo



OPTION:
MB1016 Prono-supination hand support for Elbo



castop


Humeral fracture brace

M725



INDICATIONS

- Stable fractures of medial distal third of humerus
- Post-surgery brace after humerus fractures

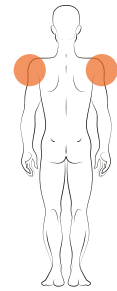

FEATURES

- Structure made of Polyethylene
- Internal pad made of close cells against moisture
- Velcro® straps
- Ventilation holes
- Two tubular sleeves included

Item	M725			
Size	S	M	L	XL
Proximal circ. humerus cm	23/28	28/33	33/38	38/43
Distal circ. humerus cm	18/23	23/28	28/33	33/38
Brace length cm	19	20	21	23
Colour	white			

fits right and left

castop


Shoulder cap humeral brace

M726



INDICATIONS

- Stable fractures of medial distal third of humerus
- Post-surgery brace after humerus fractures
- Fractures due to non-operative bone metastasis

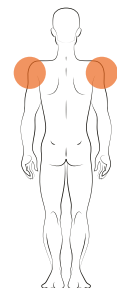


FEATURES

- Structure made of polypropylene bivalve
- Internal pad made of close cells against moisture
- Extension cap to shoulder-humeral joint for better control of movements
- Velcro® straps
- Ventilation holes
- Two tubular sleeves included

Item	M726				
Size	XS	S	M	L	XL
Proximal circ. humerus cm	20/25	23/28	28/33	33/38	38/43
Distal circ. humerus cm	15/20	18/23	23/28	28/33	33/38
Brace length* cm	34	34	34	34	34
Colour	white				

fits right and left

*can be cut to length required

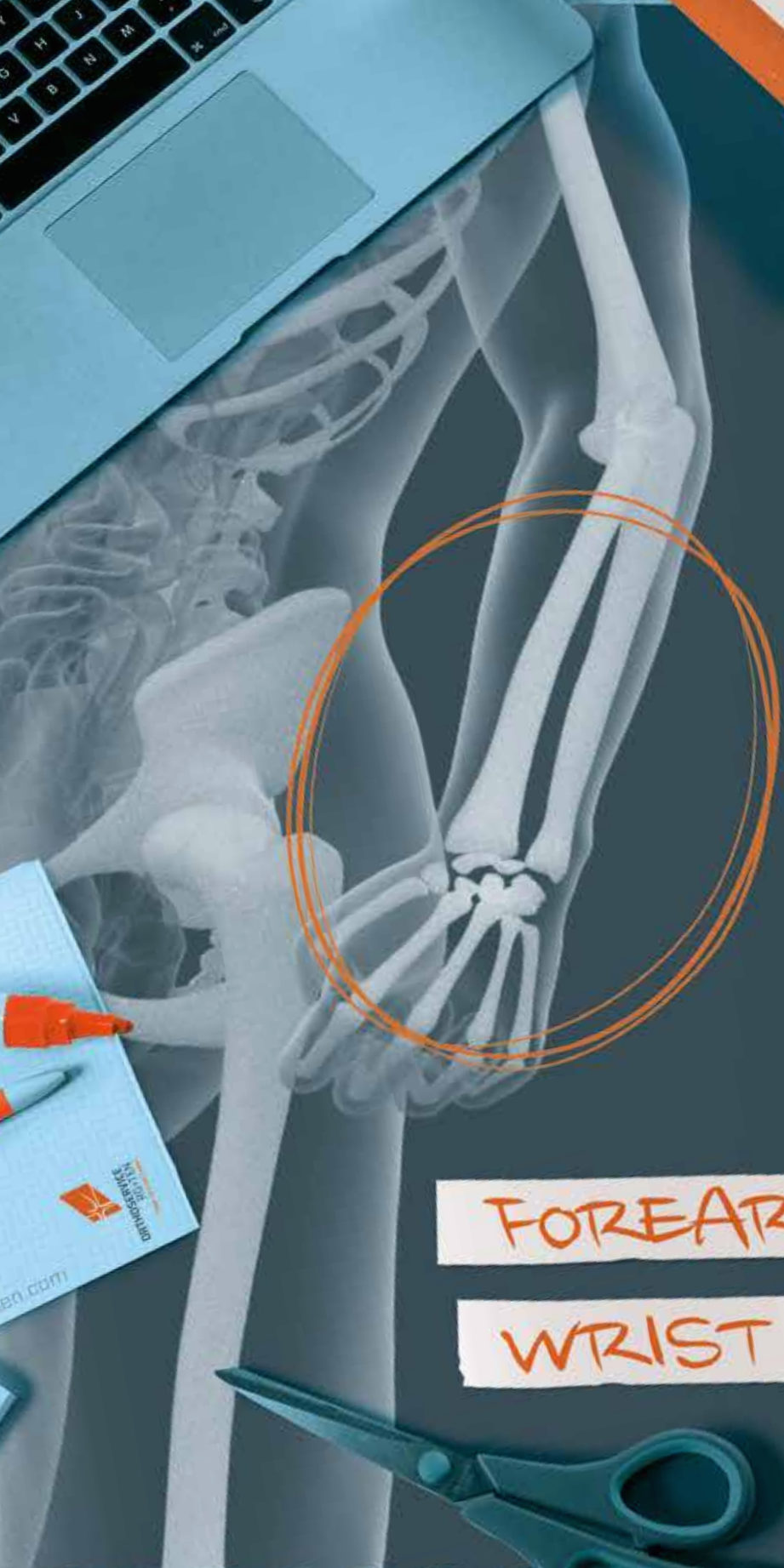

FOREARM

WRIST



Forearm-wrist > Arm slings

sling-up 2.0

Standard arm sling

M8404



INDICATIONS

- Forearm and wrist support
- Support when cast
- Use in case of acromioclavicular (after 1° subluxation) or glenohumeral rest is required
- Use in case of upper arm immobilization with elbow in flexed position

FEATURES

- Structure made of cotton blend
- Padded MTP Shoulder strap with velveteen protection
- Velcro® fastening system

Item	M8404			
Size	S	M	L	XL
Length of pouch cm	34	42	48	58
Colour	black			

fits right and left

Forearm-wrist > Epicondylitis straps

epifeel

Epicondylitis elbow strap with pressure pad

M24022



INDICATIONS

- Acute or chronic epicondylitis and epitrochleitis of the elbow
- Muscle contractures
- Prevention of inflammatory processes during sports activity



FOR
EPITROCHLEITIS

FEATURES

- Polyamide structure for optimal pressure distribution
- Pressure pad to focus the thrust; made of non-slip TPE for an easy positioning of the brace
- Lycra cell coupled with polyurethane foam
- Adjustable strap for accurate compression control
- Velcro® fastener with retaining ring
- Adjustable strap length with practical Y-shaped Velcro®

Item	M24022
Size	universal
Colour	grey

fits right and left

OPTION: ICEGEL Pad for cold therapy with Velcro® fastening system

epi020

Medial epicondylitis strap providing compression

M91020



INDICATIONS

- Severe and chronic epicondylitis and medial epicondylitis
- Muscle tensions
- Preventing inflammation during sports



FEATURES

- Rigid polypropylene C-shape design
- Velcro® fastening system
- Soft pad with double-lined MTP
- Targeted compression

Item	M91020				
Size	XS	S	M	L	XL
Wrist circ. forearm cm	20/22	23/25	26/28	29/31	32/34
Colour	black				

fits right and left

epi320

Medial epicondylitis strap providing compression

M4320



INDICATIONS

- Severe and chronic epicondylitis and medial epicondylitis
- Muscle tensions
- Preventing inflammation during sports



FEATURES

- Double-lined neoprene with leather-like support
- Velcro® fastening with a nylon loop
- Non-slip fabric pressure pad

Item	M4320	
Size	S	L
Wrist circ. forearm cm	< 27	> 27
Colour	beige	

fits right and left

object

Neoprene wrist support

object 300 M82300



INDICATIONS

- Wrist sprains
- Tendinitis and tenosynovitis
- Inflammatory degenerative wrist diseases

object 600 M82600



WITH THUMB SUPPORT

INDICATIONS

- Sprained thumb
- Severe carpometacarpal osteoarthritis
- Thumb arthrosis

FEATURES

- Double-lined neoprene
- Velcro® fastening system
- **Only object 600:** Stay and stability of thumb

Item	M82300 · M82600
Size	universal
Colour	beige

fits right and left

manulastik

Elastic wrist support

manulastik 19 M90119 R M90119 L



INDICATIONS

- Wrist and carpal sprains
- Tendinitis wrist
- Arthrosis and arthritis of wrist
- After wrist fractures
- Post-surgery wrist treatment



manulastik PLUS M90119F R M90119F L



FEATURES

- Stretch knit fabric with different areas of compression
- Anatomically design
- Thumb hole
- Polymeric palm stay
- Velcro® fastening system
- Silicone insert to fit the styloid process
- Forearm edging for softness, traction resistant during application

Item	M90119 R - M90119 L M90119F R - M90119F L						
Size	0	1	2	3	4	5	6
Wrist circumf. cm	13/14	14/15	15/16	16/17	17/18	18/19	19/20
Colour	M90119 R/L blue - M90119F R/L yellow flou						

specify right or left

carpo 41

Adjustable wrist-hand support brace

MP1341 R
MP1341 L



INDICATIONS

- Both conservative and surgical treatment of meta-distal epiphysis of forearm, fractures of carpus (except scaphoid), metacarpal fractures
- Effects of the fractures
- Hand and wrist tendinopathy
- Effects of flexors and finger extensors/tenorrhaphy
- Radio-carpal arthritis
- Neurologic deficits with subsequent paralysis
- Contraindications: mild and serious spasticity

FEATURES

- Adjustable structure made of 2 metals with different stiffness
- Velcro® fastening system
- Ergonomic shape made of silver fibre in contact with the body

Item	MP1341 R - MP1341 L		
Size	S	M	L
Palm width* cm	5,5/7	7/8,5	8,5/10
Brace length cm	30	34	37
Colour	beige		

* (metacarpus) - specify right or left

Forearm-wrist > Wrist immobilisers

manulite

Wrist immobilisers with a rigid and malleable stay

M90100



INDICATIONS

- Wrist sprain
- Stable fractures of radius and ulna edges
- Wrist tendinitis
- Arthritis and arthrosis
- After wrist fractures or plaster's early removal



FEATURES

- Fully welded, no seams and low profile
- External: inextensible velveteen
- Cell: made of MTP
- Internal: soft touch material, skin-compatible and breathable
- Thumb hole bordered in soft touch material for greater comfort
- Aluminum palm splint, preformed and removable on medical prescription
- Reinforcing rigid back of hand splints
- Velcro® fastening system

Item	M90100		
Size	S	M	L
Wrist circumf. cm	13/18	18/23	23/28
Colour	black		

fits right and left

polFIT

Wrist immobilisers, open, with a rigid and malleable stay

polFIT 19
MP1119 R
MP1119 L



SHORT

INDICATIONS

- Wrist sprain
- Stable fracture of forearm radius and ulna
- Wrist tendinitis
- Arthritis and arthrosis
- After wrist fractures or plaster's early removal



polFIT 17
MP1117 R
MP1117 L



SHORT WITH THUMB SUPPORT

INDICATIONS

- Wrist sprain
- Wrist tendinitis
- Arthritis and arthrosis
- After wrist fractures or plaster's early removal
- Arthrosis of trapezio-metacarpal joint and metacarpophalangeal of thumb
- Damage to ulna section of thumb joint



polFIT 21
MP1121 R
MP1121 L



LONG

INDICATIONS

- Treatment of of stable fractures of meta-distal epiphysis of radius and ulna (deemed stable by medical professional)
- Post-surgery after unstable distal fracture of the radius and ulna
- Wrist sprain
- Tendinitis
- Fractures of carpus (except scaphoid) whether treated surgically or conservatively
- Complications of rheumatoid arthritis
- Post-surgery after metacarpal fractures
- Radio-carpal arthrosis



polFIT 22
MP1122 R
MP1122 L



LONG WITH THUMB SUPPORT

INDICATIONS

- A compound fracture of the first metacarpal
- A sprain of the radial ulnar collateral ligament of the metacarpophalangeal joint
- A suspected scaphoid fracture



FEATURES

- Full dorsal opening
- 3D breathable fabric for structure
- Reinforcing dorsal stays
- Velcro® fastening system
- Aluminum palm splint, preformed and removable on medical prescription
- **Only Polfit 17 and Polfit 21:** aluminum thumb splint, preformed and mouldable on medical prescription

Item	MP1119 R - MP1119 L - MP1117 R - MP1117 L MP1121 R - MP1121 L - MP1122 R - MP1122 L			
Size	S	M	L	XL
Wrist circumf. cm	13/16	16/18	18/20	20/23
Colour	black			

specify right or left



HAND



daumfix

Thumb splint

M5006 R
M5006 L



INDICATIONS

- Rhizarthrosis
- Thumb tendinitis
- Tendinopathy of De Quervain
- Metacarpal/phalange sprain
- Steiner's lesion
- Effects of Bennet's fracture
- After trapezium plastic surgery and thumb surgery

FEATURES

- Smooth polypropylene structure
- Velcro® fastening system
- Equipped with antibacterial protective glove. Wear before brace
- Thermoformable

Item	M5006 R - M5006 L			
Size	S	M	L	XL
Wrist circ. cm	13/17	17/19	19/23	23/28
Colour	beige			

specify right or left

rizostab

Thumb spica

M21006



INDICATIONS

- Rhizarthrosis
- Thumb tendonitis
- De Quervain's tenosynovitis
- Metacarpophalangeal sprain
- A Stener lesion
- The after-effects of Bennet's fracture
- The after-effects of trapezium reconstructive surgery and thumb surgery

FEATURES

- A seamless, welded design for maximum comfort
- Anatomically designed
- The aluminium alloy core can be moulded to fit that shape of the patient's hand
- Covered with a strong, padded technical fabric that is breathable and very fine
- Adjustable Velcro® strap for easy and intuitive fastening
- Fits right and left

Item	M21006	
Size	S	L
Wrist circ. cm	14-18	18-22
Colour	grey	

fits right and left

stax

Finger splint (Distal phalanx)

M2001



INDICATIONS

- Injury to extensor tendon
- Segond's lesion
- Fractures of the distal phalanx
- Sprain of distal interphalangeal joint
- Bruised nails
- Post-surgery of extensor tendon

FEATURES


- Clear plastic
- Rigid plastic and transpiration holes

Item	M2001							
Available sizes	1	2	3	4	5	5 1/2	6	7
Colour	transparent							




FINGER MEASUREMENT

To identify the correct size of **Stax**, use the drawing at the bottom of the hand section.

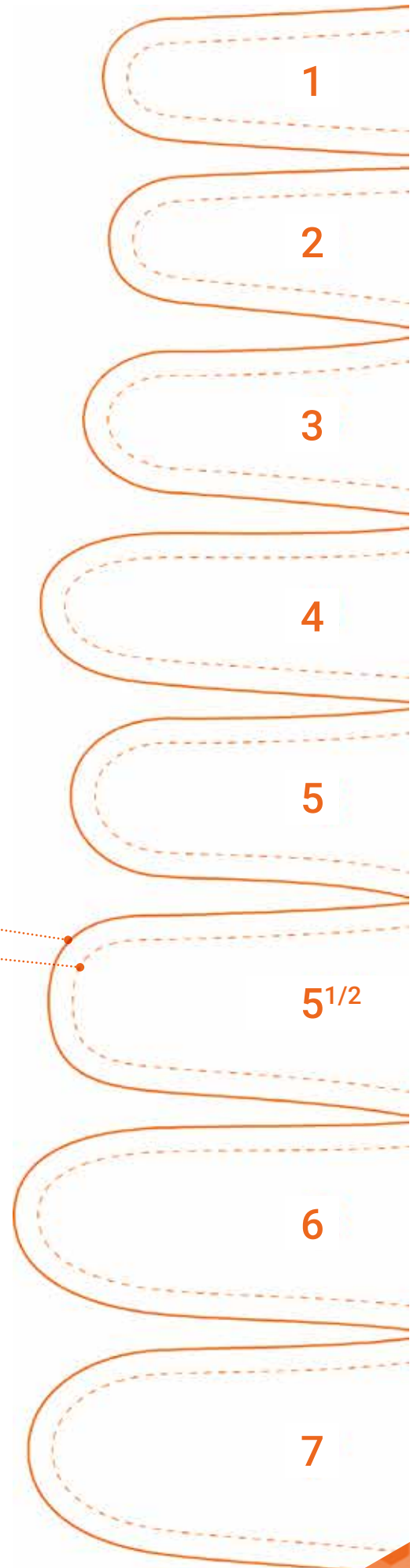

External profile
Internal section



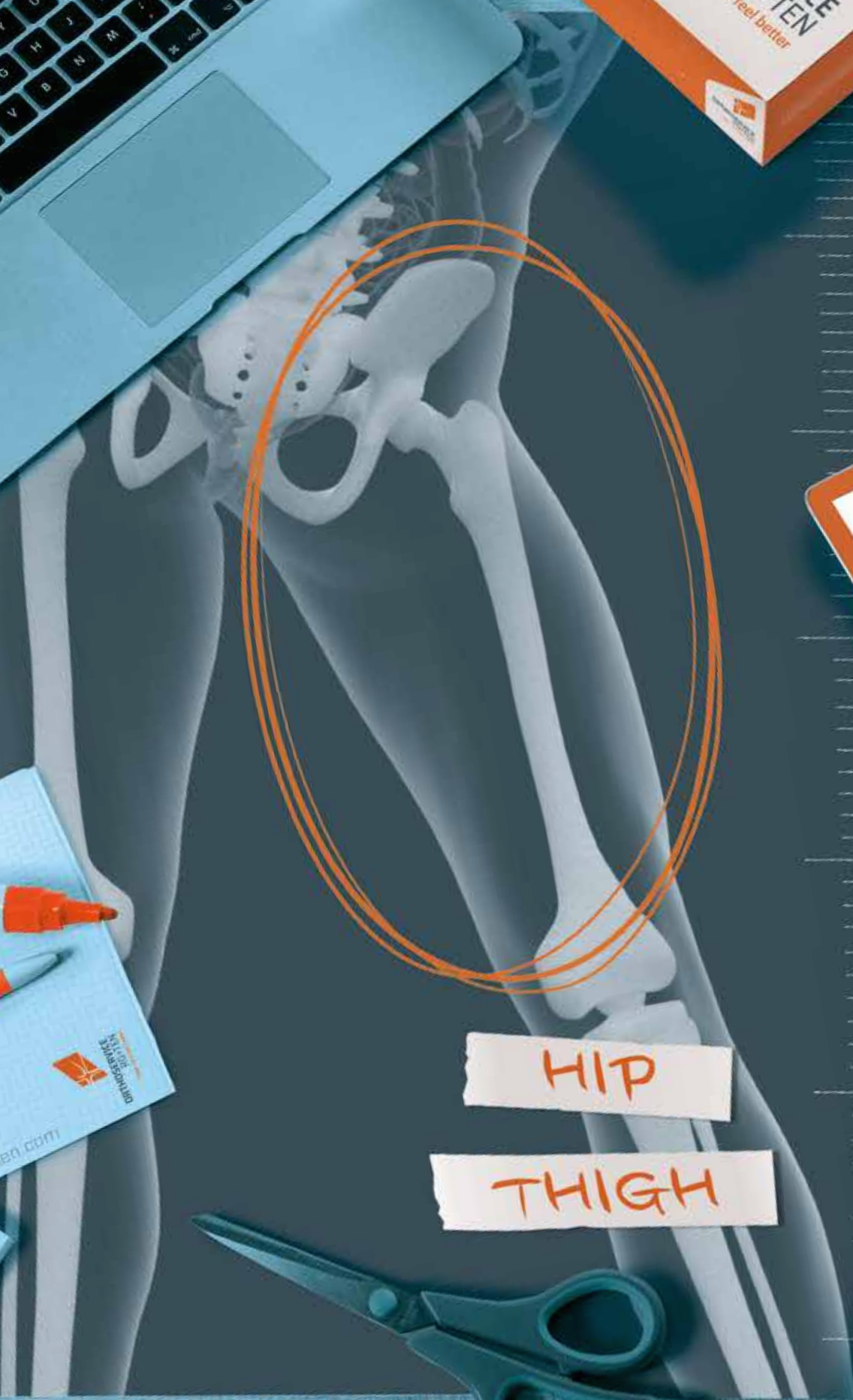

HIP

THIGH



Hip-Thigh > Thigh supports

mioFIT 33

Thigh support

MH1133



IN Aer-OS FABRIC

INDICATIONS

- Muscular contractions and tears of the thigh
- Aid to resuming sporting activity after trauma
- Prevention of bruising

FEATURES

- Made from breathable Aer-OS fabric

Item	MH1133				
Size	S	M	L	XL	XXL
Circ. middle of thigh cm	48/52	52/56	56/61	61/66	66/72
Colour	black				

fits right and left

object 33

Thigh support

M8233



IN NEOPRENE

INDICATIONS

- Muscular contractions and tears of the thigh
- Aid to resuming sporting activity after trauma
- Prevention of bruising

FEATURES

- Double-lined orthopaedic neoprene
- Strengthened neoprene patch
- Washable

Item	M8233				
Size	S	M	L	XL	XXL
Circ. middle of thigh cm	48/51	52/55	56/60	61/65	66/71
Colour	blue				

fits right and left

Hip-Thigh > Iliac supports

symplus

Semi-rigid corset for the sacroiliac joint

MT1032



INDICATIONS

- Sacroiliac joint (SI) pain or dysfunction
- Arthrosis of the sacroiliac joint
- Symphysis Pubis Dysfunction (SPD) (after childbirth) – Note: In this case do not use the padded pelottes

Two padded pelottes which can be placed in variable positions using Velcro® surfaces, designed to provide targeted pressure

FEATURES

- Innovative tensioning system with pulley technology for optimum stabilisation and for simple application with a minimum of force required
- Firm, form-stable and breathable fabric for safe hold, with border of skin-friendly material. Particularly light
- The large Velcro® closure surface enables the tensioning strap to be simply secured in variable pressure settings
- Reinforced paravertebral area

Item	MT1032	
Size	Regular	Long
Hip circ. cm	< 89	89/120
Colour	black	

specify right or left

hipoCROSS 2.0

Brace for hip and coxalgia

M91006N R
M91006N L



PATENT: EP3202376 -
US10548758



INDICATIONS

- Post-arthroscopy care
- Post-surgical care (compatible with the degree of stabilisation required)
- Conservative treatment of hip dislocations
- Arthrosis of the hip joint
- Coxalgia or inoperable joint misalignment



Gel Insert to prevent oedema

FEATURES

- Made of breathable Aer-OS fabric
- Lateral and anterolateral spiral support stays
- Front Velcro® fasteners
- Colour-coded Velcro® areas for fastening the straps.
- Elastic calibrated rods, with lateral crossing, for:
 - Prevention of excessive adduction
 - Partial limitation of flexion-extension
- A rigid bar that locks the joint in flexion and neutral abduction during the initial days following a dislocation

Item	M91006N R • M91006N L		
Size	S	M	L
Pelvis circ. cm	72/104	104/137	137/170
Proximal thigh circ. cm	40/58	54/74	68/90
Colour	black		

specify right or left

M91007N R
M91007N L



Adjustable rod (telescopic) for Hipocross 2.0

Item	M91007N R • M91007N L
Size	universal
Colour	black

specify right or left

FEATURES

- Adjustable (telescopic) rod for Hipocross:
 - restrict extension (0° to 120° in 10°-increments)
 - restrict flexion (0° to 120° in 10°-increments)
 - regulate adduction



hipo

Hip orthosis

M2000 R
M2000 L



INDICATIONS

- Pre and/or post-surgery (arthroplasty or hip revision)
- Inoperable or insufficiently consolidated femoral neck fractures
- Abduction and flexion-extension control for patients with potential joint problems

FEATURES

- Anatomically shaped and perfectly adaptable plastic parts
- Modular system that permits replacement and adaptation of parts
- Available in 4 sizes, right and left

Item	MSERIE 2000 R • MSERIE 2000 L			
Size	S	M	L	XL
HIP BONE				
Model right	PR3-2112	PR3-2113	PR3-2114	PR3-2115
Model left	PR3-2122	PR3-2123	PR3-2124	PR3-2125
Circ. waist cm	60/82	80/96	94/115	99/120
Circ. iliac crests cm	68/90	90/110	106/126	111/131
THIGH				
Model right	PR3-2212	PR3-2213	PR3-2214	PR3-2215
Model left	PR3-2222	PR3-2223	PR3-2224	PR3-2225
Proximal circ.	44/53	48/60	57/68	62/76
Distal circ.	34/41	36/45	42/49	46/56
Colour	white			

kafo

Lower leg extension for hip orthosis Hipo

M2320 R regular
M2320 L regular
M2321 R long
M2321 L long



INDICATIONS

- Kafo is an additional device which guarantees the control over minimal rotation movements for the hip orthosis Hipo, in this way it ensures that the joint stays in a natural position



FEATURES

- Improves patient's wearing comfort: lightweight materials, comfortable brace
- The monolateral design reduces the complications of bedsores, especially in geriatric patients
- Maintains the natural position of the limb during hip flexion
- Reduces the risk of dislocation
- Guarantees optimum control of flexion-extension, abduction and intra / extra rotation of the hip

Item	M2320 R/L	M2321 R/L		
Size	Regular	Long		
Limb length (groin - heel) cm	63/75	76/93		
Colour	white			
specify right or left				
Shoe size	S 36/39	M 40/43	L 44/46	XL >46

kinesio

Rods for Hipo

kinesio
STANDARD
M2303



HEIGHT <1,80 M

kinesio
LONG
M2304




HEIGHT >1,80 M

Item	M2303	M2304
Height m	< 1,80	> 1,80

FEATURES

- Abduction/adduction adjustable in steps of 7,5°
- Flexion-Extension settings from 0° to 120° in steps of 15°

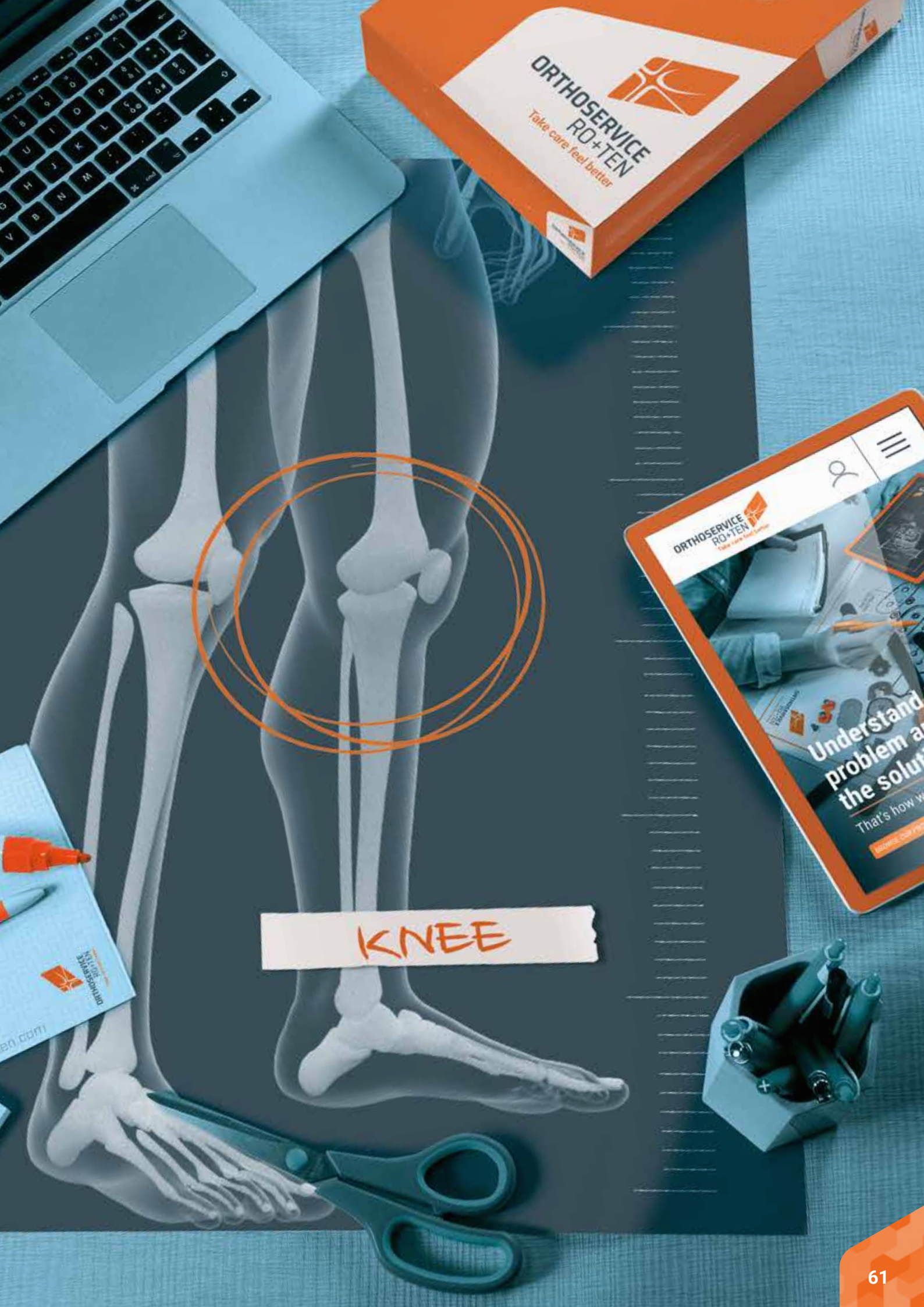

ORTHO SERVICE
RO+TEN
Take care feel better

ORTHO SERVICE
RO+TEN
Take care feel better

Understand
a problem and
the solution

That's how we...

KNEE

ice

Patella strap

M90000



INDICATIONS

- Patellar tendinitis and tendinopathy
- Morbus Osgood-Schlatter disease



FEATURES

- Front elastic strap to allow constant pressure in motion
- Velcro® fastening strap padded for easy application. Soft edge for better comfort especially in knee pit
- Foam pad with hypoallergenic fabric for easiear positioning
- Skin-friendly gel pad concentrating pressure on tendon, slip resistant
- Optional: pad for cold therapy, with Velcro® fastening system replacing pad

Item	M90000	
Size	small	universale
Circumference cm	up to 30	from 31
Colour	black	

fits right and left

OPTION:

MICEGEL Pad for cold therapy with Velcro® fastening system replacing pad

Knee > Patella-stabilising knee supports

object 10

Short knee brace for patella instability

M90010



INDICATIONS

- Knee pain due to bruises
- Early stage of arthritis
- Chondropathy/ Chondromalacia patella
- Mild patella instability
- Prevention of patella luxation during sports

FEATURES

- Multifunctional: reversible for both knees
- C-shaped foam stabiliser for mild instability
- Double-face brace that may be positioned both medial and lateral positions
- Open kneecap
- Black and low profile structure made of orthopaedic neoprene
- Comfortable edges

Item	M90010					
Size	XS	S	M	L	XL	XXL
Circ. middle of the knee cm	30/33	33/36	36/39	39/43	43/48	48/53
Colour	black					

fits right and left

genuFIT 16


Aer-OS knee brace for patella stability

MG1116 R
MG1116 L



INDICATIONS

- Mediolateral patella instability
- Patellofemoral arthrosis
- Gonarthrosis

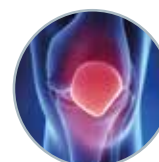

RIGHT: for medial gonarthrosis of the right knee (varus right knee)

for lateral gonarthrosis of the left knee (valgus left knee)

LEFT: for medial gonarthrosis of the left knee (varus left knee)
for lateral gonarthrosis of the right knee (valgus right knee)

FEATURES

- Structure made of Breathable Aer-OS fabric
- Open Kneecap with a lateral J-shaped foam stabiliser
- Medial and lateral spiral stays
- With winches (adjustable)
- Soft-X fabric for kneecap insert for better comfort

Item	MG1116 R+ MG1116 L					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	black					

specify right or left

genu416

Neoprene knee brace for patella stability

M416 R
M416 L



INDICATIONS

- Mediolateral patella instability
- Patellofemoral arthrosis
- Gonarthrosis



FEATURES

- Structure made of bi-layer orthopaedic neoprene
- Open kneecap with a lateral J-shaped foam stabiliser
- Velcro® fastening system
- Medial and lateral spiral stays
- Lined with soft velveteen to avoid sweating and skin irritations

Item	M416 R · M416L					
Size	XS	S	M	L	XL	XXL
Circumf. middle of knee cm	30/33	33/36	36/39	39/43	43/48	48/53
Colour	black					

specify right or left

Knee > Knee orthoses

genuSKILL

Tubular knee brace

genuSKILL 04
M33104



WITH PATELLA STABILISER

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- Chondropathy/ Chondromalacia patella
- Mild patella instability
- Prevention of patella luxation during sports



FEATURES

- Structure made of BreathSkill:
 - soft and breathable
 - 3D fabric
 - high resistant mesh
- CompresSkill back support:
 - stretchy multi-fibre and compression material
 - low profile anti-irritation (knee flexion)
 - fully adaptable
 - maximum adherence
- Medial and lateral stays to keep the brace correctly positioned



Item	M33104					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	grey					

fits right and left

genuFIT

Tubular Aer-OS knee brace

genuFIT 60 MG1160



SHORT

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- First degree sprain to collateral ligaments
- Tendinitis

genuFIT 63 MG1163



SHORT WITH PATELLA STABILISER

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- Chondropathy/ Chondromalacia patella
- Prevention of patella luxation during sports



genuFIT 04 MG1104



WITH PATELLA STABILISER

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- Chondropathy/ Chondromalacia patella
- Mild patella instability
- Prevention of patella luxation during sports

FEATURES

- New anatomical, ultra-flat design for a flexible movement and perfect adherence
- Breathable Aer-OS fabric
- New softer Soft-X fabric for kneepit insert (with no central stitch) for better comfort
- Softer and elastic edge
- **Only Genufit 63:** Reinforced patella stabiliser
- **Only Genufit 04:**
 - Patella opening with foam stabiliser
 - Thin lateral stays to keep the brace correctly positioned



Item	MG1160 • MG1163 • MG1104					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	black					

fits right and left

object

Tubular knee brace

object 60 M460



SHORT

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- First degree sprain to collateral ligaments
- Tendinitis

object 63 M463



SHORT WITH PATELLA STABILISER

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- Chondropathy/ Chondromalacia patella
- Prevention of patella luxation during sports



object 04 M404N



WITH PATELLA STABILISER

INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- Chondropathy/ Chondromalacia patella
- Mild patella instability
- Prevention of patella luxation during sports

FEATURES

- Structure made of bi-layer orthopaedic neoprene
- Lined with soft velveteen to avoid sweating and skin irritations
- Washable
- **Only Object 63:** Patella stabiliser
- **Only Object 04:** Patella pocket for U-shaped stabiliser

Item	M460 · M463					
Size	XS	S	M	L	XL	XXL
Circumf. middle of knee cm	27/30	31/33	34/36	37/39	40/43	44/47
Colour	black and blue					

fits right and left

Item	M404N					
Size	XS	S	M	L	XL	XXL
Circumf. middle of knee cm	30/33	33/36	36/39	39/43	43/48	48/53
Colour	black and blue					

fits right and left

genulastik

Mesh elastic knee brace

genulastik 04 M90104



genulastik PLUS M90104F




INDICATIONS

- Knee pain due to bruises
- Early stage arthrosis
- Chondropathy/ Chondromalacia patella
- Mild patella instability
- Prevention of patella luxation during sports



FEATURES

- Stretch knit fabric featuring areas of different targeted compression
- Anatomically design
- Silicone patella stabiliser
- Soft lateral stays to keep the brace correctly positioned (anti-migration)
- Reinforced silicone edge, softness and traction-resistant during application

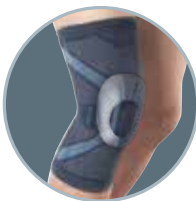

Item	M90104 · M90104F							
Size	0	1	2	3	4	5	6	7
Thigh circ. 14 cm above mid-patella of the kneecap cm	35/38	38/41	41/44	44/47	47/50	50/53	53/56	56/59
Calf circ. 12 cm below the middle of the kneecap cm	25/28	28/31	31/34	34/37	37/40	40/43	43/46	46/49
Colour	M90104 blue · M90104F yellow fluo							

fits right and left



genuSKILL

Knee brace with spiral stays

genuSKILL 08 M33108



TUBULAR
WITH PATELLA STABILISER

INDICATIONS

- Early stage knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)
- Mild mediolateral patella instability
- Patellofemoral arthrosis


genuSKILL 08A M33108A



WRAPAROUND

INDICATIONS

- Early stage knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)

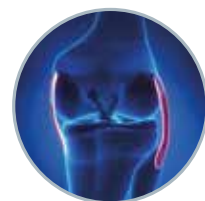

FEATURES

- Structure made of BreathSkill:
 - soft and breathable
 - 3D fabric
 - high resistant mesh
- CompresSkill back support:
 - Stretchy multi-fibre and compression material
 - Ultra-low profile anti-irritation for better comfort (knee flexion)
 - Fully adaptable
 - Perfect adherence
- Medial and lateral stays to keep the brace correctly positioned
- Upper and lower stabilising straps:
 - Symmetrical fastening system to avoid brace rotation when fastening
 - Articulated central buckle for "automatic" adaptation to leg
 - Rubber tab for easy Velcro® opening
- **Only Genuskill 08:**
 - FlexSkill front support (bi-layer technical fabric) with special low-profile anti-irritation welding seam
 - Patella stabiliser
- **Only Genuskill 08A:**
 - Rubber tab for easy Velcro® opening
 - New Velcro® fastening system for easier fastening

Item	M33108 • M33108A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	grey					

fits right and left

genuFIT

Aer-OS knee brace with spiral stays

genuFIT 08 MG1108




34 cm

TUBULAR WITH PATELLA STABILISER

INDICATIONS

- Early stage knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)
- Mild mediolateral patella instability
- Patellofemoral arthrosis

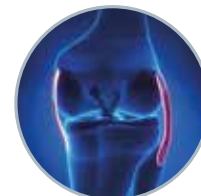

genuFIT 08A MG1108A



34 cm

WRAPAROUND

INDICATIONS

- Early stage knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)

FEATURES

- New anatomical, ultra-flat design for a flexible movement and perfect adherence
- Breathable Aer-OS fabric
- New softer Soft-X fabric for kneepit insert (with no central stitch) for better comfort
- Softer and elastic edge
- New fastening system: stabilization straps with buckles and elastic strap to optimise pressure
- New low profile pockets made of flexible foam, stay and strap covers
- Supporting lateral spiral stays
- **Only Genufit 8:** Patella foam stabiliser
- **Only Genufit 8A:** New double front opening to apply the brace with ease: internal fastening system for prefixing and external fastening system for final fixing. Patella cut-out

Item	MG1108 • MG1108A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	black					

fits right and left

object 08

Neoprene tubular knee brace with spiral stays

M408N




33 cm

TUBULAR WITH PATELLA STABILISER

INDICATIONS

- Early stage knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)
- Mild mediolateral patella instability
- Patellofemoral arthrosis

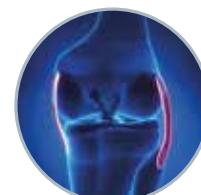

FEATURES

- Structure made of bi-layer orthopaedic neoprene
- Patella pocket for U-shaped stabiliser
- Linear Velcro® fastening system
- Washable
- Lined with soft velveteen to avoid sweating and skin irritations
- Medial and lateral spiral stays

Item	M408N					
Size	XS	S	M	L	XL	XXL
Circ. middle of knee cm	30/33	33/36	36/39	39/43	43/48	48/53
Colour	black and blue					

fits right and left

genuSKILL

Knee orhosis with hinges and F-E control

genuSKILL 15 M33115



WITH PATELLA STABILISER

INDICATIONS

- Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)
- Mild mediolateral patella instability
- Patellofemoral arthrosis



Pair of ultra low-profile polycentric hinges for mediolateral stabilization (possible flexion limit to 60° or 90°)



genuSKILL 15A M33115A



WRAPAROUND

INDICATIONS

- Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)

FEATURES

- Structure made of BreathSkill:
 - soft and breathable
 - 3D fabric
 - high resistant mesh
- CompresSkill posterior support:
 - Stretchy multi-fibre and compression material
 - Ultra-low profile anti-irritation for better comfort (knee flexion)
 - Fully adaptable
 - Perfect adherence
- Upper and lower stabilising straps:
 - Symmetrical fastening system to avoid brace rotation when fastening
 - Articulated central buckle for "automatic" adaptation to leg
 - Rubber tab for easy Velcro® fastening
- **Only Genuskill 15:**
 - FlexSkill front support (bi-layer technical fabric) with special low-profile anti-irritation welding seam
 - Patella stabiliser
- **Only Genuskill 15A:**
 - New front opening with Velcro® to apply the brace with ease

Item	M33115 • M33115A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	grey					

fits right and left

genuSKILL

Knee orthosis with hinges and F-E control

genuSKILL 26
M33126

PATENT: EP3492050



SHORT

genuSKILL 26A
M33126A

PATENT: EP3492050



SHORT WRAPAROUND

genuSKILL 28A
M33128A

PATENT: EP3492050



LONG WRAPAROUND

INDICATIONS

- Functional treatment of first and second sprain injuries of lateral/medial collateral ligament
- Prevention of recurring sprain MCL and LCL injuries during sports
- Mediolateral chronic instability
- Patellofemoral arthrosis
- Instability due to arthrosis. Non-operative knee



Ultra-low polycentric CLICLACK joint easy to adjust using pins and WITHOUT any tool:

- Extension: 0°, 10°, 20°, 30°, 40°;
- Flexion: 45°, 60°, 75°, 90° ;
- Block: 0°, 10°, 20°, 30°

FEATURES

- Structure made of BreathSkill:
 - soft and breathable
 - 3D fabric
 - high resistant mesh
- CompresSkill back support:
 - Stretchy multi-fibre and compression material
 - Ultra-low-profile for a comfortable use (knee flexion)
 - Fully adaptable
 - Perfect adherence
- New straps fastening system:
 - First setting with Y Velcro®
 - Second setting with buckles, rapid ring-and-loop system, winch, Velcro® fastening system
- **Only tubular models (Genuskill 26):**
 - FlexSkill front support (bi-layer technical fabric) with special low-profile anti-irritation welding seam. Patella stabiliser included
- **Only wraparound models (Genuskill 26A, 28A):**
 - New front Velcro® fastening system designed to apply the brace with ease

Item	M33126 (tubular) • PR3-33126A (wraparound) M33128A (wraparound)					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	grey					

fits right and left

genuFIT

Aer-OS knee orthosis with F-E control hinges

genuFIT 15
MG1115



WITH PATELLA STABILISER

INDICATIONS

- Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)
- Mild mediolateral patella instability
- Patellofemoral arthrosis



Polycentric hinges with ultra-low profile (flexion limit to 60° or 90°)

genuFIT 15A
MG1115A



WRAPAROUND

INDICATIONS

- Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)

FEATURES

- New anatomical, ultra-flat design for a flexible motion and perfect adherence
- Breathable Aer-OS fabric
- New softer Soft-X fabric for kneecap insert (with no central stitch) for better comfort
- Softer and elastic edge
- New fastening system: stabilisation straps with buckles and elastic strap to optimise pressure
- **Only Genufit 15:** Open kneecap and foam patella stabiliser
- **Only Genufit 15A:**
 - New double front opening to apply the brace with ease: internal fastening system for prefixing and external fastening system for final fixing. Patella cut-out


Item	MG1115 • MG1115A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	black					

fits right and left



The fabric of our GENUFIT LINE

The material in direct contact with the skin is Aer-OS, which is a three-dimensional weave of Polyester and Elastane threads. Aer-OS mesh is lightweight and water permeable, providing breathability for extra comfort. This elastic and supportive material is also ideal for reducing discomfort and potential skin irritations over longer term use.

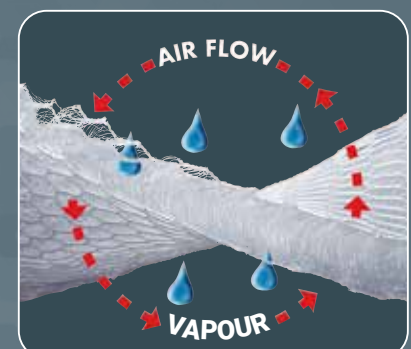

genuFIT

Knee orthosis with polycentric hinges and F-E control

genuFIT 29
MG1129



genuFIT 29A
MG1129A



genuFIT 25
MG1125



genuFIT 25A
MG1125A



INDICATIONS

- Functional treatment of first and second sprain injuries of lateral/medial collateral ligament
- Prevention of recurring sprain MCL and LCL injuries during sports
- Mediolateral chronic instability
- Patellofemoral arthrosis
- Conservative treatment of knee instability



Polycentric joint easy to adjust from outside using pins.
Flexion, extension and block of 0°, 15°, 30°, 60°, 90°



Ultra-low polycentric CLICLACK joint easy to adjust using pins and WITHOUT any tool:
Extension: 0°, 10°, 20°, 30°, 40°;
Flexion: 45°, 60°, 75°, 90° ;
Block: 0°, 10°, 20°, 30°

FEATURES

- New anatomical, ultra-flat design for a flexible movement and perfect adherence
- Breathable Aer-OS fabric
- New softer Soft-X fabric for kneepit insert (with no central stitch) for better comfort
- Softer and elastic edge
- **Only tubular models (Genufit 29, 25):**
 - Open kneecap and foam patella stabiliser
- **Only wraparound models (Genuskill 29A, 25A):**
 - New double front opening to apply the brace with ease: internal fastening system for prefixing and external fastening system for final fixing. Patella cut-out

Item	MG1129 • MG1129A • MG1125 • MG1125A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	black					

fits right and left

genuFIT

Aer-OS knee orthosis with hinges and F-E control

genuFIT 30
MG1130



genuFIT 30A
MG1130A



genuFIT 27
MG1127



genuFIT 27A
MG1127A



Polycentric joint easy to adjust from outside using pins.
Flexion, extension and block of 0°, 15°, 30°, 60°, 90°

INDICATIONS

- Functional treatment of first and second sprain injuries of lateral/medial collateral ligament
- Prevention of recurring sprain MCL and LCL injuries during sports
- Mediolateral chronic instability
- Patellofemoral arthrosis
- Conservative treatment of knee instability



Ultra-low polycentric CLICLACK joint easy to adjust using pins and WITHOUT any tool:
Extension: 0°, 10°, 20°, 30°, 40°;
Flexion: 45°, 60°, 75°, 90°;
Block: 0°, 10°, 20°, 30°

FEATURES

- New anatomical, ultra-flat design for a flexible motion and perfect adherence
- Breathable Aer-OS fabric
- New softer Soft-X fabric for kneepit insert (with no central stitch) for better comfort
- Back elastic insert in proximal area of thigh
- New, softer and elastic edge
- **Only tubular models (Genufit 30, 27):**
 - Open kneecap and foam patella stabiliser
- **Only wraparound models (Genuskill 30A, 27A):**
 - New double front opening to apply the brace with ease: internal fastening system for prefixing and external fastening system for final fixing. Patella cut-out

Item	MG1130 • MG1130A • MG1127 • MG1127A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Colour	black					

fits right and left

object

Neoprene knee orthosis with hinges

object 15 M415N



WITH PATELLA STABILISER

INDICATIONS

- Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)
- Mild mediolateral patella instability
- Patellofemoral arthrosis



Polycentric hinges with ultra-low profile (flexion limit to 60° or 90°)

object 20 M420



WRAPAROUND

INDICATIONS

- Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)

FEATURES

- Structure made of bi-layer orthopaedic neoprene
- Lined with soft velveteen to avoid sweating and skin irritations
- Front Velcro® fastening system
- Washable
- **Only Object 15:** Patella pocket for U-shaped stabiliser
- **Only Object 20:** Kneecap opening

Item	M415N • M420					
Size	XS	S	M	L	XL	XXL
Circumf. middle of knee cm	30/33	33/36	36/39	39/43	43/48	48/53
Colour	black and blue					

fits right and left

immok

Knee orthosis for immobilisation

immok 0°

M8104 | M8105
M8106 | M8107



0°

INDICATIONS

- Post-surgery or post-traumatic immobilisation with knee in extension



immok 20°

M8104/F | M8105/F
M8106/F | M8107/F



20°

INDICATIONS

- Post-surgery or post-traumatic immobilisation with knee in extension of 20°

FEATURES

- Structure made of double-lined soft foam
- Terry cloth interior
- Adjustable medial and lateral panels
- Patella opening
- Washable
- **Only Immok 0°:** medial and lateral rigid stays and back stays made of adjustable light alloy
- **Only Immok 20°:** medial and lateral rigid stays and back stays made of adjustable light alloy with flexion of 20°

Item	M8104	M8105	M8106	M8107
Immobilisation at	0°	0°	0°	0°
Lenght cm	40	50	60	70

Item	M8104/F	M8105/F	M8106/F	M8107/F
Immobilisation at	20°	20°	20°	20°
Lenght cm	40	50	60	70
Colour	black			

fits right and left

go up

Post-operative knee orthosis with graduated movement

go up BASIC
M24045



go up PRO
M24046



TELESCOPIC

INDICATIONS

- Post-operative treatment requiring protection of the knee's flexion-extension (ligament reconstruction, transposition of the anterior tibial apophysis, fractures of the distal third of the femur, fractures of the proximal third of the tibia, patella fractures, tendon injuries of the extensor system)
- Post-traumatic phase of knee sprains
- Valgus and varus osteotomies



New flexion-extension adjustment joint „PUSH&SET“, made of a lightweight polymer material; intuitively and safely adjustable:

- Flexion: from -10° to +120° with intervals of 10°
- Extension: from -10° to +90° with intervals of 10°
- Fast lock joint with specific button, from -10° to +90° with intervals of 10°

FEATURES

- Removable and washable condyle protection pads
- MTP pads to limit brace movement and protect the leg from direct contact with the hinges; lined only on the outside, they can be cut to perfectly adapt to the leg's circumference.
- **Only Go up BASIC:**
 - Lightweight aluminium alloy adjustable hinge with integrated strap loops
 - Possibility of shortening the orthosis using guide marks engraved on the lightweight
- **Only Go up PRO:**
 - Lateral articulated hinge made of a light aluminium alloy. Easy-to-use buttons on the valves for telescopic length adjustment.
 - Valves with strap-loops and structural lock joint

Item	PR3-24045	PR3-24046
Size	universal	universal
Groin-ankle length cm	56	56-65
Colour	grey	

fits right and left

go up OPEN
MG1043



WRAPAROUND VERSION

go up + go up REHAB
MG1042
MG1041



go up T
MG1046



TELESCOPIC


go up pcl
PR3-G1044



PCL

INDICATIONS

- Post-surgery for the protection of knee flexion and extension (ligament reconstruction, transposition of the anterior tibial apophysis, fracture of the distal third of femur, fractures of proximal third of tibia, fractures of patella, tendon injuries of extensor system)
- Post-traumatic stage of knee sprains
- Valgus and varus osteotomies

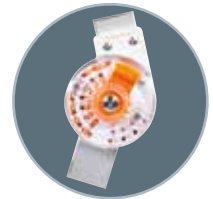

Non-metallic adjusting joint, light and intuitively adjustable

- Flexion: from 0° to +120° with intervals of 10°
- Extension: from -10° to +60° with intervals of 10°
- Fast lock joint with specific button, from 0° to +60° with intervals of 10°

FEATURES

- Structure with hinges made of light aluminium alloy
- Valves with strap-loops and structural hook and-loop system
- Removable and washable condyle protection pads
- MTP Pads to limit brace migration and protect leg from direct contact with hinges
- Lined only externally, cuttable and adaptable to leg circumference
- **Only Go up:** available in a shortened Rehab version
- **Only Go up and Go up OPEN:**
 - Possibility of shortening brace through marks on the hinges
- **Only Go up T:**
 - Adjustable telescopic length with easy-to-use buttons on valves
- **Only Go up PCL:**
 - Inflatable retro-tibial pad to relieve pressure on PCL
 - Front and back straps to ensure the stays position along the median line of leg
 - Removable and washable condyle protection pads

Item	MG1043 • MG1042		MG1041
Size	Regular	Long	Rehab
Limb length groin - ankle cm	< 81	> 81	
Length rods cm	56	65	40
Colour	black		

fits right and left

Item	MG1046	MG1044
Size	universal	universal
Length rods cm	50-65	56
Colour	black	

fits right and left

pluspoint

Functional knee orthosis

pluspoint FAST M22040 R M22040 L



PATENT: EP3492050



FRONT SOCKET

pluspoint 4 M33040 R M33040 L




PATENT: EP3492050



INDICATIONS

- Knee instability and fractures ACL, PCL, MCL and LCL or combined instabilities
- Complex instabilities of the knee joint, traumatic or degenerative
- Pre/post-surgery, rehabilitative or conservative treatment of the aforementioned injuries
- Ligament protection during sports

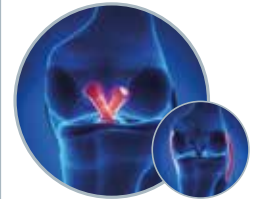

Ultra-low polycentric CLICKCLACK joint easy to adjust using pins and WITHOUT any tool:

Extension: 0°, 10°, 20°, 30°, 40°;
Flexion: 45°, 60°, 75°, 90°;
Block: 0°, 10°, 20°, 30°

FEATURES PLUSPOINT FAST

- Frame:
 - Light-weight and resistant, made of a special aluminum alloy, adjustable (facilitated modeling on thigh and calf)
 - New anatomical design for a perfect adherence
- Front socket for easy fit (no need to put the brace on from the foot)
- Fastening system:
 - Locking buckles with safety button to wear the orthosis without using the Velcro® fastening system (adjusted only at the first application)
 - Divided and reinforced Velcro® straps for better adaptation
 - Soft velveteen straps in MTP padding, seamless and with no adhesives
- Padding of the frame:
 - Hypoallergenic, anti-migration, and multi-grip material
 - Low profile CLICKCLACK to ensure safe positioning (removable for cleaning purposes)
 - Extra-thickness along the calf for optimal comfort
- Pre-installed anatomic medial condyle pad to provide excellent stability
- Two sets of condyle pads of different thicknesses are included

FEATURES PLUSPOINT 4

- New anatomical design for a perfect adherence
- Innovative frame made of a special aluminum alloy, light and stable, adjustable (facilitating modeling on thigh and calf). Matt black painted.
- New buckles (individually removable) made of special and flexible foam with anti-rotation system to apply the brace with ease;
- New reinforced Velcro® straps and divided for better adaptation
- MTP pads made of soft velveteen with no adhesive and no seam
- New and ultra-low profile for polycentric CLICKCLACK joint easy to adjust using wedges WITHOUT any tool: Extension: 0°, 10°, 20°, 30°, 40°; Flexion: 45°, 60°, 75°, 90°; Block: 0°, 10°, 20°, 30°
- Pre-installed anatomic medial condyle pad to provide excellent stability
- Two sets of condyle pads of different thicknesses are included
- Internal frame pad (removable for cleaning) with low profile clickclick to ensure safe positioning.
- Hypoallergenic, anti-migration and multi-grip material

Item	M22040 R • M22040 L M33040 R • M33040 L				
Size	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	39/47	47/53	53/60	60/67	67/75
Colour	black				

specify right or left

pluspoint 3

M9540 R
M9540 L



40 cm

pluspoint 3 short

M9530 R
M9530 L





36,5 cm

SHORT

INDICATIONS

- ACL, MCL, LCL, PCL instability and ruptures, combined instability
- Complex instabilities of knee, traumatic or degenerative
- Pre/Post-surgery both rehabilitative or conservative
- Ligament protection during sports

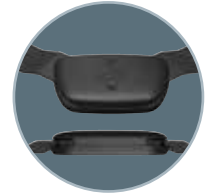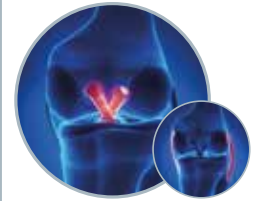

New and ultra-low profile for polycentric joint easy to be adjusted using wedges WITHOUT any tool:
Extension: 0°, 10°, 20°, 30°, 40°;
Flexion: 45°, 60°, 75°, 90°;
Block: 0°, 10°, 20°, 30°

FEATURES PLUSPOINT 3

- New anatomical design for a perfect adherence
- Innovative frame made of a special aluminum alloy, light and stable, adjustable (facilitating modeling on thigh and calf). Matt black painted.
- New buckles (individually removable) made of special and flexible foam with anti-rotation system to apply the brace with ease;
- New reinforced Velcro® straps and divided for better adaptation
- MTP pads made of soft velveteen with no adhesive and no seem Pre-installed anatomic medial condyle pad to provide excellent stability
- Two sets of condyle pads of different thicknesses are included
- Internal frame pad (removable for cleaning) with low profile clickclick to ensure safe positioning.
- Hypoallergenic, anti-migration and multi-grip material

Item	M9540 R • M9540 L M9530 R • M9530 L				
Size	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	39/47	47/53	53/60	60/67	67/75
Colour	black				

specify right or left

genuFEEL OA

Gonarthrosis orthosis

M24014 R
M24014 L



PATENT: EU, US, CN
pending



INDICATIONS

- Moderate unicompartmental gonarthrosis
- Post-operative cartilage or meniscus surgery



Flexion-extension adjustment with simplified joint
Extension:
5°, 10°, 15°, 20° or 25°
Flexion:
45°, 60°, 75° or 90°


FEATURES

- Patented fastening and tensioning system:
 - Interlocking buckles and tensioning levers make it fast and easy
 - Ring unloads brace pressure for better comfort
- Efficient:
 - Three-point pressure system lightens the load on the painful side of the knee
 - Support when needed: tensioned while walking and in stand-by when seated
- Breathable elastic fabric Aer-OS light:
 - Front opening for an easy fit
 - Soft pocket tips to facilitate fastening
- Technopolymer valve for optimal pressure distribution and effective anchorage to the limb
- Single-sided steel stay with flexion (20°, 40°) and extension (0°, 10°) limitation
- Polymer stay with integrated unloader ring, for correct positioning of the brace
- Gel pad for additional protection against pressure from the thrust ring
- Padding for additional valve protection
- Velcro® strap adjustment

Item	M24014 R · M24014 L						
Size	XS	S	M	L	XL	XXL	3XL
Circumf. 15 cm above knee center cm	33/39	39/45	45/52	52/58	58/66	67-74	74-81
Colour	grey						

RIGHT = for medial gonarthrosis of the right knee (varus right knee) or for lateral gonarthrosis of the left knee (valgus left knee)

LEFT = for medial gonarthrosis of the left knee (varus left knee) or for lateral gonarthrosis of the right knee (valgus right knee)

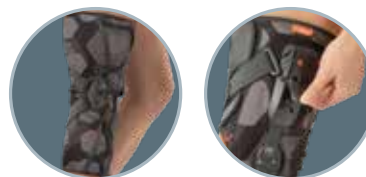

How do our knee braces for osteoarthritis work

- **RIGHT** =
 - for medial gonarthrosis of the right knee (varus right knee)
 - for lateral gonarthrosis of the left knee (valgus left knee)
- **LEFT** =
 - for medial gonarthrosis of the left knee (varus left knee)
 - for lateral gonarthrosis of the right knee (valgus right knee)



KOA

Orthosis for gonarthrosis

M20014



INDICATIONS


- Moderate unicompartmental gonarthrosis
- Post-surgery cartilage or meniscus treatment



A single-sided stay made of light aluminum alloy with limit extension of 0°, 10°, 20°, 30°, 40° and flexion of 45°, 60°, 75°, 90°

FEATURES

- Patented strap system:
 - effective single-sided reduction of compression using a three-point system of load
 - the system comes into operation when required: decreased pressure when sitting and tension when standing
 - easy adjustable tension using winch system
 - easy regulation
- Stretchy PlusComfort fabric: cotton inside for a maximum comfort and polyester outside for maximum resistance
- Light foam valves adaptable to leg with buckles locking system
- Fixing buckles with built-in blocking pull for easy tension, magnet to secure pull after using it
- Technogel condyle pad to dissipate pressure of straps
- Velcro® easy-to-adjust straps
- Thigh and calf straps with stretchy element for better comfort when moving and movable pad with Velcro®

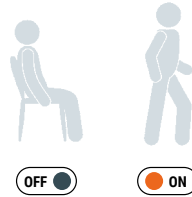

Item	M20014					
Size	XS	S	M	L	XL	XXL
Circumf. middle of the knee cm	30/33	33/36	36/39	39/43	43/48	48/54
Colour	grey					

specify right or left

RIGHT = for medial gonarthrosis of the right knee (varus right knee) or for lateral gonarthrosis of the left knee (valgus left knee)

LEFT = for medial gonarthrosis of the left knee (varus left knee) or for lateral gonarthrosis of the right knee (valgus right knee)

ORTHO-A

Orthosis for gonarthrosis

PR3-91014 R
PR3-91014 L



INDICATIONS

- Moderate unicompartmental gonarthrosis
- Post-surgery cartilage or meniscus treatments



Wedges to limit extension of 0°, 5°, 10°, 15°, 20° or 25° and flexion to 45°, 60°, 75° or 90°

FEATURES

- Light structure made of foam and aluminum alloy
- Single-sided hinge for improved and effective brace application. Lever mechanism to relieve mediolateral knee
- Hoop-and-look straps to apply the brace without using Velcro® straps (first setting)
- Soft, removable and washable pads
- Included accessories: - Additional condyle pad
- Double regulation joint of medial and lateral load
- Locking buckles with safety button

Item	M91014 R • PR3-91014 L
Size	universal
Colour	grey

specify right or left

RIGHT = for medial gonarthrosis of the right knee (varus right knee) or for lateral gonarthrosis of the left knee (valgus left knee)

LEFT = for medial gonarthrosis of the left knee (varus left knee) or for lateral gonarthrosis of the right knee (valgus right knee)





LOWER LEG

ANKLE



Calf-ankle > Calf supports

mioFIT 34

Calf support in Aer-OS fabric

mioFIT 34 MA1134



INDICATIONS

- Muscular contractions and tears of the calf
- Aid to resuming sporting activity after muscle trauma

FEATURES

- Made from breathable Aer-OS fabric

Item	MA1134			
Size	S	M	L	XL
Circumf. calf cm	33/35	35/38	38/43	43/48
Colour	black			

fits right and left

Calf-ankle > Ankle supports

malleoLASTIK

Elastic knit ankle support

malleoLASTIK 37 M90137



INDICATIONS

- 1st degree tibiotarsal sprains
- Peroneal or instep tendinopathy

malleoLASTIK PLUS M90137F



FEATURES

- Stretch knit fabric featuring areas of different targeted compression
- Anatomically designed
- Silicone inserts shaped to fit the malleoli
- The upper edging is strengthened with silicone trim so the support feels softer and keeps its shape when being pulled on

Item	M90137 • M90137F						
Size	0	1	2	3	4	5	6
Ankle circumf. cm	15/17	17/19	19/21	21/23	23/25	25/27	27/29
Colour	M90137 blue • M90137F yellow fluo						

fits right and left

malleoFIT 37

Ankle support in Aer-OS fabric

MA1137



INDICATIONS

- 1st degree tibiotarsal sprains
- Peroneal or instep tendinopathy

FEATURES

- Made from breathable Aer-OS fabric
- Open heel

Item	MA1137			
Size	S	M	L	XL
Ankle circumf. cm	15/20	20/25	25/30	30/36
Colour	black			

fits right and left

object

Neoprene ankle support

object 37
M8237



INDICATIONS

- 1st degree tibiotarsal sprains
- Peroneal or instep tendinopathy

object 80
M8280



WITH SPIRAL STAYS

INDICATIONS

- Minor tibiotarsal sprains 1st degree
- Moderate inflammatory conditions (peroneal tendinopathy)
- Mild joint pain
- Tibiotarsal arthrosis

FEATURES

- Double-lined orthopaedic neoprene
- Open heel
- Washable
- **Only Object 80:**
 - Spiral stays for support on both sides
 - Fastens around the ankle with Velcro® straps

Item	M8237 • M8280			
Size	S	M	L	XL
Ankle circumf. cm	15/20	20/25	25/30	30/36
Colour	M8237 beige • M8280 blue			

fits right and left

Calf-ankle > Ankle supports with figure-8 straps

malleoLITE

Ankle support with elasticated figure of 8 straps

M33080



INDICATIONS

- 1st degree tibiotarsal sprains
- Peroneal or instep tendinopathy
- Mild joint pain



FEATURES

- Made of Sensitive® fabric which is stretchy, breathable, rip-proof and antistatic
- The anatomic and open design make it extremely easy to put on
- Fastens at the back with Velcro®
- Elasticated straps cross over the instep

Item	M33080		
Size	S	M	L
Circumf. of ankle from the malleoli cm	24-28	28-33	33-38
Colour	black		

fits right and left

malleoFIT 81

Ankle support in Aer-OS with elasticated figure of 8 straps and spiral stays

MA1181



WITH SPIRAL STAYS

INDICATIONS

- Conservative treatment following a grade 2 sprain trauma
- Acute tendinopathies
- Post-surgical rehabilitation
- Aid to resuming sporting activity after grade 2 and 3 sprain trauma
- Chronic joint pain caused by tibiotarsal arthrosis



FEATURES

- Breathable Aer-OS fabric construction
- Top closure system with stabilisation strap and buckle
- Spiral reinforcement slats on both sides
- Cross straps on the instep, with sub-plantar passage for maximum stability, with Velcro® closure
- Opening at the heel

Item	MA1181			
Size	S	M	L	XL
Circ. instep to heel cm	26/29	29/32	32/35	35/38
Colour	black			

fits right and left

object 81

Neoprene ankle support with elasticated figure of 8 and spiral stays

M8281



WITH SPIRAL STAYS

INDICATIONS

- Conservative treatment following grade 2 sprain trauma
- Acute tendinopathies
- Postoperative rehabilitation
- Aid to resuming sporting activity after grade 2 and 3 sprain trauma
- Chronic joint pain caused by tibiotarsal arthrosis



FEATURES

- Double-lined orthopaedic neoprene
- Spiral reinforcing rods on both sides
- Open heel
- Crossover instep straps with Velcro® fastenings pass under the sole of the foot and around the malleoli for maximum stability
- Fastens at the top with a Velcro® strap

Item	M8281			
Size	S	M	L	XL
Circ. instep to heel cm	26/28	29/31	32/34	35/37
Colour	black			

fits right and left

OSC active

Ankle brace with non-elastic figure of 8 straps

M2036 R
M2036 L



LACE FASTENING AND SIDE REINFORCEMENTS

INDICATIONS

- Conservative treatment after grade 2 sprain
- Acute Tendinopathy
- Post-surgical Rehabilitation
- Support for sports activities following a grade 2 and 3 sprain
- Chronic arthralgia of ankle due to osteoarthritis



FEATURES

- Structure in nylon with double layer around eyelets
- Protective tongue on the instep
- Crossed nylon tightening straps with ends in Velcro®
- Ankle guards in plastic to ensure greater stability
- Open heel
- Washable
- Laces

Item	M2036 R - M2036 L			
Size	S	M	L	XL
Height cm	34	36	39	41
Shoe size	35/39	40/43	41/46	> 46
Colour	black			

fits right and left

malleorapid

Ankle brace with steel stays and figure-8 elastic straps

M24081



WITH STEEL RODS

INDICATIONS

- Conservative treatment following grade II sprain injuries
- Acute tendinopathies
- Post-surgical functional recovery
- Supportive in the resumption of sports following grade II and III injuries
- Chronic arthritic arthralgias of the tibio-tarsus



FEATURES

- Three-dimensional mesh fabric for tightness and maximum breathability
- Additional lateral and medial reinforcement fabric to protect against contact with the stays
- Moldable medial and lateral stabilising stays made of steel rods
- Open heel with Velcro® closure for easy donning
- Drawstring adjustment system with quick-release buckle for fast and precise adaptation of the brace to the morphology of the ankle.
- Lycra pocket for storing the lanyard
- Elastic crossover straps with Velcro® closure on the instep for maximum stability
- Anatomical and low-profile design to fit most shoes

Item	M24081		
Size	1	2	3
Circumference ankle cm	19/22	22/26	26/30
Colour	grey		

fits right and left

malleostrong PRO

Bivalve ankle brace with foam cell and latero-medial strap

MA1028 R
MA1028 L



INDICATIONS

- Grade 2 and 3 sprains
- Post-surgical functional recovery
- Functional recovery following malleolar fractures
- Chronic instability of the tibiotarsal and in sporting activity



FEATURES

- Thin Breath-o-prene® structure
- Medial and lateral plastic valves are inserted in the brace and are protected by soft MTP cell
- Anatomical
- Additional rear cell for improved comfort
- Rear elastic fastening strap
- Latero-medial reinforcing and fastening strap (can be shortened to fit)

Item	MA1028 R - MA1028 L				
Size	XS	S	M	L	XL
Shoe size	30/34	35/38	39/42	43/47	> 47
Colour	black				

specify right or left

Calf-ankle > Ankle braces with two support shells

airstrong

Bivale ankle brace with inflatable air cell

MA1029



INDICATIONS

- Grade 2 and 3 sprains
- Post-surgical functional recovery
- Functional recovery following malleolar fractures
- Chronic instability of the tibiotarsal and in sporting activity



PUMP INCLUDED

FEATURES

- Inflatable air pads
- Anatomically shaped lightweight durable plastic bivale structure
- Velcro® adjustment to fit heel width
- Easy to wear thanks to full opening
- Pump included

Item	MA1029
Size	universal
Colour	white

fits right and left

Calf-ankle > Night-time positioning splints

malleonite

Night positioning ankle brace

M90062



INDICATIONS

- Plantar fasciitis
- Achilles tendinopathy
- Foot drop
- Pain upon awakening

FEATURES

- The hinged polymer splint can be positioned for personalised support using the adjustable plantar flexion strap
- Comfortable, soft cell
- The adjustable Velcro® calf and foot straps are padded for optimum comfort
- Extra strap included that can be used at ankle height

Item	M90062	
Size	S/M	L/XL
Shoe size	36/42	43/48
Colour	black	

fits right and left

equal

Walker

equal 400 M400/OV



INDICATIONS

- Conservative treatment of of malleolar fractures
- Severe sprains (grade 3)
- Post-surgical phase of osteosynthesis of malleolar fractures
- Ligamentous reconstructions of the ankle and of the tenorrhaphy of the Achilles tendon
- Stable fractures of the foot, ankle, tibia and fibula distal

CONTRAINDICATIONS

The application of this brace is advised only when it is documented that the fracture is stable and there are acceptable limits of angular and rotational misalignments

equal 450 M450/OV



SHORT

INDICATIONS

- Soft tissue injuries
- Mild to severe ankle sprains
- Injuries and pain in the forefoot, foot, and heel
- Stable fractures in the foot and/or ankle
- Post-operative care



OPTION: M501

FEATURES

- Structure made from lightweight, durable plastic
- Non-slip rubber sole
- Inner cell of the sole is made of non-toxic Evazote
- Struts held in place with Velcro® for a firm grip on the cell
- Soft foam pad with double lining in brushed nylon for Velcro® grip
- Velcro® fastening straps
- Washable cell

Item	M400/OV - M450/OV				
Size	XS	S	M	L	XL
Shoe size	33/36	36/39	39/44	44/47	> 47
Colour	black				

fits right and left

OPTION:

M501 wedges for the heel, specify right or left

airstep		High	Short	1 Inflatable Air cell	2 Inflatable Air cells	Diabetic set included	Option: 501 Wedges
M500	Airstep Walker	●		●			●
M510	Airstep Walker Short		●	●			●
M520	Airstep Tight Walker Diabetic	●			●	●	
M530	Airstep Walker Diabetic	●		●		●	
M550 N	Airstep Tight Walker	●			●		●
M560 N	Airstep Tight Walker Short	●	●		●		●

airstep walker

Walker with inflatable air cell

M500



INDICATIONS

- Conservative treatment of malleolar fractures
- Severe sprains (grade 3)
- Post-surgical phase of osteosynthesis of malleolar fractures
- Ligamentous reconstructions of the ankle and of the tenorrhaphy of the Achilles tendon
- Stable fractures of the foot, ankle, tibia and fibula distal

CONTRAINDICATIONS

The application of this brace is advised only when it is documented that the fracture is stable and there are acceptable limits of angular and rotational misalignments



OPTION: M501



OPTION: M505

M510



SHORT

INDICATIONS

- Soft tissue injuries
- Mild to severe ankle sprains
- Injuries and pain in the forefoot, foot, and heel
- Stable fractures in the foot and/or ankle
- Post-operative care

FEATURES

- Reinforcing shells on the front and back, for increased stability
- Inflatable air pads for optimum fit to the limb and improved stability
- Structure in lightweight, sturdy plastic

Item	M500 • M510			
Size	S	M	L	XL
Shoe size	35/39	39/44	44/47	>47
Colour	grey			

fits right and left

OPTION:

M501 Wedges for walker, specify right or left

OPTION:

M505 Diabetic Set, fits right and left

500/P Pump included

airstep walker diabetic

Walker with inflatable air cell and Diabetic Set

M520



TIGHT WITH 2 INFLATABLE AIR CELLS

M530



FEATURES

M520

- Adaptable foam inserts for partial pressure relief from the sole
- Special stockings for diabetes mellitus and the associated diabetic foot syndrome
- Front reinforcement shell for stable hold of the foot
- Optimum wearing comfort thanks to two individual inflatable air pads in the inner shoe
- Basic body made of durable, light plastic
- Comfortable wear, thanks to the foam cell
- Correct fixing thanks to circular Velcro® fastening
- Optimised tilting sole
- Optional cell for further adaptation to the leg
- Diabetic set included

M530

- Adaptable foam inserts for partial pressure relief of the foot sole
- Special stockings for diabetes mellitus and the associated diabetic foot syndrome
- Reinforcing shells on the front and back, for increased stability
- Inflatable air pad for optimum fit to the limb and improved stabilisation
- Structure in lightweight, sturdy plastic
- Diabetic set included

INDICATIONS


- Particularly suited for use with patients with sensitivities, for example, diabetes mellitus and the associated diabetic foot syndrome
- Ulcer on the metatarsus, malleolus, hallux or heel
- Charcot foot

CONTRAINDICATIONS

The application of this brace is advised only when it is documented that the fracture is stable and there are acceptable limits of angular and rotational misalignments



INCLUDED: M505

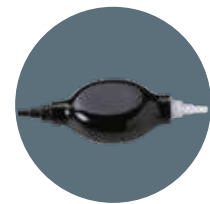

INCLUDED: PUMP

Item	M520 • M530		
Size	S	M	L
Shoe size	35/39	39/44	>44
Colour	grey		

fits right and left

INCLUDED:

M505 Diabetic Set, fits right and left

INCLUDED:

Pump

airstep tight walker

Walker with two inflatable air cells

M550N



M560N



SHORT

INDICATIONS

- Soft tissue injuries
- Mild to severe ankle sprains
- Injuries and pain in the forefoot, foot, and heel
- Stable fractures in the foot and/or ankle
- Post-operative care

CONTRAINDICATIONS

The application of this brace is advised only when it is documented that the fracture is stable and there are acceptable limits of angular and rotational misalignments



OPTION: M501



OPTION: M505

FEATURES

- Reinforcing shells on the front and back, for increased stability
- Inflatable air pads for optimum fit to the limb and improved stability
- Structure in lightweight, sturdy plastic; terminal part in rubbery material
- Integrated pump to inflate/deflate the cell
- Sole in non-slip material
- Optional cell for additional adaptation to the leg

Item	M550N • M560N			
Size	S	M	L	XL
Shoe size	35/39	39/44	44/47	>47
Colour	grey			

fits right and left

OPTION:

M501 Wedges for walker, specify right or left

OPTION:

M505 Diabetic Set, fits right and left

r.o.m. walker Walker with range of motion

M600



INDICATIONS

- Malleolar fractures conservative treatment
- Grade 3 ankle sprains
- Post-surgical care for osteosynthesis of malleolar fractures, ankle ligament reconstruction and Achilles tendon tenorrhaphy
- Walking recovery in consolidation delay of tibial fractures
- Charcot foot

CONTRAINDICATIONS

The application of this brace is advised only when it is documented that the fracture is stable and there are acceptable limits of angular and rotational misalignments

FEATURES

- Structure made of durable, light plastic
- Non-slip rubber sole
- Inner cell of the sole made from non-toxic Evazote®
- Struts with Velcro for cell grip
- Soft foam pad with double lining in brushed nylon for Velcro® grip
- Velcro® fastening tapes
- Washable cell

Item	M600		
Size	S	M	L
Shoe size	36/39	39/44	44/47
Plantar flexion	0° - 7,5° - 15° - 22,5° - 30° - 37.5° - 45°		
Dorsal flexion	0° - 7,5° - 15° - 22,5° - 30° - 37.5° - 45°		
Colour	black		

fits right and left

Calf-ankle > Ankle foot orthoses

carbon afo Carbon-fibre ankle foot orthosis

**MKTAFO R
MKTAFO L**



INDICATIONS

- Antalgic gait foot
- Ictus
- Multiple sclerosis
- Neurological illnesses
- Amputation of forefoot

CONTRAINDICATIONS

- Macroscopic deformity of the foot and ankle
- Acute edema
- From moderate to severe spasticity
- Contracture of front and rear tibial muscles that cannot be healed with the simple apparatusposture

FEATURES

- Spiral structure for resistance and elasticity; pre-loaded to provide constant support to the foot
- Structure in reinforced carbon fibre for excellent support. Sole in thin carbon fibre, that can be adapted to different shoes
- Light
- Calf Velcro® closure, with return loop
- Calf cell in MTP finished in technical fabric in contact with the body
- Washable
- Functional calf grip: right mix of flexibility (to permit flexible adjustment to different calf circumferences) and stiffness (to avoid twisting during use)

Item	MKTAFO R --MKTAFO L		
Size	S	M	L
Height cm	36	39	42
Shoe size	33/38	38/43	43/48
Colour	black		

specify right or left

afo

Ankle foot brace

afo light

M9050 R
M9050 L



afo extrastrong

M9051 R
M9051 L



EXTRASTRONG

INDICATIONS

- Foot drop
- Light to moderate spastic equinus

FEATURES

- Polypropylene structure with rounded edges
- Soft cell for protection of the tibia and the calf (MTP cell, with light velour covering on both sides)
- Velcro® fastening strap at calf height
- Velcro® fastening on the instep
- **Only afo light:** Thermoformable

Item	M9050 R • M9050 L • M9051 R • M9051 L			
Size	S	M	L	XL
Height cm	28	29	34	39
Shoe size	35/39	40/43	41/44	> 44
Colour	M9050 R • M9050 L white M9051 R • M9051 L blue			

specify right or left

affix

Brace for foot drop

affix

M20050



affix OPTION

M20051 R
M20051 L



INDICATIONS

- Flaccid paralysis of the foot
- Light to moderate spastic equinus

FEATURES

- To be worn **ONLY** with lace-up shoes
- Low profile
- Breathable micropore fabric
- A soft pad attaches the support to the shoe
- Tensioning elastic to adjust the degrees of flexion
- Gel cell over the Achilles tendon
- Fastens with Velcro® straps
- Additional midfoot band for wearing with bare feet (optional)

Item	M20050			
Size	XS	S	M	L
Ankle circumference cm	13/17	17/21	21/25	25/29
Colour	black			

fits right and left

Midfoot band for bare feet

Item	M20051 R • M20051 L		
Size	S	M	L
Midfoot circumference cm	17/21	21/25	25/27
Colour	black		

specify right or left



FOOT



heelift

Suspension boot for heel pressure ulcer prevention and management - Distributed by Orthoservice

Classic Smooth Interior

MHL9041 | MHL9164
MHL7702



Smooth interior

Classic Convoluted Interior

MHL7603 | MHL9348



Convoluted interior

Glide Smooth Interior

MHL6477 | MHL6279
MHL6460



Smooth interior

Glide Convoluted Interior

MHL6484 | MHL6453



Convoluted interior

Glide Ultra

MHL5500 | MHL5203
MHL5401



Ultra

INDICATIONS

Prevention of pressure ulcers and all case histories of heel pressure ulcers or injuries/postoperative wounds of the heel, including high-risk patients with two or three of the following clinical characteristics:

- a Braden score of 18 or less
- prolonged immobilisation
- diabetes with or without neuropathy

Model	Size	Calf circumference measured 23 cm above the floor	Item
Heelift Classic Smooth Interior	Petite	6-10 in (15-25 cm)	MHL9041
	Standard	10-15 in (25-38 cm)	MHL9164
	Bariatric	15-23 in (38-58 cm)	MHL7702
Heelift Classic Convoluted Interior	Petite	6-10 in (15-25 cm)	MHL7603
	Standard	10-15 in (25-38 cm)	PR5-HL9348
Heelift Glide Smooth Interior	Petite	6-10 in (15-25 cm)	MHL6477
	Standard	10-15 in (25-38 cm)	MHL6279
	Bariatric	15-23 in (38-58 cm)	MHL6460
Heelift Glide Convoluted Interior	Petite	6-10 in (15-25 cm)	MHL6484
	Standard	10-15 in (25-38 cm)	MHL6453
Heelift Glide Ultra	Petite	6-10 in (15-25 cm)	MHL5500
	Standard	10-15 in (25-38 cm)	MHL5203
	Bariatric	15-23 in (38-58 cm)	MHL5401

OPTION: MHL9898 Anti-Rotation Wedge

heelift

Suspension boot for heel pressure ulcer prevention and management - Distributed by Orthoservice

Afo Smooth Interior MHL8105



Afo Convoluted Interior MHL8204



Afo Ultra MHL5302



INDICATIONS

Prevention of pressure ulcers and all case histories of heel pressure ulcers or injuries/postoperative wounds of the heel, including high-risk patients with two or three of the following clinical characteristics:

- a Braden score of 18 or less
- prolonged immobilisation
- diabetes with or without neuropathy



Smooth interior




Convoluted interior




Ultra

Model	Size	Item
Heelift AFO Smooth Interior	Universal	MHL8105
Heelift AFO Convoluted Interiors	Universal	MHL8204
Heelift AFO Ultra	Universal	MHL5302

OPTION: MHL9898 Anti-Rotation Wedge

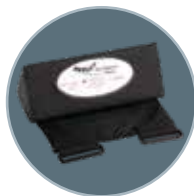

Traction MHL9321

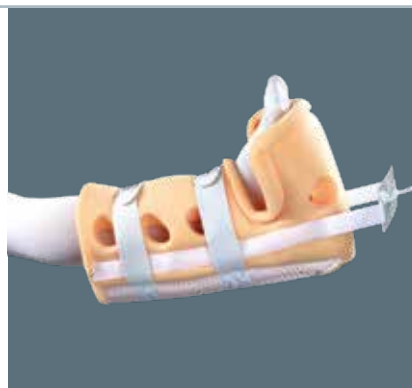

Model	Size	Item
Heelift Traction	Universal	MHL9321

heelift

Suspension boot for heel pressure ulcer prevention and management - Distributed by Orthoservice

FEATURES	HEELIFT CLASSIC		HEELIFT GLIDE			HEELIFT AFO		
	Smooth	Convolved	Smooth	Convolved	Ultra	Smooth	Convolved	Ultra
Suspended heel support	●	●	●	●	●	●	●	●
Opening for visual inspection of the heel	●	●	●	●	●	●	●	●
Durable exterior			●	●	●	●	●	●
Ventilation holes	●	●	●	●	●	●	●	●
Nylon support	●	●	●	●	●	●	●	●
Toe protection			●	●	●	●	●	●
Soft fastening straps	●	●	●	●	●	●	●	●
Forefoot strap			●	●	●	●	●	●
Semi-rigid rear splint						●	●	●
Smooth interior	●		●			●		
Convolved interior		●		●			●	
Inner liner					●			●



Heelift is supplied in d₂w certified biodegradable packaging

talufIT

Shock absorber heel cups

talufIT 50 MF1350



INDICATIONS

- Tallonitis and anterior talalgia
- Sever-Haglund disease
- Achilles tendinopathy
- Calcaneal spur
- Retrocalcaneal Bursitis



talufIT 52 MF1352



REINFORCED

FEATURES

- Made of scented and elastic polymer in a double-grid mesh
- Anatomically contoured
- Easily insertable into every shoe
- **Only Taufit 50:** Ideal for those who remain standing or walk for many hours
- **Only Taufit 52:** Ideal for sports activities

Item	Taufit 50 MF1350		Item	Taufit 52 MF1352		
Size	2	3	Size	1	2	3
Body weight	< 80 kg	> 80 kg	Body weight	< 50 kg	< 80 kg	> 80 kg
Colour	transparent		Colour	anthracite		

fits right and left, supplied in pairs

fits right and left, supplied in pairs

talusoft04

Heel cups with honeycomb heel relief

MF1304



WITH CALCANEAR SPUR

INDICATIONS

- Calcaneal sprur
- Tallonitis and anterior talalgia
- Sever-Haglund disease
- Achilles tendinopathy
- Retrocalcaneal bursitis
- Prevention of talalgia during sports activities



FEATURES

- Anatomical structure made of viscoelastic material
- Ideal for city shoes, work shoes and sports shoes
- Area in contact with the foot is covered with soft material
- Relief area with honeycomb structure to accommodate the calcaneal spur

Item	PR4-F1304		
Size	S	M	L
Shoe size	34/38	39/42	43/47
Colour	anthracite		

supplied in pairs

hallus valgus

Hallux valgus retractor

M360 R
M360 L



INDICATIONS

- Hallux valgus (bunions)
- Big toe sprain



FEATURES

- The inside of the plastic frame is padded
- Tensioning straps with Velcro® fastening
- Soft cell for the straps

Item	M360 R • M360 L		
Size	S	M	L
Shoe size	34/36	37/40	41/46
Colour	white		

specify right or left

havar

Dynamic bunion splint

M21361



INDICATIONS

- Hallux valgus (bunion)
- Mild or moderate malpositioning of the big toe
- Bursitis in the metatarsophalangeal joint of the big toe



FEATURES

- The polymer splint is hinged at the metatarsophalangeal joint.
- Soft foam cell
- Adjustable Velcro® straps
- Suitable for wearing at night, barefoot or in comfortable shoes

Item	M21361
Size	universal
Colour	white

fits right and left

podifer

Insole

podifer 20

M90020



PHLEBOLOGICAL WITH TRANSVERSE REINFORCEMENT

INDICATIONS

- Vasculopathy
- Flatfoot or high arched foot
- Metatarsalgia



The presence of the transverse reinforcement supports the metatarsals and cushions the contact with the floor, in addition it strengthens the pump function of the insole, improving the venous return and, at the same time, stimulating the reflex zones for heart and lungs.

FEATURES

- Top layer of fabric made with woven silver threads with natural antibacterial and odour-inhibiting effect
- The middle visco-elastic layer with form memory effect, in an optimised thickness, aids venous return thanks to the pump effect
- The bottom layer is made of PU and acts as a protective layer for the layers above, providing protection against wear and deformation. It also makes it easier to position the insole in footwear
- Lower shell made of rigid PU for the stabilisation and support of the foot in pronation and supination

Item	M90020						
Shoe size	35	36	37	38	39	40	41
Colour	black						

supplied in pairs

podifer 30

M90030



PHLEBOLOGICAL

INDICATIONS

- Vasculopathy
- Flatfoot or cavus foot



FEATURES

- Top layer of fabric made with woven silver threads with natural antibacterial and odour-inhibiting effect
- The middle visco-elastic layer with form memory effect, in an optimised thickness, aids venous return thanks to the pump effect
- The bottom layer is made of PU and acts as a protective layer for the layers above, providing protection against wear and deformation. It also makes it easier to position the insole in footwear
- Lower shell made of rigid PU for the stabilisation and support of the foot in prone supination

Item	M90030						
Shoe size	40	41	42	43	44	45	46
Colour	black						

supplied in pairs

podifer 40

M90040



REHUMATIC

INDICATIONS

- Hammer toe, in the initial stages
- Bunions, in the initial stages
- Arthralgia
- Metatarsalgia
- Flatfoot or cavus foot



The special EVA layer automatically shapes itself according to the pressure it is exposed to, filling out the areas exposed to lower pressure, thus ensuring increased wearing comfort.

FEATURES

- Top layer of fabric made with woven silver threads with natural antibacterial and odour-inhibiting effect
- The middle visco-elastic layer with form memory effect, in an optimised thickness and with damping function
- The bottom layer is made of PU and acts as a protective layer for the layers above, providing protection against wear and deformation. It also makes it easier to position the insole in footwear
- Lower shell made of rigid PU for the stabilisation and support of the foot in prone supination

Item	M90040											
Shoe size	35	36	37	38	39	40	41	42	43	44	45	46
Colour	black											

supplied in pairs

podifer

Insole

podifer 50 M90050



CUSHIONING WITH TRANSVERSE REINFORCEMENT

INDICATIONS

- Inflammation of the heel
- Plantar fasciitis
- Flatfoot or cavus foot
- Metatarsalgia



The presence of the transverse reinforcement supports the metatarsals and cushions the contact with the floor, in addition it strengthens the pump function of the insole, improving the venous return and, at the same time, stimulating the reflex zones for heart and lungs.

FEATURES

- Top layer made from PA and elastane, special fabric with sections containing higher thread densities in order to increase resistance to wear
- The Podo-Dry felt protects the health of the foot thanks to leading sweat away from the foot
- The middle visco-elastic layer with form memory effect, in an optimised thickness and with damping function
- The bottom layer is made of PU and acts as a protective layer for the layers above, providing protection against wear and deformation. It also makes it easier to position the insole in footwear
- Lower shell made of rigid PU for the stabilisation and support of the foot in pronation and supination
- Shock absorbing material in the heel region to cushion when setting the heel down

Item	M90050											
Shoe size	35	36	37	38	39	40	41	42	43	44	45	46
Colour	black											

supplied in pairs

podifer 60 M90060



CUSHIONING

INDICATIONS

- Inflammation of the heel
- Plantar fasciitis
- Flatfoot or cavus foot



FEATURES

- Top layer made from PA and elastane, special fabric with sections containing higher thread densities in order to increase resistance to wear
- The Podo-Dry felt protects the health of the foot thanks to leading sweat away from the foot
- The middle visco-elastic layer with form memory effect, in an optimised thickness and with damping function
- The bottom layer is made of PU and acts as a protective layer for the layers above, providing protection against wear and deformation. It also makes it easier to position the insole in footwear
- Lower shell made of rigid PU for the stabilisation and support of the foot in pronation and supination
- Shock absorbing material in the heel region to cushion when setting the heel down

Item	M90060											
Shoe size	35	36	37	38	39	40	41	42	43	44	45	46
Colour	black											

supplied in pairs

stepper

Postoperative shoes

stepper SURG M23001



FOREFOOT

INDICATIONS

- For feet requiring stability and offloading after surgery (e.g. osteotomy of the foot, hallux valgus, etc.)



Unisex

Low-profile design so no height compensation is required for the opposite leg

FEATURES

SOLE

- The stiff sole reduces pressure on the heel and forefoot
- The rocker-style sole facilitates rollover when walking
- The square toe design acts as a bumper to protect the toes
- The removeable EVA insole can be adapted or replaced if necessary

UPPER

- Lightweight and breathable
- Heel counter for enhanced stability in the heel area
- Opens fully so is always quick and easy to don
- Pull-back Velcro® strap across the instep with square D-ring and protective cell
- Buckle-free symmetrical fastening over the forefoot

Item	M23001					
Size	XXS	XS	S	M	L	XL
Shoe size	34-36,5	37-38,5	39-41	41,5-43	43,5-45	45,5/47

Sold individually - fits right or left

stepper DIGIT M23002



FOREFOOT OFFLOADING

INDICATIONS

- A postoperative shoe designed to offload pressure from the forefoot



Unisex

A 15° wedge offloads pressure from the forefoot

FEATURES

SOLE

- The square toe design acts as a bumper to protect the toes
- The removeable EVA insole can be adapted or replaced if necessary

UPPER

- Lightweight and breathable
- Heel counter for enhanced stability in the heel area
- Opens fully so is always quick and easy to don
- Pull-back Velcro® strap across the instep with square D-ring and protective cell
- Buckle-free symmetrical fastening over the forefoot

Item	M23002					
Size	XS	S	M	L	XL	
Shoe size	33.5/37	37.5/39.5	40/41.5	42/44	44.5/47	

Sold individually - fits right or left

stepper CAL M23003



HEEL OFFLOADING

INDICATIONS

- Postoperative wear after surgery involving soft tissue and heel bones
- Targeted offloading relieves pressure following fractures
- Aids the healing of ulcers in the heel area



Unisex

Targeted offloading shifts pressure away from the heel

FEATURES

SOLE

- The square toe design acts as a bumper to protect the toes
- The removeable EVA insole can be adapted or replaced if necessary

UPPER

- Lightweight and breathable
- Heel counter for enhanced stability in the heel area
- Opens fully so is always quick and easy to don
- Pull-back Velcro® strap across the instep with square D-ring and protective cell
- Buckle-free symmetrical fastening over the forefoot

Item	M23003					
Size	XS	S	M	L	XL	
Shoe size	33.5/37	37.5/39.5	40/41.5	42/44	44.5/47	

Sold individually - fits right or left





TRACTIONS

OTHER

cervitrac


Cervical traction kit

M3019
M3020



INDICATIONS

- Cervical osteoarthritis
- Cervical discopathies
- Cervicobrachialgia
- Reduction of intervertebral spaces
- Straightening of the physiological lordosis cervical

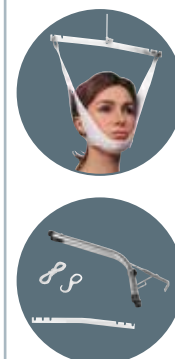

FEATURES

Structure composed of:

- Over door clamp and pulley
- Head support structure, universal, white
- Spreader bar, universal, silver
- Nylon cord
- Hook
- **Only M3019** Linen bag
- **Only M3020** Water bag, universal, max weight 9 kg

Item	M3019	M3020
Size	universal with linen bag	universal with water bag

trac OPTION

Cervical traction components

M3021 / 3024 / 3034



FEATURES

Head harness:

- Soft double-lined foam design
- Brushed cotton inner lining
- Brushed nylon outer lining for attaching Velcro® fasteners
- D-rings for attaching the spreader bar
- Closure with Velcro® straps
- Washable

Item	M3021 (head harness)	M3024 (pulley system)	M3034 (spreader bar)
Length cm	universal	universal	34
Colour	white	silver	silver

sacchetto
water bag
M3022



sacchetto
linen weight
M3023



Item	M3022
Size	universal
Capacity max.	9 kg / 20 lb
Colour	transparent

Item	M3023
Size	universal
Colour	beige

duraband

Elastic natural latex band

M6001 / M6006 5m
M6008 / M6013 30m



INDICATIONS

- Post-surgical rehabilitation

FEATURES

- Elastic natural latex band for resistance and functional rehabilitation



OPTION: M6007

Item	Colour	Strength	
M6001	yellow	light	0,15 mm x 5 m
M6002	orange	moderate	0,2 mm x 5 m
M6003	green	medium	0,25 mm x 5 m
M6004	pink	strong	0,3 mm x 5 m
M6005	purple	super	0,4 mm x 5 m
M6006	silver	extra	0,45 mm x 5 m
M6008	yellow	light	0,15 mm x 30 m
M6009	orange	moderate	0,2 mm x 30 m
M6010	green	medium	0,25 mm x 30 m
M6011	pink	strong	0,3 mm x 30 m
M6012	purple	super	0,4 mm x 30 m
M6013	silver	extra	0,45 mm x 30 m

OPTION: M6007 pair of handles for Duraband

bi-bike

Motorised trainer for passive mobility

M4KM-500



INDICATIONS


- General Rehabilitation
- Gymnastics for the elderly
- Supporting action for venous pump and lymphatic drainage
- Exercise which complements traditional kinesiotherapy in severe forms of chronic and progressive disabilities




FEATURES

- Anti-spasm device with rotation activity block and acoustic signal
- Electronic control device with on-board computer
- 4 levels of pedaling speed adjustment: 15 cycles / min (low) - 30 cycles / min (medium-low) 45 cycles / min (medium-high) - 60 cycles / min (high)
- 5 levels of duration adjustment: 3-6-9-12-15 min
- 2 types of movement selectable: pedaling forwards and backwards, with 2 second pause for change of direction
- Operation with easy-to-fix knobs and pedals
- Adjustable pedalling radius: 13 cm (long user) - 11 cm (average user) - 9 cm (short user) based on the height of the user
- Tachometer
- Calorie counter
- Countdown
- Pre-programmed exercise function
- Electronic display to view data and functions: set time, cycles / minute, real time of use, number of cycles performed, calorie consumption, activation of pre-programmed exercise

Item	M4KM-500
Size	universal
Colour	white

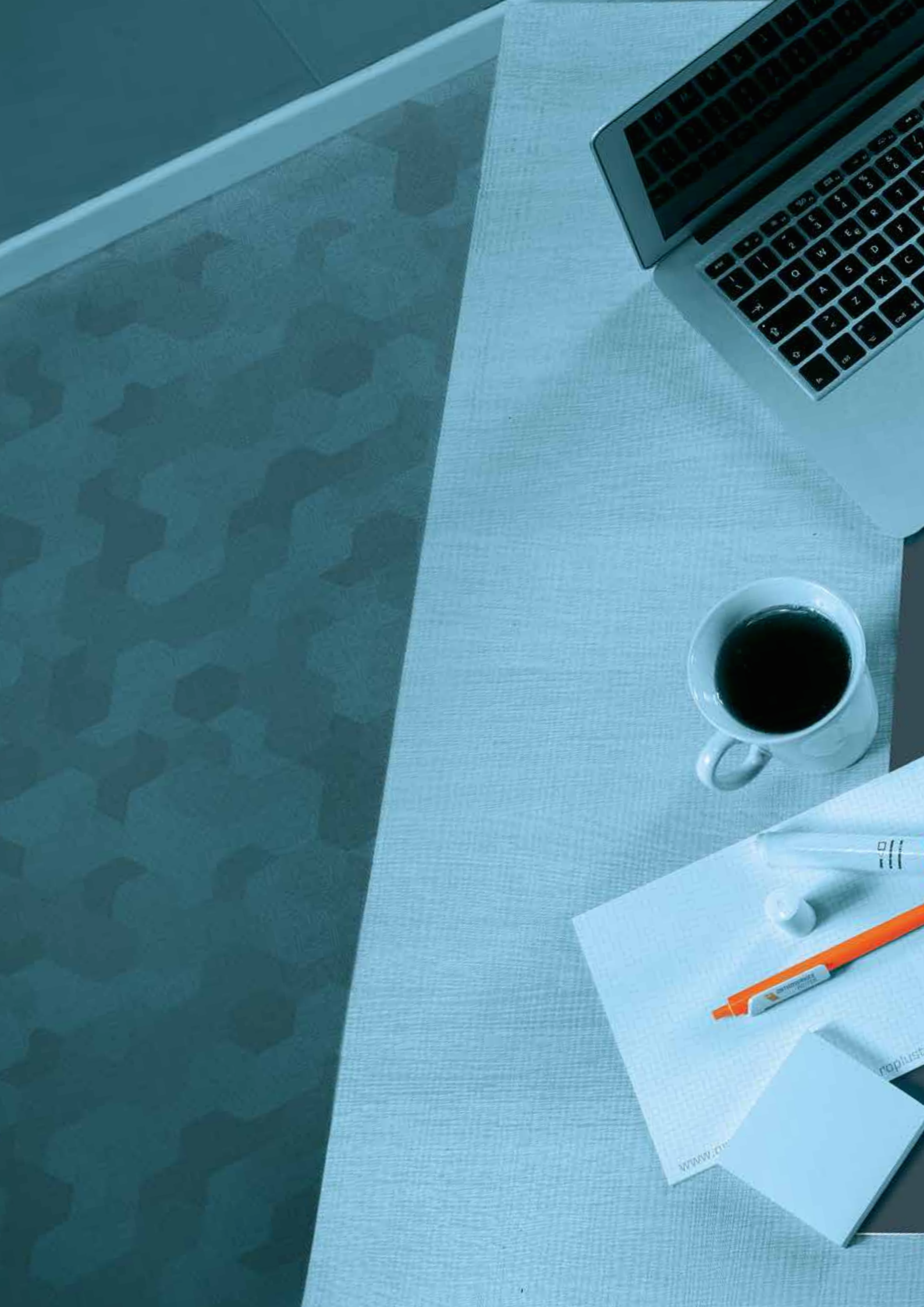

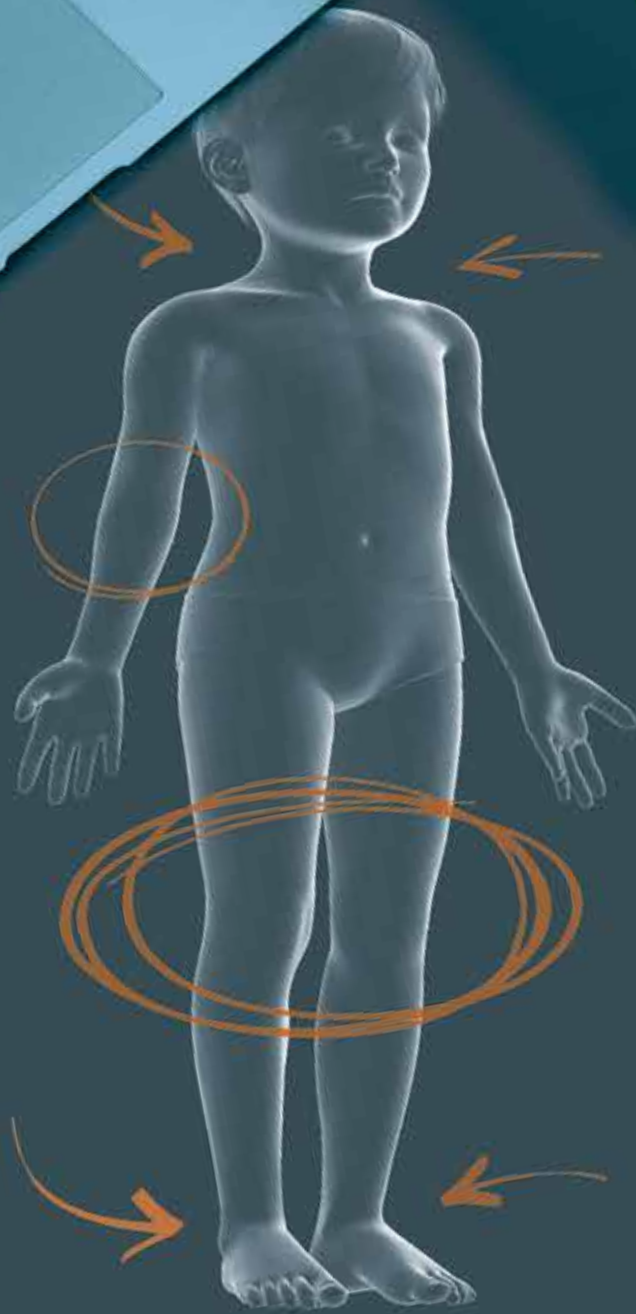

ORTHO 14



ORTHO 14-100

Soft foam cervical collar (paediatric)

MOFT100



INDICATIONS

- Slight cervical spine distortions (whiplash)
- Stiff necks
- Neck-induced vertigo

FEATURES

- Core made of semi-rigid foam
- Tubular jersey cover
- Velcro® fastening at back of neck
- X-ray transparent

Item	MOFT100
Size	universal
Height cm	6,5
Circumf. cm	20/30
Colour	beige

ORTHO 14-105

Bivalve cervical collar with tracheal hole

MOFT105 I/P



INDICATIONS

- Medium severity to severe distortion of the cervical spine (whiplash)
- Multiple discopathies of the cervical spine
- Cervical spine traumas
- Pathological instability as a result of cervical spine metastases
- Support of Post-surgical spinal surgery care

FEATURES

- Non-toxic PLASTAZOTE® structure
- Consists of two enveloping valves with Velcro® closure
- Central front and back rigid support plates
- Radiolucent
- Shaped for perfect fit at chin, neck and shoulders
- Trachea hole

Item	MOFT105I	MOFT105P
Size	Kids 0 – 3 years old	Kids 4 – 9 years old
Colour	beige	

ORTHO 14-305

Immobiliser for arm and shoulder, with elbow support

MOFT305



INDICATIONS

- Compound humerus fracture
- Glenohumeral subluxation
- Shoulder sprain
- Post-surgical care of humeral fractures and interventions on humeral subluxations
- Postoperative care of the long head of the biceps tendon

FEATURES

- Cotton blend fabric sling
- Soft padded fabric immobiliser swathe
- Shoulder straps with Velcro® fastenings
- Elbow support

Item	MOFT305
Size	universal
Length bag cm	23
Height cm	22
Colour	black

fits right and left

ORTHO 14-310

Arm sling

MOFT310



INDICATIONS

- Forearm and wrist support aid
- As support in cases of cast limb
- Useful if resting of the acromioclavicular (after a subluxation grade 1) or glenohumeral joint is required
- Useful in case of immobilisation of the upper arm with the elbow in flexed position

FEATURES

- Structure made of cotton blended fabric
- Velcro® fastening
- Shoulder strap with neck protection cell

Item	MOFT310
Size	universal
Length bag cm	24
Colour	blue

fits right and left

ORTHO 14-315

Clavicle splint

MOFT315



INDICATIONS

- Clavicle fractures
- Subluxation of the acromionclavicular joint

FEATURES

- Made of gauze fabric with cell
- Applied similarly to the figure of eight bandage with Velcro® closure at the back

Item	MOFT315
Size	universal
Colour	white

fits right and left

ORTHO 14-500

Wrist immobiliser

MOFT500 R
MOFT500 L



INDICATIONS

- Wrist sprain
- Compound fracture of the radius and ulna extremities
- Wrist tendonitis
- Consequences of fracture of the wrist or after early removal of the plaster

FEATURES

- Aluminium palm splint, preformed and pliable (removable)
- Double-lined foam design
- Reinforcing dorsal stays
- Fastening with Velcro® straps and buckles

Item	MOFT500 R • MOFT500 L
Size	universal
Colour	blue

specify right or left

ORTHO 14-502

Wrist immobiliser

MOFT502 R
MOFT502 L



WITH THUMB GRIP

INDICATIONS

- Wrist sprain
- Wrist tendonitis
- Consequences of wrist fracture or after early removal of the plaster
- Damage to ulnar section of metacarpophalangea of the thumb

FEATURES

- Aluminum thumb splint preformed and pliable (removable)
- Aluminum palm splint preformed and pliable (removable)
- Double-lined foam cell
- Reinforcing dorsal plastic stays
- Fastening with Velcro® straps and buckles

Item	MOFT502 R • MOFT502 L
Size	universal
Length cm	16
Colour	blue

specify right or left

ORTHO 14-700/2-/3

Knee immobiliser

MOFT700/2 (20 CM)
MOFT700/3 (30 CM)



INDICATIONS

- Immobilisation with knee in extension position ((post-surgical or post-trauma care)

FEATURES

- Structure made of double-lined soft foam
- Internal lining in terrycloth
- Rigid but moldable medial, side and back rods made of light metal alloy
- Adjustable supporting plates, inside and lateral
- Patella opening

Item	MOFT700/2	MOFT700/3
Immobilisation at	0°	0°
Length cm	20	30
Thigh circumference cm	24/35	33/45
Colour	blue	

fits right and left

ORTHO 14-800

Bivalve ankle brace

MOFT800



INDICATIONS

- Grade 2 and 3 sprains
- Functional recovery phase after surgical treatment
- Functional recovery after malleolar fractures
- Chronic instability of TT and sporting activity

FEATURES

- Double-lined foam cell
- Anatomically shaped lightweight durable plastic bivalve structure
- Velcro® adjustment fits any heel width
- Easy to wear thanks to full opening

Item	MOFT800
Size	universal
Colour	black

fits right and left

ORTHO 14-805

Walker

MOFT805



INDICATIONS

- Conservative treatment of malleolar fractures
- Severe sprains grade 3
- Post-surgical phase after osteosynthesis of ankle fractures, stabilisation of ligament injuries of the ankle and Achilles tendon
- Stable foot, ankle, tibia and distal fibula fractures

FEATURES

- Structure made from lightweight, durable plastic
- Non-slip rubber sole
- Inner cell of the sole is made from non-toxic Evazote®
- Struts held in place with Velcro® for a firm grip on the cell
- Soft foam pad with double lining in brushed nylon for Velcro® fastening
- Velcro® fastening straps

Item	MOFT805		
Size	M	L	XL
Shoe size	23/25	26/28	29/31
Height from ground cm	22	25	27
Colour	black		

fits right and left

ORTHO 14-810

Shower for lower leg and foot

MOFT810

M9391 R

M9391 L



INDICATIONS

- Foot drop
- Mild to moderate spastic equinus



M9391 R
M9391 L
semifinished of
MOFT810
specify right or left


FEATURES

- Polypropylene splint
- Can be heated and moulded to fit
- Fastens with a Velcro® strap around the calf

Item	MOFT810 • M9391 R • M9391 L				
Size	1.	2.	3.	4.	5.
Calf circumference cm	15/19	19/23	22/24	23/27	25/29
Foot section length cm	10	11	12	13	15
Splint height cm	16	19	23	25	28
Colour	semitransparent				

specify right or left



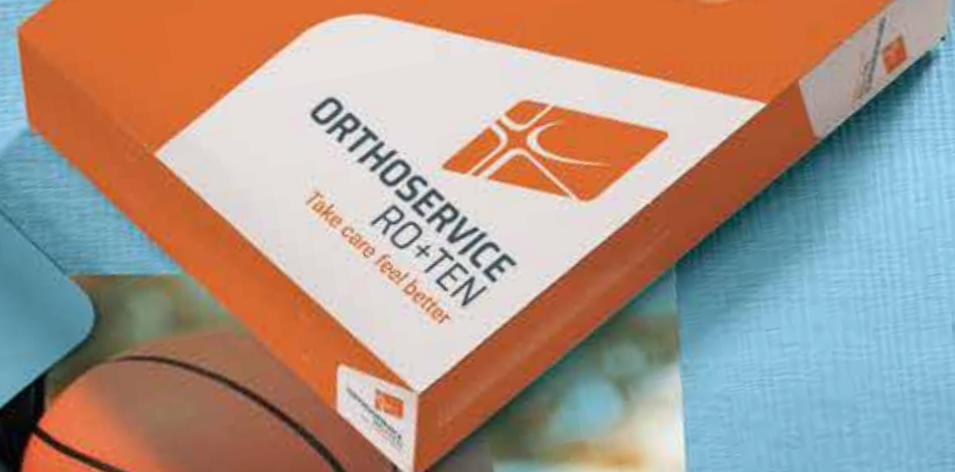

ORTHOSELF

BY ORTHOSERVICE



EPI-S32

Elbow support

M19032



INDICATIONS

- Chronic pain from bursitis or tendinitis

FEATURES

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Composition: 70% polyamide, 30% elastane

Item	M19032				
Size	XS	S	M	L	XL
Circumf. cm*	20/23	23/27	27/31	31/35	35/40

fits right and left / *forearm circumference

MANU-S19

Wrist support

M19019



WITH THUMB SUPPORT

INDICATIONS

- Aftercare for wrist sprains
- Tendonitis and tenosynovitis
- Inflammatory degenerative diseases of the wrist

FEATURES

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Thumb hole
- Composition: 70% polyamide, 30% elastane

Item	M19019				
Size	XS	S	M	L	XL
Circumf. cm*	14,5/15,5	15,5/16,5	16,5/17,5	17,5/18,5	18,5/20

fits right and left / *fits wrist circumference

MIO-S33

Thigh support

M19033



INDICATIONS

- Muscular contractions and cramps of the thigh
- Aid to resuming sporting activity after trauma

FEATURES

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Composition: 70% polyamide, 30% elastane

Item	M19033				
Size	S	M	L	XL	XXL
Circumf. cm*	45/48	48/52	52/56	56/61	61/67

fits right and left / *fits thigh center circumference

GENU-S60

Knee support

M19060



WITH NON-SLIP INSERT

INDICATIONS

- Gonalgia from bruising
- Early stage arthrosis
- Tendinitis

FEATURES

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Nonslip silicone upper insert
- Composition: 70% polyamide, 30% elastane

Item	M19060					
Size	XS	S	M	L	XL	XXL
Circumf. cm*	35/38	38/41	41/45	45/49	49/54	54/60

fits right and left / *high circ. taken 15 cm above mid-patella

MIO-S34

Calf support

M19034



INDICATIONS

- Muscular contractions and cramps of the calf
- Aid to resuming sporting activity after trauma

FEATURES

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Composition: 70% polyamide, 30% elastane

Item	M19034			
Size	S	M	L	XL
Circumf. cm*	33/36	36/39	39/43	43/48

fits right and left / *calf circumference

MALLEO-S37

Ankle support

M19037



SPECIAL WEAVE ON THE HEEL

INDICATIONS

- Bruising
- Tendinopathy of the instep and peronei

FEATURES

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Special weave on the heel for a stronger, more durable designs
- Composition: 70% polyamide, 30% elastane

Item	M19037				
Size	XS	S	M	L	XL
Circumf. cm*	14,5/15,5	15,5/16,5	16,5/17,5	17,5/18,5	18,5/20

fits right and left / *ankle circumference

GENERAL TERMS OF SALES

1. Entire Agreement This document contains all of the terms and conditions relating to the purchase of goods and/or services (collectively, "Goods") by ORTHOSERVICE AG and its affiliates ("Buyer"), from the supplier identified herein ("Seller") and constitutes a legally binding agreement between ORTHOSERVICE AG and Seller ("Agreement"). Buyer and Seller are individually referred to herein as a "Party" and collectively as the "Parties".

2. Orders From time to time Buyer may submit an order ("Order") for Goods with Seller. Any Order submitted by Buyer to Seller shall be subject to and deemed to incorporate the terms and conditions of this Agreement. Unless agreed to in writing by a duly authorized representative of Buyer, Buyer objects to, and is not bound by, any term or condition that differs from or adds to Buyer's Order or the terms and conditions herein. Commencement of performance by Seller or shipment of any part of the Goods covered by an Order shall be deemed to constitute assent to the Order and the terms and conditions herein.

3. Price Buyer shall pay Seller the price for the Goods shown in the Order. The price is firm and not subject to increase without Seller's prior written communication. Price will be displayed in the Order Confirmation of Seller for final approval of Buyer.

4. Taxes Unless specified in an Order, the prices stated do not include sales, use or similar taxes directly applicable to the Goods sold to Buyer. Notwithstanding the foregoing, Buyer shall only be responsible for such taxes when Seller is required by law to collect such taxes from Buyer. Such taxes shall be separately stated on Seller's invoice. Seller shall not add such taxes to Buyer's invoice if Buyer has furnished a valid tax exemption certificate to Seller.

5. Payment Unless otherwise stated in the Supplier Information document attached hereto or different agreement, Buyer shall prepay Seller the price for the Goods. Seller shall send Buyer a detailed invoice for each Order. Buyer shall make payment to Seller by electronic funds transfer.

6. Delivery Seller shall deliver Goods in accordance with the schedule in Buyer's Order. Time is of the essence in the performance of the Order. Buyer may select the mode of transportation and the carrier for the Goods. Seller shall pre-pay shipping charges and add such charges as a separate line item on the invoice, unless otherwise agreed in writing. If Buyer requests that Goods be shipped "collect", Seller shall use a Buyer approved transportation carrier and Buyer shall not pre-pay shipping charges. Seller shall be responsible for excess transportation costs resulting from any deviation from Buyer's shipping instructions. If, prior to the delivery of the Goods, Seller has reason to believe that it will be unable to meet the schedule in Buyer's Order, Seller shall immediately notify Buyer in writing, indicate the cause of delay, and use its best efforts to cure the anticipated delay. Upon receipt of notice of the anticipated delay or upon occurrence of an actual delay in performance, Buyer may (i) direct expedited shipment of the Goods with excess costs paid by Seller, or (ii) cancel the Order. Buyer may return at Seller's expense Goods delivered early to Buyer. Goods shall be delivered by Seller to the delivery location specified in the Order or, if delivery location is not specified in the Order, then to Buyer's place of business from which the Goods were ordered. Risk of loss of the Goods while in transit is regulated by the usual international European regulations.

7. Packing The Goods to be furnished on the Order shall be suitably packed to protect the Goods from damage, prepared for shipment to secure the lowest transportation rates possible (unless a premium shipment method is specified on the Order) and comply with carrier regulations. No charges shall be allowed for packing, crating, expedited shipping or cartage unless authorized on the Order.

8. Returns Seller applies merchandise return policies for defective goods only after Seller investigation and written confirmation, which do not comply with European Community standards regulation, and/or for deliveries of products different from the order. In this case only, the cost of transport is belonging to the Seller. For any other reason for returning the goods, mediation between the seller and the buyer is desirable, but in any case, acceptance of the return remains at the discretion of the Seller and at the buyer's transport costs.

9. Warranties Seller warrants that all Goods delivered pursuant to an Order shall strictly conform to the terms, conditions, specifications, descriptions, drawings and data specified in the Order or furnished herewith and shall be of good design, material, and workmanship, free from defects, merchantable and fit for their intended purpose and shall meet applicable governmental safety standards. Seller further warrants that Buyer will have free and clear title to the Goods furnished on the Order and the right to sell such Goods at the time of delivery to Buyer. All Goods shall be new and unused (unless otherwise specified in the Order) at the time of delivery to Buyer. Seller shall also, without any additional charges, assign or otherwise transfer to Buyer all warranties on Goods and services incorporated into the Goods sold on the Order. All warranties herein mentioned shall survive any intermediate or final inspections, delivery, acceptance or payment by Buyer, and all such warranties shall run to Buyer, its successors, assigns, customers, and users of the Goods. No warranties hereunder shall be deemed disclaimed or excluded except in writing signed by an authorized representative of Buyer. If Buyer deems the Goods defective, Seller, at Buyer's election, shall promptly repair or replace the defective Goods, or give Buyer a credit or refund for the price paid for the defective Goods, at any time within Seller's standard European warranty period for the Goods. Alternatively, Buyer may repair defective Goods at Buyer's facility and charge Seller the reasonable cost thereof. Seller shall be responsible for all shipping charges (both to and from Seller's facility) for Goods which are repaired or replaced by Seller.

10. Order Modification Buyer may at any time, by written notice to Seller, make changes in the specifications, quantities, delivery schedules and shipping instructions in an Order. Seller will notify Buyer within 10 days if any such change increases or decrease Seller's cost of performing the Order or the time required for its performance in which case an equitable adjustment shall be agreed upon by the Parties. Any stenographic or clerical errors of Buyer in any Order are subject to correction by Buyer.

11. Order Termination Buyer may at any time terminate for its convenience Seller's performance of an Order, in whole or in part, by written notice to Seller, whereupon Seller shall terminate its performance on such date of notice. If Seller has commenced performance of the Order, Buyer shall negotiate reasonable termination charges with Seller. Seller shall use best efforts to mitigate its losses as the result of any Order termination by Buyer. Buyer may terminate an Order for default with respect to all or any part of undelivered Goods if Seller (a) does not meet the delivery or performance schedule specified in the applicable Order, (b) breached any terms hereof, including warranties, or (c) becomes insolvent or commits or becomes subject to an act of bankruptcy.

12. Set-Off Buyer may set off any amount Buyer owes to Seller against any amount that Seller owes to Buyer.

13. Infringement Seller shall indemnify Buyer for any losses, costs, damages and expenses (including court costs and attorneys fees) arising from a claim of infringement of any intellectual property right of a third party relating to any of the Goods delivered. Seller shall defend or settle, at its own expense, any lawsuit or proceeding brought against Buyer by a third party for such infringement. Furthermore, in the event Buyer should be enjoined from the use or sale of the Goods as a result of such lawsuit or proceeding, Seller shall pay all Buyer's costs and expenses including

consequential damages and at Buyer's option shall: (a) procure the right for Buyer to continue using and selling the Goods at Seller's expense; (b) replace the Goods with non-infringing Goods at Seller's expense; and/or (c) remove the Goods from Buyer's premises and refund to Buyer the amount paid for the Goods. The provisions of this paragraph shall not apply to any claims, demands, lawsuits or injunctions brought by a third party where the Goods have been manufactured by Seller in accordance with Buyer's specific instructions, specifications, designs or drawings.

14. Indemnification Seller shall indemnify Buyer from and against any and all losses, costs, damages, and expenses (including court costs and attorneys' fees) arising from any claims against Buyer for (i) injury or death to any individual, or damage to property, alleged to have been caused in whole or in part by defective Goods furnished by Seller or by any act or omission, negligence or otherwise, of Seller or any subcontractor of Seller or of any of their respective employees, workmen, or agents, or (ii) Seller's breach of its warranties and obligations hereunder. Seller will be exempted from any indemnification of the Buyer or any injury, death, or loss caused exclusively by the negligence or willful misconduct of Buyer.

15. Regulatory Buyer is a medical device manufacturer and is required to comply with different governmental regulatory requirements. If requested by Buyer, Seller agrees to allow Buyer and/or its representatives to conduct an onsite supplier audit of Seller's facilities in connection with Buyer's regulated operations. Alternatively, Buyer may elect to send Seller a mail-in audit which Seller agrees to complete and timely return to Buyer. Seller agrees to cooperate with Buyer in any such audit or any follow up to such audit.

16. Compliance with Laws In the performance of work hereunder, Seller shall comply with all applicable international, federal, state, and local laws, rules and regulations.

17. Regulation (EU) 2017/745 [MDR] In case of Medical Devices (MD) the Seller agrees to supply only Goods fully complying with MDR 2017/745 (and its modifications) providing, at Buyer's request, all technical documentation (i.e. EU Declaration of Conformity, Reach / RoHS / Oeko-Tex declarations, instructions, extract of the technical dossier, etc) and all other information useful in order to avoid a misuse by the final Consumer and grant fully compliance to all regulatory duties. Signing this contract the Seller confirms he will supply only Goods manufactured without using raw materials, substances of very high concern (SVHC), semi-finished products and/or other components who may constitute an effective risk/danger (individually or as a whole) according to Regulation (EC) 1907/2006, EN ISO 10993, MDR 745/2017 and their modifications. Every single package must be labeled by the Seller accordingly with MDR 745/2017 using the information, layouts and instructions provided by the Buyer. Non complying Goods will be automatically rejected once the Buyer, during the quality checking of incoming goods, has identified them informing immediately the Seller who'll be in charge for quickly activating the restitution process and replace (at his own expense) the products as soon as possible. In this case, the purchasing order will be consider only partially fulfilled so the Buyer will be entitled to ask for an indemnification for any suffered damage.

18. Assignment Seller shall not assign any of its rights or interest in this Agreement or any Order or subcontract its performance of this Agreement or any Order, without Buyer's prior written consent. Any prohibited assignment shall be void.

19. Applicable Law Laws of European Community, without regard to conflict of law principles thereof, will govern this Agreement, and any order issued hereunder. The Parties expressly consent, and submit themselves, to the exclusive jurisdiction of the Switzerland Confederation for adjudication of any claim.

20. Confidentiality Seller shall not, except as required in performance of an Order, disclose or use at any time any information which is not publicly known about Buyer's products, processes or services and which is disclosed to or learned by Seller prior to or during such performance and fully comply to the MNDA. Seller shall, upon demand by Buyer, surrender to Buyer any and all confidential or proprietary documents and other information of Buyer in whatever form which are in Seller's possession or control.

21. Buyer's Information Buyer retains title to all information and materials (including, but not limited to, all drawings, designs, specifications, technical data and production or product "know-how") furnished to Seller to facilitate performance of an Order, and the same shall be (i) treated as Buyer's confidential information, (ii) used exclusively by Seller to complete the Order, and (iii) returned to Buyer at Buyer's direction or upon completion, termination, or cancellation of the Order, along with all copies or reproductions thereof, unless otherwise agreed to in writing by Buyer. Seller shall thereafter make no further use of any such drawings, designs, specifications, technical data and production or product "know-how" or of any information derived therefrom in the performance of work for any other customer without Buyer's prior written consent.

22. Seller's Information Seller retains title to all information and materials (including, but not limited to, all drawings, designs, specifications, technical data and production or product "know-how") furnished to Buyer to facilitate performance of Sales, and the same shall be (i) treated as Seller's confidential information, (ii) used exclusively by Buyer to complete the sales, and (iii) returned to Seller at Seller's direction or upon completion, termination, or cancellation of the Order, along with all copies or reproductions thereof, unless otherwise agreed to in writing by Seller. Buyer shall thereafter make no further use of any such drawings, designs, specifications, technical data and production or product "know-how" or of any information derived therefrom in the performance of work for any other customer without Seller's prior written consent.

23. No Publicity Without Buyer's prior written consent, Seller shall not release any news release or publicity regarding this Agreement, an Order, or the fact that Seller is furnishing Goods to Buyer.

24. Non-Waiver No waiver of any provision of this Agreement or any Order or any right or obligations of either Party shall be effective, except pursuant to a writing signed by the Party waiving compliance, and any such waiver shall be effective only for the specific instance and purpose stated in such writing

25. Final provisions For all that is not regulated in this document, the European regulations in force at the time of conclusion of the contract are valid




ORTHOSERVICE RO+TEN

Take care feel better



www.orthoservice.com

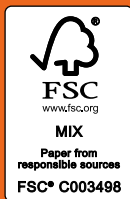

UNI CEI EN ISO 13485:2011
UNI EN ISO 9001:2015



9DCC100027EN005_0424

Headquarter: ORTHOSERVICE AG
Via Milano 7 · CH-6830 Chiasso (TI) · Switzerland
Tel. 0041 (0) 91 822 00 88 · Fax 0041 (0) 91 822 00 89
info@orthoservice.com · www.orthoservice.com



ORTHOSERVICE DEUTSCHLAND GmbH
Flugstraße 8 · D-76532 Baden-Baden · Deutschland
Tel. 0049 (0) 7221 991 39 11 · Fax 0049 (0) 7221 991 39 13
info@orthoservice.de · www.orthoservice.de

ORTHOSERVICE POLSKA Sp. z o.o.
ul. Warszawska 416a · 42-209 Częstochowa · Polska
Tel. +48 (0) 34 340 13 10 · (NIP): 9492246785
info@orthoservice.pl · www.orthoservice.pl

OSUK Ltd
Units 47&50 North East BIC
Wearfield
Sunderland Enterprise Park East
Sunderland SR5 2TA
Tel. +44 (0) 191 51 66 220
info@orthoservice.uk · www.orthoservice.com

RO+TEN S.r.l.
Sede legale: Via Marco De Marchi 7 · I-20121 Milano (MI) · Italia
Sede operativa e amministrativa:
Via Comasina 111 · I-20843 Verano Brianza (MB) · Italia
Tel. 0039 039 601 40 94 · Fax 0039 039 601 42 34
info@roplusten.com · www.roplusten.com

Copyright Orthoservice Ro+Ten April 2024

The descriptions and images in this document are for example and commercial purposes only. Orthoservice reserves the right to modify them according to its needs.

For UK customers:
OSUK Ltd

Units 47&50 North East BIC
Wearfield
Sunderland Enterprise Park East
Sunderland SR5 2TA
Tel. +44 (0) 191 51 66 220
info@orthoservice.uk
www.orthoservice.com

For international customers:
ORTHOSERVICE AG

Via Milano, 7
CH-6830 Chiasso (TI)
Switzerland
Tel. +41 (0) 91 822 00 88
fax +41 (0) 91 822 00 89
info@orthoservice.com
www.orthoservice.com

