

An ally for the unstable shoulder: **Shouldercross**

SHOULDER INSTABILITY CAN BE:

- **Idiopathic:** congenital excess laxity and elasticity of the ligamentous structures and capsule
- **Microtraumatic:** common in athletes at the extremes caused by repetitive athletic movements
- **Post-traumatic:** complete or partial loss of the joint relationship between the humeral head and the glenoid.

The new **Shouldercross**, version is the innovative therapeutic alternative

For preventive or post-traumatic phases

Wearing **Shouldercross** for 2 to 6 hours during "at-risk" sports and work activities helps stabilize the joint: it limits dislocation and subluxation episodes, restoring confidence in the patient.

For painful loose shoulders

Using **Shouldercross** for 2 to 4 months, combined with proper rehabilitation, reduces recovery time and the percentage of necessary stabilization surgeries.

In post-operative phases

Shouldercross allows for a much more tolerable recovery and rehabilitation phase: it assists without immobilizing.

In all cases

The use of **Shouldercross** is strongly recommended while the patient perceives benefit.

9DCDP0014ZEN003.09/24

shouldercross

Functional shoulder brace

NEW

REF.20726 R
REF.20726 L

Size	S	M	L	XL	XXL
Circ. biceps cm	22/26	26/30	30/35	35/40	40/45
Colour	grey				

Specify right or left



Indications

- Conservative treatment of instability after both front and back glenohumeral dislocations
- Protection during high-risk sports or work activities, while awaiting surgery or after stabilization surgery
- Use for shoulder rehabilitation treatment of painful hyperlaxity or functional instability
- Useful after repair of large rotator cuff tears or for conservative management of massive rotator cuff tears
- Useful in the rehabilitation management of scapular dyskinesia.

Contraindications

- Currently none.



www.orthoservice.com



Headquarter: **ORTHOSERVICE AG**
Via Milano 7 · CH-6830 Chiasso (TI) · Switzerland
Tel. 0041 (0) 91 822 00 88 · Fax 0041 (0) 91 822 00 89
info@orthoservice.com · www.orthoservice.com

ORTHOSERVICE DEUTSCHLAND GmbH
Flugstraße 8 · D-76532 Baden-Baden · Deutschland
Tel. 0049 (0) 7221 991 39 11 · Fax 0049 (0) 7221 991 39 13
info@orthoservice.de · www.orthoservice.de

ORTHOSERVICE POLSKA Sp. z o.o.
ul. Warszawska 416a · 42-209 Częstochowa · Polska
Tel. +48 (0) 34 340 13 10 · (NIP): 9492246785
info@orthoservice.pl · www.orthoservice.pl

OSUK Ltd
Units 47&50 North East BIC
Wearfield
Sunderland Enterprise Park East
Sunderland SR5 2TA
Tel +44 (0) 191 51 66 220
info@orthoservice.uk · www.orthoservice.uk

RO+TEN S.r.l.
Sede legale: Via Marco De Marchi 7 · I-20121 Milano (MI) · Italia
Sede operativa e amministrativa:
Via Comasina 111 · I-20843 Verano Brianza (MB) · Italia
Tel. 0039 0362 18 51 200
info@roplusten.com · www.roplusten.com

shouldercross

Functional shoulder brace

NEW



**ORTHOSERVICE
RO+TEN**

Take care feel better



**ORTHOSERVICE
RO+TEN**

Take care feel better

**NEW ADJUSTABLE
ARM GRIP**

The arm grip can be adjusted with a **practical Velcro® closure** to perfectly fit the limb.

NEW



**CROSS-REINFORCED
ELASTIC STRAPS**

Limit the desired movement according to medical prescription:
Position 1 or **Position 2**



**CONTAINMENT ALSO ON
THE POSTERIOR PART**

**BREATHSKILL FABRIC
STRUCTURE**

Soft and breathable thanks to its **special 3D design**.
The external mesh weave provides strength.



**ADJUSTABLE AND
PADDED CONTRALATERAL
FIXING STRAP**

Easy to close with a **spring balancer**.
It stabilizes the orthosis and prevents migration.
The strap can be cut to size, ensuring the padding is placed under the armpit.



Innovative and functional, it stabilizes the shoulder without immobilizing it.